# SELECT

## PROFCARE

Made in China  
SELECT SPORT A/S • Fabriksparken 46 • DK-2600 Glostrup  
select-sport.com

UK

### Back Support with splints - Art. 6411

#### Function:

- Warmth, support and stability
- Suitable as preventive protection against strains and stiffness
- Provides pain-relief for muscles in the back and loin
- Removable splints and straps for extra support
- For protection against minor impacts from falling, protection against bruises, abrasions and light burns

#### Instruction for use:

Place the support as shown on the front of the packaging.

#### Material:

- Lining: 100% Polyamide Jersey
- Foam: 70% SBR Neoprene, 30% CR Neoprene
- Fasteners: Velcro

#### Important information:

- Do not use on patients with a known sensitivity to any of the product components
- The product is not flameproof, avoid temperature above 120°C/248°F.
- The function of the product must not be changed or compromised
- The product should only be used for its original purpose as described above
- Time of use should not exceed 3-4 hours
- If the product contains latex or rubber and you are allergic to latex/rubber, we advise you to consult a prescriber before using the support
- If you are allergic to heat, we advise you to use a product that does not contain SBR Neoprene

To ensure correct fit, try the product on.

#### Washing instructions:

1. Close Velcro
2. Remove splint
3. Mesh laundry bag is recommended
4. Stretch product while wet and let dry

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