Version : November 2021

SELECT

PROFCARE

Classification

It is a class I device by Rule 1 of the European Medical Device Regulations 2017/745.

Medical Device Class I, Rule 1, for non-sterile and non-invasive devices for short term use, by the European Medical Device Regulatory 2017/745
Basic UDI: 5703543U1014LB

Made in China

For further information and inquiries regarding this product, please contact:

SELECT SPORT A/S • Fabriksparken 46 • DK-2600 Glostrup
select-sport.com • (+45) 43 96 96 66 • select@select.dk

UK

Profcare Elastic Bandage II - Style no: 720092/720092/720107

Intended use:

The PROF CARE Elastic Bandage II is intended to be used as support and relief for ankle sprains and distortions. Compression and support of muscles and joints. It is for use on the skin only. Duration of use should not exceed 3-4 hours.

PROFCARE Elastic Bandage II consists of a beige wrap knit elastic fabric, which can be stretched approx. 180%.

The bandages can be washed at 40 degrees Celsius. Comes with 2 fastening clips.

After repeated use, the binding loses some of its elasticity and supportive effect. When washed, the binding becomes resilient again. Do not wear the bandage at night.

Warnings and contraindications:

Do not use the bandage if you have allergies or are sensitive to some of the ingredients. Contraindications:

Use of the product is contraindicated when:

- the skin is broken skin
- Do not stick on the wound directly
- the user Duration of use should not exceed 3-4 hours

Warnings:

Do not apply too tight

Do not use if the user has known allergies to rubber. Do not use the tape if the packaging is damaged or affected with damp and mildew. Consult your Health Care Provider if you experience any reaction. Do not store above 25 degrees Celsius. Store in original packaging to protect from light

Profcare Elastic Bandage II consists of:

- Fabric Material: 70% Polyester, 30 Natural rubber
- Metal clips: 78% aluminium, 12% rubber, 10% polyester

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