Version : November 2021

SELECT

PROFCARE

Classification

It is a class I device by Rule 1 of the European Medical Device Regulations 2017/745.

Medical Device Class I, Rule 1, for non-sterile and non-invasive devices for short term use, by the European Medical Device Regulative 2017/745

Basic UDI:Product: 5703543U1007LE

Made in China

For further information and inquiries regarding this product, please contact:

SELECT SPORT A/S • Fabriksparken 46 • DK-2600 Glostrup

select-sport.com • (+45) 43 96 96 66 • select@select.dk

UK

Fixing tape - Style no: 720057

Intended use:

The Profcare Fixing Tape is a self-adhesive non-woven tape used for fixation of wounds dressings, particularly on joints, frequently moved and contoured parts of the body.

Instructions for use:

Clean and dry the skin before fixing various dressings.

Duration of use should not exceed 3-4 hours.

Warnings and contraindications:

Contraindications:

Use of the product is contraindicated when:

- the skin is broken
- there is swelling at the application site

Do not stick Fixing Tape directly on the wound.

Warnings:

Do not apply too tight

Do not store above 25 degrees Celsius.. Store in original packaging to protect from sun light.

Ensure that the skin is cleaned before application.

Symptoms of overexposure of the acrylic acid may include redness or discolouration, swelling, itching, burning, or blistering.

Consult your Health Care Provider if you experience any reaction.

Material:

Fabric Material: 60% Terylene polyester, 40% Viscose

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