Version : November 2021

#### Classification

Medical Device Class I, Rule 1, for non-sterile and non-invasive devices for short term use, by the European Medical Device Regulatory 2017/745

Basic UDI: 5703543U1005LA

Made in China

For further information and inquiries regarding this product, please contact:

SELECT SPORT A/S • Fabriksparken 46 • DK-2600 Glostrup  
select-sport.com • (+45) 43 96 96 66 • select@select.dk

UK

#### Profcare Stretch Tape - Style no: 720060, 720061

##### Intended use:

The PROF CARE Stretch tape is intended to be used to support the body parts of sports athletes.

This product is intended to be used as a secondary bandage for sports athletes to keep the primary bandage securely in place with sporting injuries.

The tape is adhesive to the skin and offers elastic support.

It is for use on the skin only.

It is for single, short-term use and is a non-sterile device.

##### Instructions for use:

Skin needs to be cleaned before use.

Duration of use should not exceed 3-4 hours.

The tape is latex free.

##### Warnings and contraindications:

Do not use the tape if you have allergies or are sensitive to some of the ingredients.

Contraindications:

Use of the product is contraindicated when:

- the skin is broken skin
- there is swelling at the application site
- do not stick on the wound directly
- the duration of use should not exceed 3-4 hours
- skin needs to be cleaned before use.
- the user has known allergies to rubber

##### Warnings:

Do not apply too tight.

Ensure that the skin is cleaned before application.

Do not use the tape if the packaging is damaged or affected with damp and mildew.

Consult your Health Care Provider if you experience any reaction.

Do not store above 25 degrees Celcius. Store in original packaging to protect from light.

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