Version: November 2021

SELECT

PROFCARE

Classification

Medical Device Class I, Rule 1, for non-sterile and non-invasive devices for short term use, by the European Medical Device Regulatory 2017/745

Basic UDI: 5703543U1006LC

Made in China

For further information and inquiries regarding this product, please contact:

SELECT SPORT A/S • Fabriksparken 46 • DK-2600 Glostrup
select-sport.com • (+45) 43 96 96 66 • select@select.dk

UK

Foam tape - Style no: 720058

Intended use:

The PROFCARE Foam tape is intended to be used as a protective layer between tape and skin and enables strapping tape removal to be painless and prevents rubbing and irritation. It is for use on the skin only. It is for single, short-term use and is a non-sterile device.

Instructions for use:

Skin needs to be cleaned before use.
Duration of use is maximum 3-4 hours

Warnings and contraindications:

Do not use the tape if you have allergies or are sensitive to some of the ingredients

Contraindications:

Use of the product is contraindicated when:

- the skin is broken
- there is swelling at the application site
- the user has known allergies to rubber

Warnings:

Do not apply too tight.
Ensure that the skin is cleaned before application.
Do not store above 25 degrees Celsius. Store in original packaging to protect from sun light.
Symptoms of overexposure of the acrylic acid may include redness or discoloration, swelling, itching, burning, or blistering.
Consult your Health Care Provider if you experience any reaction.

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