SELECT

PROFCARE

Classified as category 1
EU 2016/425
Conformity Declaration: this product meets the regulation
EU 2016/425.

Made in China
SELECT SPORT A/S • Fabriksparken 46 • DK-2600 Glostrup
select-sport.com

UK

Knee Support with splints - Art. 6204

Function:

- Warmth and support
- Preventive protection against tenderness around the kneecap
- Increase stability
- For protection against minor impacts from falling, protection against bruises, abrasions and light burns

Instruction for use:

Use the Velcro-straps to tighten the bandage around the knee to ensure the right compression and improve stability. The aluminium splints can be adjusted slightly at the ends by bending them.

Material:

- Lining: 100% Polyamide Jersey
- Foam: 70% SBR Neoprene, 30% CR Neoprene
- Fastener: Velcro
- Splint: Aluminium

Important information:

- Do not use on patients with a known sensitivity to any of the product components
- The product is not flameproof, avoid temperature above 120°C/248°F.
- The function of the product must not be changed or compromised
- The product should only be used for its original purpose as described above
- Time of use should not exceed 3-4 hours
- If the product contains latex or rubber and you are allergic to latex/rubber, we advise you to consult a prescriber before using the support
- If you are allergic to heat, we advise you to use a product that does not contain SBR Neoprene

To ensure correct fit, try the product on.

Washing instructions:

1. Close all Velcro
2. Remove splint
3. Mesh laundry bag is recommended
4. Stretch product while wet and let dry

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