Version : November 2021


SELECT

PROFCARE

Classification
Medical Device Class I, Rule 1, for non-sterile and non-invasive devices for short term use, by the European Medical Device Regulatory 2017/745

Basic UDI: 5703543U1009LJ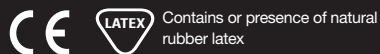

Made in China

For further information and inquiries regarding this product, please contact:

SELECT SPORT A/S • Fabriksparken 46 • DK-2600 Glostrup
select-sport.com • (+45) 43 96 96 66 • select@select.dk

UK

Profcare Plaster - Style no: 720053

Intended use:

The PROF CARE Plaster is intended to be used to cover injuries and wounds. To protect the wound and scab from contamination and friction to aid in the healing process. It is for use on the skin only. It is for single, short-term use and is a non-sterile device.

Instructions for use:

Skin needs to be cleaned before use. Duration of use should not exceed 3-4 hours, change the plaster if necessary.

Warnings and contraindications:

Do not use the tape if you have allergies or are sensitive to some of the ingredients. Contraindications: Use of the product is contraindicated when:
- there is swelling at the application site
- do not stick on the wound directly
- the duration of use should not exceed 3-4 hours, change the plaster if necessary
- Skin needs to be cleaned before use.
- has known allergies to rubber

Warnings:

Do not apply too tight
Ensure that the skin is cleaned before application
Do not use the tape if the packaging is damaged or affected with damp and mildew.
Contains or presence of natural rubber latex
Consult your Health Care Provider if you experience any reaction.
Do not store above 25 degrees Celsius. Store in original packaging to protect from light.

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