SELECT

PROFCARE

Classified as category 1
EU 2016/425
Conformity Declaration: this product meets the regulation
EU 2016/425.

Made in China
SELECT SPORT A/S • Fabriksparken 46 • DK-2600 Glostrup
select-sport.com

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Shoulder Support - Art. 6500

Function:

- Warmth and support
- Provides pain-relief around the shoulder joint
- Suitable for overloaded shoulder and rehabilitation
- For protection against minor impacts from falling, protection against bruises, abrasions and light burns

Instruction for use:

Ensure the support is placed comfortable on the shoulder and around the arm to make the right compression and stability. Use the Velcro-straps to ensure a comfortable fit. Consider using the SELECT Compression T-shirt underneath the support.

Material:

- Lining: 100% Polyamide Jersey
- Foam: 70% SBR Neoprene, 30% CR Neoprene
- Fasteners: Velcro

Important information:

- Do not use on patients with a known sensitivity to any of the product components
- The product is not flameproof, avoid temperature above 120°C/248°F.
- The function of the product must not be changed or compromised
- The product should only be used for its original purpose as described above
- Time of use should not exceed 3-4 hours
- If the product contains latex or rubber and you are allergic to latex/rubber, we advise you to consult a prescriber before using the support
- If you are allergic to heat, we advise you to use a product that does not contain SBR Neoprene

To ensure correct fit, try the product on.

Washing instructions:

1. Close all Velcro
2. Mesh laundry bag is recommended
3. Stretch product while wet and let dry

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