Version : November 2021

# SELECT

## PROFCARE

### Classification

Medical Device Class I, Rule 1, for non-sterile and non-invasive devices for short term use, by the European Medical Device Regulatory 2017/745

Basic UDI: 5703543U1004L8

**Caution: This Product Contains Natural Rubber Latex Which May Cause Allergic Reactions.**

Made in China

For further information and inquiries regarding this product, please contact:

SELECT SPORT A/S • Fabriksparken 46 • DK-2600 Glostrup  
select-sport.com • (+45) 43 96 96 66 • select@select.dk

UK

Proficare Stretch Extra Bandage - Style no: 720062, 720063

### Intended use:

The PROFCARE Stretch Extra Bandage is intended to be used for compression and support of muscles and joints. The bandage is self-adhesive, sticks to itself and does not stick to skin. It is for use on the skin only. It is for single, short-term use and is a non-sterile device. Duration of use should not exceed 3-4 hours

### Warnings and contraindications:

Do not use the bandage if you have allergies or are sensitive to some of the ingredients  
Contraindications:  
Use of the product is contraindicated when:  
- the skin is broken  
- do not stick on the wound directly  
- the duration of use should not exceed 3-4 hours  
- skin needs to be cleaned before use.

### Warnings:

Do not apply too tight.  
Do not use if the user has known allergies to rubber.  
Do not use the tape if the packaging is damaged or affected with damp and mildew.  
Consult your Health Care Provider if you experience any reaction  
Do not store above 25 degrees Celsius. Store in original packaging to protect from light.

### Proficare Stretch Extra Bandage consists of:

- Fabric Material: 95% Cotton, 5% Spandex / weight 230 g/m<sup>2</sup>
- Glue: Latex adhesive (natural rubber latex)
- Component: 60% fabric, 40% adhesive

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