Version : November 2021

SELECT

PROFCARE

Classification

Medical Device Class I, Rule 1, for non-sterile and non-invasive devices for short term use, by the European Medical Device Regulative 2017/745

Basic UDI: 5703543U1002L4

Made in China

For further information and inquiries regarding this product, please contact:

SELECT SPORT A/S • Fabriksparken 46 • DK-2600 Glostrup
select-sport.com • (+45) 43 96 96 66 • select@select.dk

UK

Profcare Stretch Soft Tape - Style no: 720065, 720066

Intended use:

The PROF CARE Stretch soft tape is intended to be used to support the body parts of sports athletes. The tape is an adhesive compression bandage tape. The tape is adhesive to the skin and offers elastic support. It is for use on the skin only. It is for single, short-term use and is a non-sterile device. Duration of use should not exceed 3-4 hours.

Warnings and contraindications:

Do not use the tape if you have allergies or are sensitive to some of the ingredients.

Contraindications:

Use of the product is contraindicated when:

- the skin is broken skin
- there is swelling at the application site
- do not stick on the wound directly
- the duration of use should not exceed 3-4 hours
- skin needs to be cleaned before use.
- allergies to rubber are known

Duration of use should not exceed 3-4 hours.

Warnings:

Do not apply too tight.

Ensure that the skin is cleaned before application.

Do not use the tape if the packaging is damaged or affected with damp and mildew.

Consult your Health Care Provider if you experience any reaction.

Do not store above 25 degrees Celsius. Store in original packaging to protect from light.

Profcare Stretch soft tape consists of:

- Fabric Material: 100% Cotton / weight 115-125 g/m²
- Glue: Hot melt Zinc Oxide adhesive/ 115-125 g/m²
- Component: 60% fabric, 40% adhesive

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