Version : November 2021

SELECT

PROFCARE

Classification

It is a class I device by Rule 1 of the European Medical Device Regulations 2017/745.

Medical Device Class I, Rule 1, for non-sterile and non-invasive devices for short term use, by the European Medical Device Regulatory 2017/745.

Basic UDI: 5703543U1003L6

Made in China

For further information and inquiries regarding this product, please contact:

SELECT SPORT A/S • Fabriksparken 46 • DK-2600 Glostrup
select-sport.com • (+45) 43 96 96 66 • select@select.dk

UK

Proficare Supreme Tape - Style no: 720103, 720104

Intended use:

The PROFICARE Supreme tape is intended to be used to support the body parts of sports athletes.

Is intended to prevent injury or reinjury by stabilizing body parts in sports athletes. To prevent and treat injuries by immobilization.

This is a non-elastic tape.

It is for use on the skin only.

It is for single, short-term use and is a non-sterile device.

Instructions for use:

Fix the top of the tape. Keep a constant traction for the first circle. Overlap the next circle with half width on the first circle. To assure good stickiness, let the final circle fully cover the previous circle. Keep a constant pressure with the palm for a few second on the end of the tape. Skin needs to be cleaned before use. Duration of use should not exceed 3-4 hours.

Warnings and contraindications:

Do not stick on the wound directly

Contraindications:

Use of the product is contraindicated when:

- the skin is broken
- there is swelling at the application site
- the user has known allergies to rubber
- the product contains Rosin

Warnings:

Do not apply too tight.

Do not store above 25 degrees Celsius. Store in original packaging to protect from sun light.

Do not use it if the packaging is damaged or affected with damp and mildew.

Ensure that the skin is cleaned before application.

Symptoms of overexposure of the acrylic acid may include redness or discolouration,

swelling, itching, burning, or blistering.

Consult your Health Care Provider if you experience any reaction.

Material:

- Fabric material: 100% woven rayon: 110-120 g/m2

- Glue: Hot melt adhesive with a composition of: 50% synthetic rubber, 30% Zinc oxide, 20% Resin 110-130 g/m2

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