# SELECT

## PROFCARE

Classified as category 1  
EU 2016/425  
Conformity Declaration: this product meets the regulation  
EU 2016/425.

Made in China  
SELECT SPORT A/S • Fabriksparken 46 • DK-2600 Glostrup  
select-sport.com

UK

### Wrist Support - Art. 6700

#### Function:

- Warmth, support and compression
- Provides relief when wrist is under heavy strain
- Double velcro closure ensures optimal support
- For protection against minor impacts from falling, protection against bruises, abrasions and light burns

#### Instruction for use:

Place the support as shown on the front of the packing. Placing the 3.8 cm Velcro-closure closest to the wrist will increase the support and reduce the flexibility. Placing the 2.5 cm Velcro-closure closest to the wrist will increase flexibility and reduce support.

#### Material:

- Lining: 100% Polyamide Jersey
- Foam: 70% SBR Neoprene, 30% CR Neoprene
- Fasteners: Velcro

#### Important information:

- Do not use on patients with a known sensitivity to any of the product components
- The product is not flameproof, avoid temperature above 120°C/248°F.
- The function of the product must not be changed or compromised
- The product should only be used for its original purpose as described above
- Time of use should not exceed 3-4 hours
- If the product contains latex or rubber and you are allergic to latex/rubber, we advise you to consult a prescriber before using the support
- If you are allergic to latex, we advise you to use a product that does not contain SBR Neoprene

To ensure correct fit, try the product on.

#### Washing instructions:

1. Close all Velcro
2. Mesh laundry bag is recommended
3. Stretch product while wet and let dry

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