QUICK USE GUIDE

HOW TO INSTALL

1. Slide the silicone cushion over the breast shield body, wrapping around the edge of the breast shield body, and then make sure both fit snugly.
2. Insert the diaphragm cup into the round hole at the upper end of the three-way vacuum cleaner, make sure that the edge of the diaphragm cup's groove covers the edge of the round hole of the three-way vacuum cleaner, and both are tightly fitted, then put the diaphragm cap and tighten it.
3. Put the duckbill valve into the bottom outlet of the breast shield body to ensure that the installation is in place without air leakage.
4. Screw in the breast shield body assembly into the wide-mouth baby bottle.
5. Install one or bilateral silicone cushion as required, and insert the tubing connector of the tubing into the diaphragm cap and the relative position of the host respectively to ensure the installation in place without air leakage. (Remove the dust cover when using the product, and put the dust cover back for better storage.)

PREPARATION BEFORE USE

1. Charge fully before using the product for the first time.
2. Before each use, all parts in direct contact with breast milk should be cleaned and sterilized in advance: wide-mouth baby bottle, silicone cushion, breast shield body, duckbill valve and baby bottle lid. And make sure that each component is clean without water droplet, the breast shield body connects with the host tightly.
3. Before milk suction, please use a wet hot towel to clean, apply hot compress and massage the breast and areola, and relax the body to relieve swelling and milk pain.
4. During milk suction, relax, sit in a comfortable place, and lean forward slightly (use a pillow to support your back).
5. In the process of milk suction, if it is difficult to pump milk, it is recommended to relax the body and try again. If you feel unwell, stop using and consult medical staff immediately.
6. It is recommended that a single use is for no more than 30 minutes.

KEY DESCRIPTION

Mode key: short press to switch modes between lactation, pumping, deep suction, lying suction.

Lactation mode: massage the breast, faster to stimulate the milk reflex. Adjustable suction intensity / frequency.

Pumping mode: imitate the pace and volume of the baby sucking, better milk discharge. Adjustable suction intensity / frequency.

Deep suction mode: increase the duration of suction, better discharge of the after-milk. Adjusts the suction intensity.

Lying suction mode: need to match with lying suction accessories for realizing lying suction. Adjusts the suction intensity.

Custom mode key: long press to start the operator program memory recording, custom mode icon goes into flashing. After the operation, long press the button again to finish recording, custom mode icon goes off.


After recording, short press the custom mode key to begin suction with the last recorded operation program, and the custom mode icon is steady on.

QUICK USE GUIDE


HOW TO USE

1. Long press the power key to turn on power. After boot, the default setting is bilateral function, lactation gear 1, frequency gear 3. 30 minutes countdown begins.
2. Select the corresponding functions (left, right, bilateral) and mode (lactation, pumping, deep suction, lying suction). Adjust the suction intensity / frequency.
3. After milk suction, long press the on / off button to shut down.


1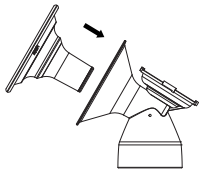


2

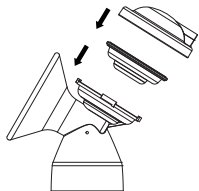
3

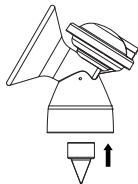4

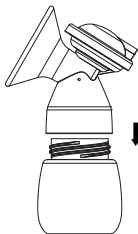
5

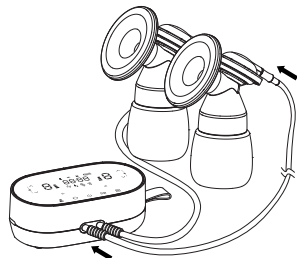Note: Please double check that all the parts are properly assembled before using.

FLANGE FOR DOUBLE ELECTRIC BREAST PUMP QUICK USE GUIDE

COMPONENT DESCRIPTION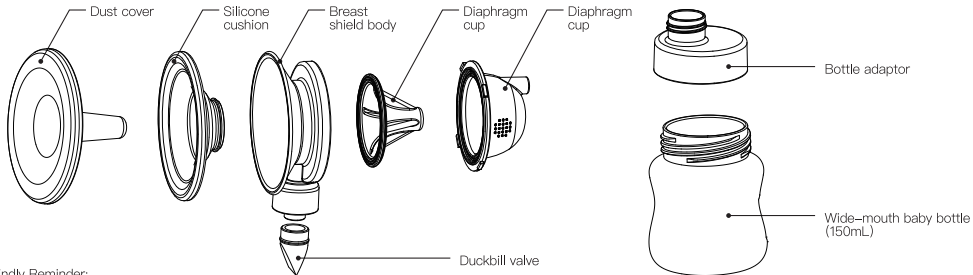


Kindly Reminder:

1. Before first use and after each use, disassemble and hand wash all parts of the pump set that have come into contact with the breast and breast milk. It can be sterilized by steam, ultraviolet light, or by boiling for 2–3 minutes. Sufficient water should be filled during boiling and sterilization to prevent the parts from contacting the bottom and wall of the pot.
2. Wash your hands thoroughly before handling the pump.

HELPFUL ADVICE


1. When pumping, please press the breast shield body tightly, and do not touch the bottle, so as to avoid the movement of the bottle, which may cause the breast shield body not to fit tightly to the breast;
2. Don't lie on your one side when lying down, keep 45° supine, and the supine position should not be lower than 45°;
3. Put a tissue under the breast before you take the breast shield body away, and you can wipe your breast with the tissue to prevent the breast milk from flowing down.

HOW TO USE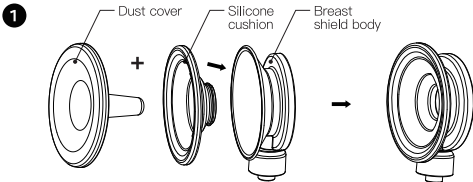


Push the dust cover onto the silicone cushion, and push the silicone cushion onto the breast shield body as far as it will go.

Dust cover, Silicone cushion, Breast shield body.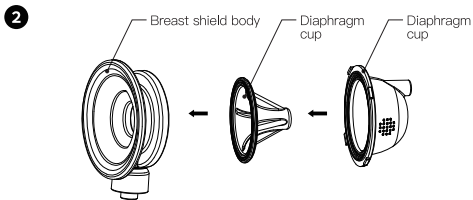

Push the diaphragm cup down into breast shield body as far as it will go and push diaphragm cap over the top as far as it will go.

Breast shield body with silicone cushion, Diaphragm cup, Diaphragm cap.

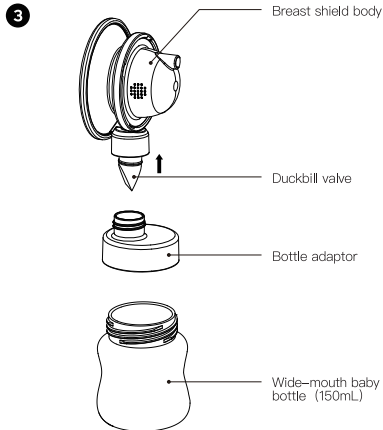Push the duckbill valve onto the bottom of breast shield body, and push the bottle adaptor onto the duckbill valve as far as it will go.

Then screw the wide-mouth bottle onto the bottle adaptor.

Breast shield body with other parts, Duckbill valve, Bottle adaptor, Wide-mouth bottle.

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