


Easy steps to 'ProxTalking'

Press a **Sound** tag on any button. The ProxTalker will recognise the tag and produce the stored sounds. The tags can be placed and pressed in sequence or individually.

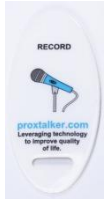

Record New Sounds

-**press** the **Record** Programming tag on any button

-next **press** the **Sound** tag to be recorded*
(*press and release sound tag for 2 second recording or

*press and hold sound tag for up to 8 second recording)

-follow the verbal instructions to record sound

n.b. **You cannot record different sounds on two tags with the same code**



Erase Sounds

-**press** the **Erase** Programming tag on any button

-next **press** the **Sound** tag you want to erase
The tag will have reverted to original sound which may be blank



Change Volume

- **press** the **Increase** or **Decrease** Programming tag on any button to change the volume


A double bleep indicates top or bottom of volume range



Check Battery

- **press** the **Battery** Programming tag on any button

The ProxTalker will tell you how much battery life is left e.g. '95%'

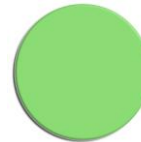

Speak All

Placing this function tag on any button will result in all tags on the ProxTalker buttons being played automatically in sequence from left to right.

visit www.proxtalker.com to download free tag making resources