



GoodSleepco™

How to clean your device

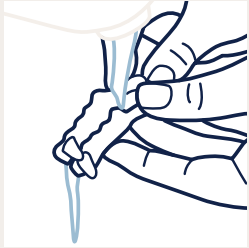


Cleaning Step 1

- **Rinse**
Rinse device with water.

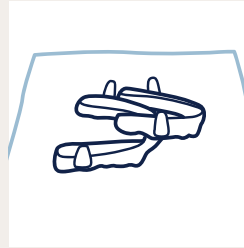


Cleaning Step 5

- **Dry**
Allow to dry.

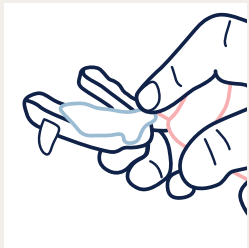

Cleaning Step 2

- **Use Cleaner**
Dispense Cleaner foam on device (ensure you clean both upper and lower arches if using Hushd Pro).

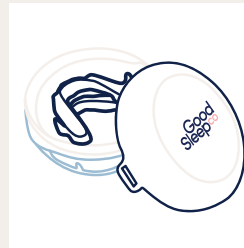


Cleaning Step 6

- **Store**
Place device in storage dish until next use.




Cleaning Step 3

- **Brush**
Gently brush foam on device with a toothbrush.

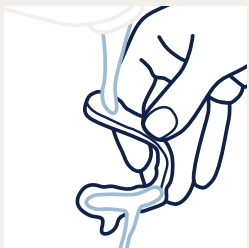


Cleaning Step 4

- **Rinse**
Rinse foam off with water.

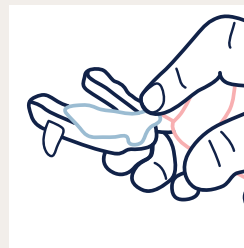

Before bed

- **Cleaner in mouth**
To enhance teeth whitening, dispense Cleaner foam into the device (upper and lower arch) and then put in your mouth with the foam in.

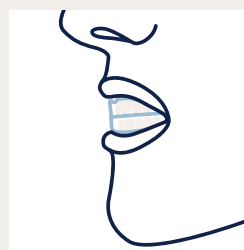

Ordering

- **Ordering Cleaner is easy**
To restock your Cleaner supply, order through the App now.



If you would like more information, please contact us:

hello@goodsleepco.health