



There is not an official sizing guide for the DreamWear Mask frame. Most customers will find a Medium Frame fits. Although, smaller individuals and some women may consider a Small Frame.

To get a measurement estimate measure from the edge of the nose where it meets the cheek, up the cheek and over the head (between the forehead and the crown of the head) to the other edge of the nose.

- Length of the Small Frame: 19 in
- Length of the Medium Frame: 20 in
- Length of the Large Frame: 22 in

**Instructions:**

- 1 Print out this Adobe PDF File at 100% on your printer. Use the print accuracy check at right to ensure printing at the proper size.
- 2 Fold the bottom part of this page on the dashed line.
- 3 Cut along the dotted line around the fitting gauge.
- 4 Place the fitting gauge on your nose as shown in the picture below.
- 5 Determine the appropriate size by which area your nose reaches. (S – Small, M – Medium or L - Large)


Philips Respironics always recommends consulting a physician or registered Sleep Therapist in fitting a mask to ensure a proper sizing and fit.

**Print accuracy check**



**Fitting gauge**

CUT along dotted line

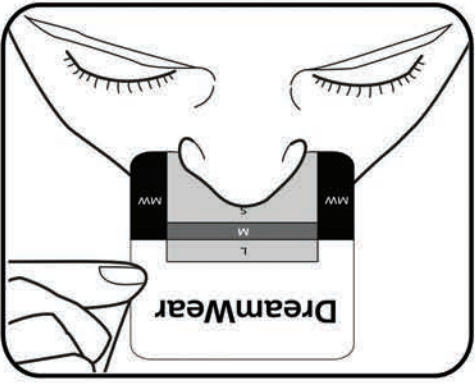

FOLD along dashed line

