



Determining your best fit for the Quattro™ FX / Quattro™ FX For Her Mask

Measure from a point at the top of the nasal bridge down to the indentation below your lips and above your chin as shown in the illustration above.

Use the measurements below as a guide to find your best size.

Please note that every user is different. These sizes should be used as a guide only.

Large Quattro FX	4 3/8 inches	111 millimeters
Medium Quattro FX / Quattro FX For Her	4 inches	101 millimeters
Small Quattro FX / Quattro FX For Her	3 9/16 inches	90 millimeters