

## **Chronic Wound Treatment Protocol**

1. Debride the wound and clear of necrotic tissue.
2. Hydrate the wound bed with BioClense by spraying entire wound bed area.
3. Apply BioRelease over entire wound. BioRelease should be cooled to below 70F, so it goes into the wound bed water thin. BioClense will gel at body temperature. BioRelease needs to go below the surface (water thin state) and then cover wound bed (as a gel) to the thickness of a quarter (coin).
4. For first 2-3 weeks of treatment, apply AGFresh over the BioRelease to a thickness of a quarter (coin). This will reactivate the wound's innate immune response and re-mineralize the wound bed.
5. Cover with secondary dressing.
6. Change dressing twice a week. Flush wound bed with DI or sterile water. Debride as required. Repeat treatment protocol.