

**MCCORD**

# AGFRESH™

## Wound & Burn Dressing

### Description

AgFresh is an antimicrobial wound dressing that releases ionic silver known to protect the wound from pathogens. It is dispensed with a non-ionic hydrogel that maintains a moist environment and helps dissolve wound biofilms. AgFresh contains Fentonite that supports the preservation of silver by chelating and binding chloride known to block the activity of silver. The dressing is non-cytotoxic.

### Ingredients

Ionic silver, glycerine, Fentonite, poloxamer, phenoxeythenol, ocnidine, water, propanediol

### Indications for Use

AgFresh is intended for the management of wounds such as stage I-IV pressure ulcers, partial and full thickness wounds, diabetic foot and leg ulcers, post surgical wounds, first and second-degree burns, grafted and donor sites.

### Contraindications

AgFresh should not be used if there is a hypersensitivity to any of the ingredients

### Warning and Precautions

AgFresh can be used during pregnancy as allowed by healthcare provider. Avoid contact with eyes. No specific drug interaction studies have been undertaken; therefore use with other products should follow healthcare providers instructions. Do not use with saline solution. Do not use if sensitive to silver. **Use Caution if using in combination with any of the following ingredients (Hypochlorous acid, carbomer, acrylates/C10-C30 alkyl acrylate crosspolymer, dentin, bovine serum albumin, copolymer M239144, sodium lauryl sulfate, heat-killed microbes, triethanolamine, bark cork, sodium hypochlorite, EDTA, ethanol, and andnystatin)** as it may change the effectiveness of the antimicrobial properties.

[https://www.researchgate.net/publication/304365219\\_Evidence\\_of\\_incompatibility\\_for\\_topical\\_anionic\\_agents\\_used\\_in\\_conjunction\\_with\\_chlorhexidine\\_gluconate\\_A\\_systematic\\_review](https://www.researchgate.net/publication/304365219_Evidence_of_incompatibility_for_topical_anionic_agents_used_in_conjunction_with_chlorhexidine_gluconate_A_systematic_review)

Reference note: This list is from the Chlorhexidine review, which has the same properties as our products.

### Instructions For Use

Fold AgFresh pouch in half on fold line and tear off top as shown. Squeeze product into the wound. AgFresh should be applied directly to the wound and then covered with the approximate dressing as recommended by healthcare provider. Cover the entire wound as shown by healthcare provider.

### Storage Conditions

Product should be stored between 65F and 85F

Manufactured by  
McCord Research, Inc.  
580 N Madison  
Ave Suite 4  
North Liberty, IA 52317  
[www.mccordresearch.com](http://www.mccordresearch.com)



	Manufacturer		Keep Dry
	Catalog Number		Caution
	Batch Code		Temperature Limits
	Expiration Date		Don't Use If Package Is Damaged
	Prescription Required		Keep Away From Sunlight
	Medical Device		Consult Instructions