


Mecca Care 32mm Grab Rail 300/450/600/900/1200mm

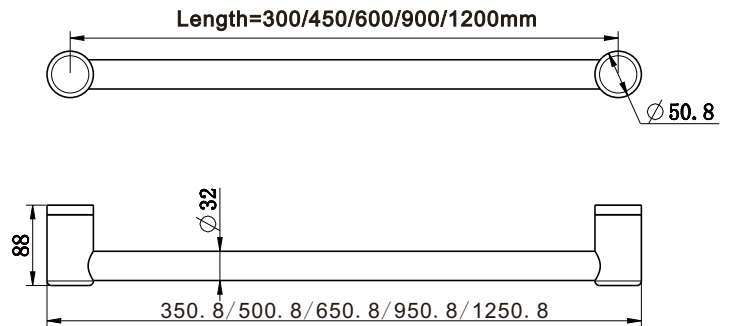

Available Colour



CHROME



MATTE BLACK



BRUSHED NICKEL



GUN METAL



BRUSHED GOLD



BRUSHED BRONZE

SKU

300MM LENGTH

NRRCR3212CH/NRRCR3212MB/NRRCR3212BN/NRRCR3212BG
NRRCR3212GM/NRRCR3212BZ

450MM LENGTH

NRRCR3218CH/NRRCR3218MB/NRRCR3218BN/NRRCR3218BG
NRRCR3218GM/NRRCR3218BZ

600MM LENGTH

NRRCR3224CH/NRRCR3224MB/NRRCR3224BN/NRRCR3224BG
NRRCR3224GM/NRRCR3224BZ

900MM LENGTH

NRRCR3230CH/NRRCR3230MB/NRRCR3230BN/NRRCR3230BG
NRRCR3230GM/NRRCR3230BZ

1200MM LENGTH

NRRCR3248CH/NRRCR3248MB/NRRCR3248BN/NRRCR3248BG
NRRCR3248GM/NRRCR3248BZ

Features

1. Australian owned and designed.
2. Sizes: 300 -1200mm lengths as standard.
3. Modular design, the rail to be mounted horizontally, vertically or at any angle and cut down to suit your bathroom.
4. 150kg capacity for all 25mm and 32mm diameter grab rails.
5. 32mm diameter rails are approved for standard AS1428.1 requirements in DDA bathrooms.
6. Gun metal, brushed gold and brushed bronze colours are PVD coated for long term durability.



MECCA CARE
DDA CARE 32MM

Installation Instruction

	<p>1. Locate wall studs, install into at least one stud. Place wall seal/mounting bracket on wall at correct height over centre of stud and mark hole positions on wall (minimum 2 supplied screws per bracket).</p>
	<p>2. Drill pilot holes into wall stud approximately 60mm deep to suit mounting screws (and wall plugs if required).</p> <p>Tip: 3mm hole for screws into timber or 8mm hole for wall plugs.</p>
	<p>3. Apply small amount of silicon to mounting bracket and wall in and around the screw holes.</p>
	<p>4. Install mounting brackets by screwing onto wall.</p> <p>Tip: Use appropriate mounting screws for the wall stud material, supplied screws are for timber wall studs.</p>
	<p>5. Slide rail tube into end bracket and push onto wall brackets.</p> <p>Tip: Rail length can be cut down to a smaller size, simply use an angle grinder, hacksaw or pipe cutter then deburr the end with a file or grinder and reinsert into the end bracket.</p>
	<p>6. Tighten grub screws with allen key.</p> <p>Tip: If rail rattles, loosen end bracket, twist slightly and tighten.</p>

WARNING:

These rails are designed to be mounted onto wall studs or masonry walls. Installation is recommended to be completed by qualified trades person who complies with the AS 1428.1-2009 design for access and mobility standard.

It is recommended rail location is determined by a qualified occupational therapist or trained professional to the AS 1428.1-2009 standard. Correct installation is required to ensure safe, long term operation. Incorrect mounting can result in reduced capacity of the rail which can result in injury to user and damage to the wall.