




First Handheld Device to Optimize Regenerative Serum Absorption


Description

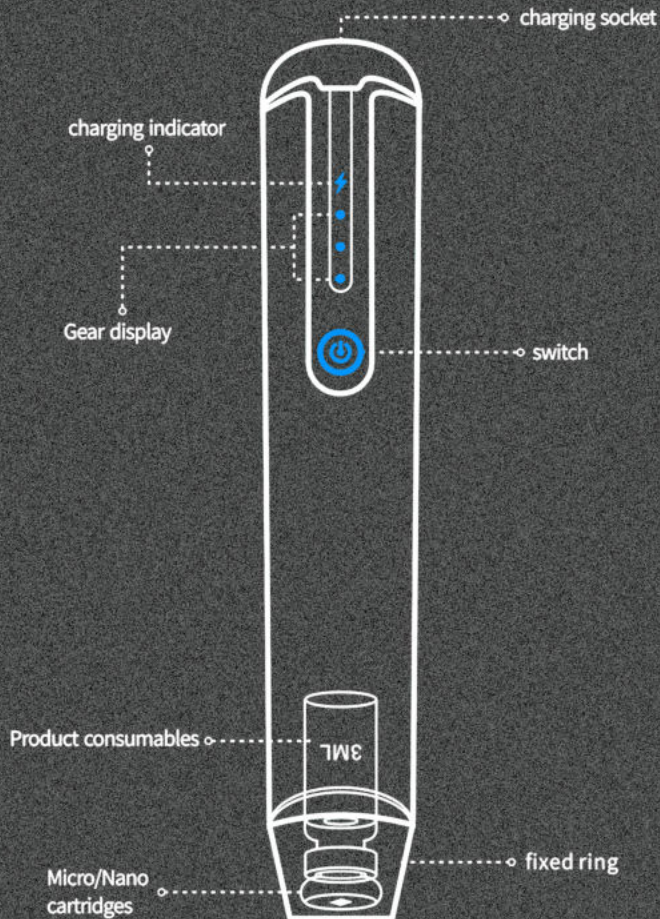

How to operate?



We have prepared the skin care essence for you, which is more convenient to use

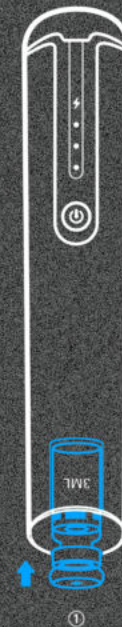

①
Insert the micro/nano chip consumable into the instrument inlet



②
Install the retaining ring and ready to use.



Power on: long press for 2 seconds
Switch gears: click
Power off: press and hold for 3 seconds

In order to protect the instrument, do not turn it upside down to prevent liquid from flowing back into the instrument.



Scan here for face and scalp instructions, protocols + tips

JUVASONIC TREATMENT STEPS

- 📶 **Step 1: Client Consultation & Skin Assessment**
- 📶 **Step 2: Pre-Treatment Preparation**
- 📶 **Step 3: Juvasonic Dry Skin Preconditioning**
- 📶 **Step 4: Juvasonic Serum Infusion**
 - Load the JUVASONIC single use cartridge
 - Apply serum to the preconditioned skin-vertical, horizontal and circular passes on each area for up to 60 seconds.
 - Use gentle circular motions to enhance serum absorption into the dermis.
 - Massage serum into the skin post JUVASONIC.
 - Finish with a MASKAD Peptide Bomb and HA moisturizer - hydrates and plumps skin
- 📶 **Step 5: Advanced Treatment Enhancements**
 - RED LED Therapy to boost skin tightening Boosts collagen synthesis and improves elasticity.
 - BLUE LED Therapy for Acne. Kills acne bacteria and reduces inflammation
- 📶 **Step 6: Post-Treatment Finishing**
 - Apply a soothing moisturizer or post-procedure barrier cream.
 - Massage in SPF 50+ to protect the skin from UV exposure.

