## Enora® Instructions for Use

Recovery and relief, powered by air

Before use, please read the following instructions. Should you have any queries, please visit [enorcare.com](http://enorcare.com) or contact Frontier Therapeutics Limited at +44 (0) 330 460 6050

Enora® and Repose® are Trademarks of Frontier Therapeutics Limited.

### Intended Purpose

Enora® utilises Repose® technology, a range of air-filled, reactive air, pressure redistribution, and reduction support surfaces with high levels of immersion and envelopment available for use in non-healthcare settings.

Immersion is the depth to which the body penetrates a support surface, and envelopment is the ability of a support surface to conform to the body's shape. Movement alters where this immersion and envelopment occurs. Repose's multi-way stretch and elasticity allow interface pressures to equalise across its surface.

The combination of unique materials and Smart Valve Technology have been clinically and scientifically proven to significantly reduce the occurrence of bed sores as well as shorten their healing time through its ability to reduce peak and average interface pressures (5.2%, 4.8%, 5.1%, 5.2%).

- Enora Air Cushion is intended for static chair usage
- Enora Air Contur is intended for use with riser recliner chairs.
- Enora Air Seat is intended for general-purpose wheelchairs fitted with armrests.
- Enora Air Sleeper is intended as a sleep aid placed on top of a mattress.
- Enora has several cover options, which provide additional durability, infection control, and enhance comfort.
- Enora is supplied with an Air Pump unless otherwise stated.

### Indications and Target Group

Enora is intended for immobile and less mobile persons at risk of developing bed sores. Enora is designed to prevent all categories of bed sores and can be used as an effective treatment as part of an overall bedsores treatment program as instructed by your healthcare professional.

### Contra-indications

Enora is unsuitable for anyone with unstable spinal fractures or requiring posture management, such as those with pelvic obliquity and persons with irregular body shapes that Enora cannot fully support.

### Maximum User Weight

Repose technology is subject to maximum user weight restrictions

Product	Weight Limit
Enora Air Sleeper	When used on a high specification foam mattress – 222kg When used on a non-high specification foam mattress – 139kg
Enora Air Cushion Enora Air Seat Enora Air Contur	139kg

### Intended Users

Enora is intuitive and easy to use and can be used in localities and on furniture where additional pressure reduction support is required. There are no special skills needed to install, operate or use Enora.

### Warnings and Precautions

Before use, check that Enora is free from any damage. If there are any visible signs of damage, do not use. Discard and seek a replacement from the manufacturer.

If you observe a change in skin condition, seek advice from a healthcare professional immediately.

During installation, the Repose valve should always be outside and away from the user and never come in contact with skin.

After the first inflation, check Enora regularly and 'top-up' with air every 14 days, using the Air Pump supplied and with the user off the device.

Ensure Repose is correctly fitted into the Enora cover.

Enora Air Cushion, Air Seat and Air Contur should only be used on chairs fitted with armrests.

Please check the position of Enora on the intended support surface regularly to ensure its continued correct and safe use.

Only use the Enora Air Pump to inflate and top up. Replacements are available at [www.enorcare.com](http://www.enorcare.com).

Do not modify any part of the medical device or accessories in any way.

The inflatable Repose inner must not be machine washed. Do not immerse and avoid fluid ingress into the valve.

Do not use abrasive cloths when cleaning, as it may cause irreversible damage.

Neither Repose nor the covers are suitable for sterilisation.

Keep away from babies and small children when deflated, as it could pose a risk of suffocation.

Keep away from pets.

Not to be used as a sling or flotation device.

Keep out of direct sunlight, heat, hot surfaces, sparks, open flames, and other ignition sources.

### Instructions for Use

#### Set-up and Inflation

Remove Repose inner from the Air Pump and re-assemble.

Connect the Air Pump to the valve on the device – push it fit and begin inflation. Continue until you hear a click; this signals that correct inflation pressure has been achieved.


Disconnect and cover the valve with the solid end of the Repose sealing cap.

Remove the Enora cover from its packaging and insert the inflated Repose. The valve and the cover zip should be on the same side.

When installing, place Enora with the valve facing down and away from the user. Where present, use the fixing straps to secure it in place.

Product	Comment
Enora Air Sleeper	Use securing straps on top and bottom of the overlay underneath the mattress it is being placed on
Enora Air Seat	i. Loop both upper fixing straps over the chair or wheelchair handles. ii. Loop the lower fixing strap around the back of the chair or wheelchair under the arm rests, clipping together and adjusting until secure.
Enora Air Contur	i. Insert the fixing straps between the seat and the arm rest, looping around the back of the chair, cross them and clip in place at the top. ii. Loop the bottom straps around either side of the foot rest.

To ensure Enora is inflated optimally, check regularly and 'top-up' with the Air Pump every 14 days, with the user off the device.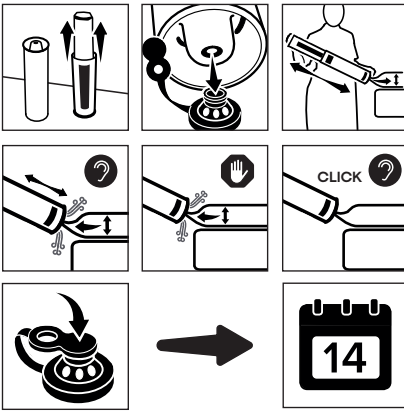

### Deflation and Storage


With the user off Enora, detach all straps and remove.

Clean as per instructions.

Once dry, remove the inner from the cover, detach the sealing cap and insert the open end into the valve, allowing air to escape.

Roll up once deflated, with the cap still in place, starting at the point furthest away from the valve, expelling any remaining air. Repeat this process as necessary.

Fold, roll up and place inside the Air Pump for safe storage.

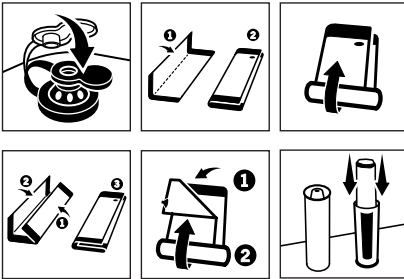### Cleaning Enora and Repose

With regular maintenance, Enora is reliable and long-lasting. These instructions help the user maximise the longevity of Enora but in no way guarantee the product's lifetime. The owner is responsible for regularly inspecting and checking Enora to ensure it performs optimally.

### Enora Covers

Regularly wipe down the cover with warm soapy water or cleaning wipes for routine care. As required, covers can be removed and machine washed up to 60°. Covers should be thoroughly dried before being refitted.

### Repose Inflatable Inner

Washing with warm soapy water or commonly used cleaning wipes is sufficient to clean Repose. Please note that the Repose Inner MUST NOT be machine washed. For ease, cleaning when the Repose inner is inflated is recommended.

### End-of-life disposal

At the end of its life, clean according to the instructions and dispose of as non-hazardous waste.

### Packaging and Storage

Enora is supplied pre-packaged and should be intact upon receipt. Before use, carefully check all components for completeness and signs of damage. Damaged devices should not be used and should be returned to Frontier Therapeutics Limited. Store in a cool, dry place and away from direct sunlight. Do not store next to radiators or other sources of heat.

### Handling

Improper use or handling may lead to damage and possible malfunction. Enora should be checked to ensure it is in working order before use. Do not use damaged or non-working devices.

### Manufacturers Guarantee

Frontier Therapeutics Limited warrants Enora for 12 months against manufacturers' defects. The warranty does not include general wear and tear or failure by other means.

### Product Complaints

Any user who has complaints or experiences dissatisfaction with the device's quality, identity, durability, reliability, safety, effectiveness, or performance should notify the distributor or Frontier Therapeutics Limited.

If any Frontier Therapeutics Limited device ever malfunctions and is suspected of having caused or contributed to the death or severe injury, Frontier Therapeutics Limited, your physician, distributor, and your local competent authority should be notified immediately by telephone or written correspondence. When filing a complaint, provide the component(s) name, reference code, LOT number(s), your name, address, the nature of the complaint, and notification of whether a written report from the distributor is requested.

### Compliance

Enora conforms to UK MDR 2002 and MDR 2017/745 and the following standards: BS EN ISO 9001:2015 Quality Management Systems. BS EN 13485:2016 Medical Devices - Quality Management Systems - Requirements for Regulatory Purposes. BS EN 14971:2019 Medical Devices - Application of Risk Management to Medical Devices. BS EN ISO 15223-1:2021 Medical Devices - Symbols to be Used with Medical Device Labels, Labelling, and Information to be Supplied - General Requirements. BS EN ISO 10993-5:2009 Biological Evaluation of Medical Devices - Tests for In Vitro Cytotoxicity. BS EN 10993-10:2013 Biological Evaluation of Medical Devices - Tests for Irritation and Skin Sensitisation.

### Fire Testing

Product	Standards
Enora Air Seat	BS 5852 : Part 1 - Sch. 4 & Sch. 5 EN 1021 - 1 & 2 16 CFR 1640 TB 117-2013
Enora Air Sleeper	BS 7177 - Source 0 & 1 BS EN 597 parts 1 & 2 16 CFR 1632 TB 117-2013
Enora Air Contur	BS 5852 : Part 1 - Sch. 4 & Sch. 5 EN 1021 - 1 & 2 16 CFR 1640 TB 117-2013
Enora Air Cushion	BS 5852 : Part 1 - Sch. 4 & Sch. 5 EN 1021 - 1 & 2 16 CFR 1640 TB 117-2013

Neither Enora nor Repose incorporate a substance as an integral part, which, used separately, may be considered a medicinal product as defined by European Communities Medicinal Products Directive (Directive 2001/83/EC, as amended) and Human Medicines Regulation 2012 (SI 2012/1916). Nor do they incorporate as an integral part any substance or human blood derivative as defined in point 10 of Article 1 European Communities Medicinal Products Directive (Directive 2001/83/EC, as amended), nor is it manufactured utilising tissues of animal origin as defined by the same directive. Enora is Latex free.

At the time of publication, the Enora range, including Repose, does not contain any substances over 0.1% by weight described on the Candidate List of Substances of Very High Concern for Authorisation - EC Regulation 1907/2006 – REACH.

### Appendix 1

EC REP	LOT	REF		
European Authorised Representative	Batch Code	Product Code	Importer	Distributor
Country of Manufacturer	Fragile	Refer to User Manual	Caution	Postnet Information Website
Medical Device	Unique Device Identifier	Recycle	CE Mark	UKCA Mark
ABS	Keep Away From Sharp Objects	Max User Weight	60°C Wash	Do Not Hand Wash
	Keep Away From Naked Flames			

### Appendix 2

Product Code	Product Description
8002000	Enora Uplifting Blue Air Cushion & Enora Air Pump
8003000	Enora Uplifting Blue Air Contur & Enora Air Pump
8004000	Enora Uplifting Blue Air Sleeper & Enora Air Pump
8005000	Enora Uplifting Blue Air Seat & Enora Air Pump
8002001	Enora Glowing Green Air Cushion & Enora Air Pump
8003001	Enora Glowing Green Air Contur & Enora Air Pump
8004001	Enora Glowing Green Air Sleeper & Enora Air Pump
8005001	Enora Glowing Green Air Seat & Enora Air Pump
8002002	Enora Relieving Peach Air Cushion & Enora Air Pump
8003002	Enora Relieving Peach Air Contur & Enora Air Pump
8004002	Enora Relieving Peach Air Sleeper & Enora Air Pump
8005002	Enora Relieving Peach Air Seat & Enora Air Pump
8002003	Enora Luminous Yellow Air Cushion & Enora Air Pump
8003003	Enora Luminous Yellow Air Contur & Enora Air Pump
8004003	Enora Luminous Yellow Air Sleeper & Enora Air Pump
8005003	Enora Luminous Yellow Air Seat & Enora Air Pump

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