 **ceramill® a-temp**
 **ceramill® a-temp multilayer**

DE Gebrauchsanweisung	3 - 17
EN Instruction Manual	18 - 32
FR Instructions d'utilisation	33 - 48
IT Istruzioni per l'uso	49 - 63
ES Instrucciones de uso	64 - 78

- Original Gebrauchsanweisung -

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- Translation of the original Operating Instructions -

Table of Contents

Explanation of Symbols	19
General Safety Instructions	22
Intended Users	22
Properties	23
Warranty/Exclusion of Liability	25
Testing the Blanks	25
Mounting to the Worktable	26
Application	26
Process steps for producing the restoration in the dental laboratory	27
Process steps for placing the restoration in the patient's mouth in the dental practice	30
Information to be passed on to the patient	31
Environmental Protection	32
Download information	32

Explanation of Symbols

Warning indications

Warning indications in the text are marked with a triangle and boxed.

In case of hazards through electricity, the exclamation mark in the warning triangle is substituted by a lightning bolt.

Signal words at the beginning of a warning indication specify the type and severity of the consequences, if the measures to avert the hazard are not adhered to.

- _ **NOTE** means that property damage can occur.
- _ **CAUTION** means that light to fairly serious personal injury can occur.
- _ **WARNING** means that serious personal injury can occur.
- _ **DANGER** means that life-threatening personal injury can occur.

Important information

Important information that do not lead to hazards for humans or to property damage are marked with the icon aside and are boxed.

Other symbols in the Manual

Symbol	Meaning
▷	Item of an operation description
–	Item of a list
▪	Subitem of an operation description or a list
[3]	Numbers in square brackets refer to position numbers in graphics/figures

Other symbols on the product

Symbol	Meaning
	Medical device
	Unique Device Identifier
	Catalog number
	Batch code
	Manufacturer
	Country of manufacture
	Consult instructions for use
	Use by YYYY-MM-DD or YYYY-MM
Rx only	Symbol that may be used in place of statement “CAUTION”. US Federal law restricts this device to sale by or on the order of a dentist.
	Serial number

General Safety Instructions

EN

**CAUTION:**

Health impairment due to dust from PMMA!

When processing constructions made of Ceramill A-Temp/A-Temp ML, dusts can develop that may lead to mechanical irritation of the eyes and respiratory tract.

- ▷ Always make sure that the extraction system for the milling machine and the workplace for individual reworking operates properly.
- ▷ When processing, wear personal protective equipment (dust protection mask, safety glasses/goggles, ...).
- ▷ Further safety-relevant information can be found in the safety data sheet.

**NOTE:**

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Intended Users

**NOTE:**

The product may only be processed by professionally trained dental technicians/dentists.

Properties

Intended use

Ceramill A-Temp/A-Temp ML are ready to use, polymethyl methacrylate based CAD-CAM blanks, for milling long-term temporary crowns and bridges and for verifying the fit on the plaster model/mouth before the final restoration is manufactured.

Product description

Ceramill A-Temp/A-Temp ML are tooth-coloured milling blanks. These replace lost tooth substance in permanent dentures in the form of temporary crowns and bridges in the anterior and posterior region. The maximum wearing period is one year.

Intended patient group: Suitable for patients of all ages and gender.

Clinical benefit of the product:

- _ Protects the prepared tooth stumps (dentine and pulp)
- _ Maintains the bite position
- _ Promotes the health of the gums
- _ Restores aesthetics and function
- _ Serves the dentist and patient to assess the future form and function of the final restoration

Safety data sheet/Declaration of conformity/SSCP

The safety data sheet and the SSCP of the products (according to the requirements of the Medical Device Regulation (MDR)) can be downloaded from the Amann Girrbach website under *Services > Downloads > Additional documents*. The declaration of conformity of the products are available upon request from the manufacturer.

Technical data

	Unit	Ceramill A-Temp/A-Temp ML (DIN EN ISO 10477)
3-point bending strength	MPa	> 135
Density	g/cm ³	1.19
Vickers hardness	HV0.2	24
Water absorption	μg/mm ³	< 25
Solubility	μg/mm ³	< 0.6
Residual monomeric content	%	< 1

Chemical composition

Polymethyl methacrylate	PMMA	> 98.83%
Methyl methacrylate	MMA	< 1%
Colour pigments		< 0.17%

Shelf life of the blanks

When appropriately stored, the blanks can be used 4 years after the manufacturing date.

Storage

- _ Store only in original packaging.
- _ Store separated from foodstuffs.
- _ Keep container tightly sealed.

Frequency of use

As an intermediate product, the respective product is not intended for single use. The final product (temporary denture) is patient-specific.

Warranty/Exclusion of Liability

Application-technical recommendations, whether given orally, in writing or in the course of practical training, are guidelines. Our products are subject to continuous further development and are tested according to the state of the art and to legal requirements. We reserve the right to make any resulting changes in handling and composition.

The respectively valid instruction-manual version can be found under www.amanngirrbach.com/instruction-manuals. This version replaces all previous versions.

Testing the Blanks

Upon receipt, it is imperative to visually check the proper condition of the blanks. After using a (transport-)damaged blank, complaint claims are no longer possible.

Mounting to the Worktable

The mounting of the blanks is described in the Operating Instructions of the respective milling machine.

We recommend the following CAD/CAM systems for processing our product: Ceramill Mikro 4X, Ceramill Mikro 5X, Ceramill Motion 2, Ceramill Matik.

Application

Indications

- _ Temporary anterior and posterior crowns with a maximum wearing period of one year
- _ Temporary anterior and posterior bridges with a maximum of two connected pontics and a maximum wearing period of one year
- _ Checking the fit on the plaster model/mouth prior to the final restoration

Contraindications

- _ Bridge constructions with more than two connected pontics
- _ Use for definitive restorations
- _ Known incompatibilities with the components
- _ All indications not listed under "Indications"

Undesirable side effects/residual risks

- _ Biological complications (e.g. plaque accumulation, intolerance reactions due to residual monomer, marginal/secondary caries, loosening of the abutment teeth) resulting in possible loss of the restoration.
- _ Mechanical failure (e.g., fracture of the restoration) with low risk of possible swallowing or reversible mucosal injuries

Process steps for producing the restoration in the dental laboratory

Material-specific frame parameters

The following material-specific frame parameters must be observed when fabricating the restorations with Ceramill A-Temp/A-Temp ML:

Minimum wall thickness in mm		Connector cross-section in mm ²			
		Front-tooth bridge		Posterior bridge	
Occlusal	Circular	with one intermediate unit	with two intermediate units	with one intermediate unit	with two intermediate units
1.5	0.8	12	12	12	16

Shade selection and placement of the restoration in the Ceramill A-Temp ML

The tooth shade of the blank covers off the imprinted area (e. g. 0/A1). When a higher blank is selected, the respective lighter or darker shade can be achieved more significantly by vertically moving in the nesting. The closer the work is positioned to the stained side, the larger the brighter shade component.

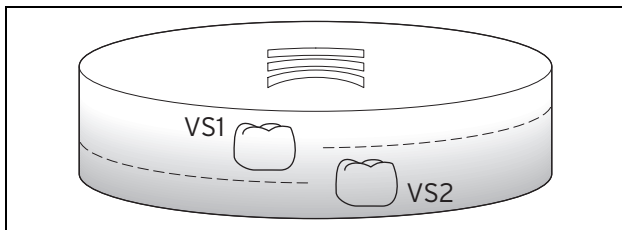
Fig. 2 Placement of the restoration in the blank

VS1 Tooth shade 1 (lighter)

VS2 Tooth shade 2 (darker)

Cutting out and processing

- ▷ Use cross-toothed tungsten carbide cutters or cutting discs suitable for plastic, for cutting out and processing constructions from the blank.
- ▷ Carefully polish crown margins, occlusal surfaces and basal contact surfaces to a high gloss.
- ▷ To avoid fitting inaccuracies after the construction has been manufactured: Prevent intense heat generation during processing and polishing.

Veneering

Milled constructions made of Ceramill A-Temp/A-Temp ML can be veneered with conventional veneering materials (e.g. light-curing veneering composite “Signum” made by Kulzer).

- ▷ Prefer materials based on MMA.
- ▷ Follow the manufacturer's instructions of the veneering-material supplier.

When veneering with composite, the connection to Ceramill A-Temp/A-Temp ML must always be made according to the instructions of the respective manufacturer.

Relining

Constructions made of Ceramill A-Temp/A-Temp ML can be relined at any time with all commercially available cold-curing polymers based on MMA.

- ▷ Condition the surface beforehand, preferably with a comparable bonding agent on the basis of MMA.
- ▷ Follow the manufacturer's instructions of the relining-material supplier.

Process steps for placing the restoration in the patient's mouth in the dental practice**Fixation**

- _ Provisional, eugenol-free cements/fixing materials
- _ Temporary cementing (e.g. "TempoCemNE" made by DMG)
- _ Permanent adhesive cementing (e.g. "Variolink Esthetic" made by Ivoclar Vivadent)

The fixation of Ceramill A-Temp/A-Temp ML must always be carried out according to the instructions of the respective manufacturer.

Information to be passed on to the patient

(e.g. by healthcare professionals).

Possible complications and residual risks for PMMA-based restorations in general and not specific to restorative materials from Amann Girrbach include:

- _ Biological complications (e.g. plaque accumulation, intolerance reactions due to residual monomer, marginal/secondary caries, loosening of the abutment teeth) resulting in possible loss of the restoration.
- _ Mechanical failure (e.g., fracture of the restoration) with low risk of possible swallowing or reversible mucosal injuries

The material is approved for temporary use only (wearing period max. 12 months).

Environmental Protection

Packaging

In terms of packaging, the manufacturer participates in country-specific recycling systems, which ensure optimal recycling.

All packaging materials used are environmentally-friendly and recyclable.

Disposal of the blanks

Dispose of the contents/container in accordance with local/regional/international regulations.

Always dispose of according to official regulations.

Download information

Further instructions/manuals can be downloaded under www.amanngirrbach.com/instruction-manuals.

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Amann Girrbach AG
Amann Girrbach GmbH