ceramill[®] map fx

DE	Betriebsanleitung	2 – 28
EN	Instructions for Use	29 – 55
FR	Mode d'emploi	56 – 82
IT	Istruzioni per l'uso	83 – 109
ES	Manual de uso	110 – 136
PT	Manual de instruções	137 – 163

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– Translation of the Original Instructions for Use –

Table of Contents

1	Explanation of Symbols	30	7	Scanning	42
2	Technical Data	31	7.1	General Information	42
3	Scope of Delivery	32	7.2	Scanning a Jaw Model.....	43
4	General Safety Information	32	7.2.1	Jaw Model on Splitex plate	43
4.1	Intended Use	32	7.2.2	Jaw Model on Universal Plate.....	43
4.2	Scannable Materials.....	32	7.3	Jaw Models in Simple Occlusion	44
4.3	User Qualification	33	7.4	Scanning Articulator.....	45
4.4	Protection against Injuries.....	33	7.4.1	Articulator Workflow.....	45
4.5	Protection against Damage to Property	34	7.4.2	Rotary Swivel Unit.....	45
5	Components	35	7.4.3	Placing the Articulator.....	46
5.1	Scanner	35	8	Multi-Die (Pro Upgrade)	46
5.2	Accessories	36	8.1	Product Description.....	46
5.2.1	Universal Plate.....	36	8.2	Usage	47
5.2.2	3D Calibration Model	37	9	Maintenance and Cleaning	48
6	Commissioning	37	9.1	Maintenance.....	48
6.1	Workplace.....	37	9.2	Cleaning	49
6.2	Unpacking.....	38	10	Fault and Repair	49
6.3	Connecting	38	10.1	Customer Service.....	49
6.3.1	Checklist Connections.....	38	10.2	First Help.....	50
6.3.2	Procedure.....	39	10.2.1	Base Check	50
6.3.3	Attaching the Rotary Swivel Unit.....	39	10.2.2	Failed Calibration.....	51
6.4	Installing	40	10.2.3	Hardware Problems	51
6.4.1	Requirements	40	10.2.4	Connectivity Problems	51
6.4.2	Installation	40	10.2.5	Scan Problems	52
6.4.3	Updates	41	11	Paid Bundles	53
6.5	Calibration	41	11.1	Bundle Activation	53
6.5.1	Placing the 3D Calibration Model	41	12	Disposal	54
6.5.2	Calibration Procedure	41	12.1	Disposing of Packaging	54
			12.2	Disposing of the Device.....	54
			13	CE Conformity	55

1 Explanation of Symbols

Symbols on the Device

Warning against electrical voltage

Conformity of the scanner with EU directives

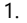
USB connector

Hollow plug connector

Symbols in these Instructions

Characters	Signal Word	Meaning
	Notice Danger Warning Caution	Warning against possible: - Damage to property - Injuries (severe – moderate) - Injuries (minor) - Danger to life
		Action
		Simple list
		Numbered list
		Cross-reference (link)
		Important note

2 Technical Data

EN

Housing

Dimensions W × H × D	337 × 347 × 322 mm
Weight	9.5 kg
Rotary swivel unit	removable
Material	PUR, Acryl, Aluminium-Sheet
Color	black and white (matt)
Power switch	LED pressure switch

E-Technology

Power supply	24 V DC
Power consumption	max. 60 W
Connections	1 × USB 1 × Power
USB cable	USB 3.0 1.8 m Plug A/B
External power supply	2.5 m AC 110/230 V

Room Climate

Operating temperature	18°C – 30°C
Storage temperature	-5°C – 50°C
Relative humidity	40 % – 60 %, max. 80 %

Measuring

Resolution	3.2 Mpx
3D measurement technology	Stripe light triangulation, Blue LightLED
Measuring field XYZ \triangleq max. object size	80 × 60 × 85 mm
Measurement accuracy acc. to ISO 12836	4 μ m

System Requirements PC

PC (minimum)	CPU: i3 with 4 × 3.6 GHz RAM: 16 GB Port: USB 3.0 HDD: approx. 40 – 100 GB free Graphics card RAM: 1 GB
PC (recommended)	CPU: i7 mit 6 × 4,7 GHz RAM: 32 GB Port: USB 3.0 SSD: ca. 100 – 250 GB frei Grafikkarte RAM: 6 MB
Operating system	Windows 10 64 bit Windows 11 64 bit
Interfaces	File formats STL and PLY

3 Scope of Delivery

Scanner Ceramill Map FX, in addition:

- Universal plate with adhesive pads
- 3D calibration model
- External power supply
- USB cable (USB 3.0)

USB stick with:

- Software Ceramill Software Manager
- Calibration data (scanner-specific)
- Manual

PC and screen are not in the scope of delivery.

4 General Safety Information

The scanner was developed according to applicable safety standards with protective measures. Nevertheless, some residual risks cannot be ruled out.

4.1 Intended Use

Use in Dental Technology

The scanner is to be used for:

- the three-dimensional optical measurement of human jaw models/jaw impressions,
- Reconstructions and archiving in prosthetics and orthodontics,
- Scans in occlusal relation with reference to the position of the skull,
- Scans of dental bite registers, dental modelling (wax-up) and screwable
- reference bodies (scan bodies).

4.2 Scannable Materials

The materials must be firm, dimensionally stable, dry, light and opaque. Moving parts must be fixed. Reflective and dark surfaces must be matted with 3D scan spray.

4.3 User Qualification

To commission, install and operate the scanner, users need specialist knowledge of dental technology, in particular CAD/CAM technology, as well as knowledge of the scanner from operating instructions, instruction or training.

4.4 Protection against Injuries

WARNING

Electric shock, burns

Electric shock and burns can be caused by:

- defective electrical devices or cables (e.g. lack of insulation),
- wetness on electricity-carrying parts,
- electrical connections without earthing,
- overloaded wall sockets and lines.

- ▷ Use electrical equipment properly and have it checked regularly. Replace defective cables and power supplies.
- ▷ Disconnect the power supply from electricity when the scanner is not in operation.
- ▷ Do not use a defective scanner.

WARNING

Magnetic disturbance of electronic implants

Magnets are included in scanners (system plate) and accessories (adhesive discs).

- ▷ Persons with implants, e.g. heart pacemakers, may only operate scanners and accessories with medical permission.

WARNING

Stripe light as a triggering stimulus

In appropriately predisposed persons, stripe light can trigger migraines or epileptic seizures.

- ▷ Cover the opening during operation, e.g. with opaque foil.

CAUTION

Pulling in of garments, jewelry, hair

Loose garments, jewelry or hair can be pulled in by automatically moving components.

- ▷ At the scanner workstation, avoid scarves, ties, long chains, open long hair, etc.
- ▷ If something is pulled in, turn off the scanner immediately and disconnect the power supply from electricity.

4.5 Protection against Damage to Property

NOTICE

Inaccurate measurements or device defect due to mechanical damage, soiling or incorrect cleaning of electronic components

All electronic components, in particular the 3D sensor (LED projector, camera) and the contact surfaces of the rotary swivel unit, are sensitive to contact, dirt, moisture, cleaning agents and sharp-edged tools.

- ▷ Keep the scanner clean. Avoid, e.g., milling, grinding and sawing work nearby.
- ▷ Use 3D scan spray outside the scanner.
- ▷ If electronic components become dirty, do not clean them yourself.

NOTICE

Data loss due to magnetic fields

Data carriers, chip cards or technical devices can be disturbed by magnetic fields. Magnets are included in scanners (system plate) and accessories (adhesive discs).

- ▷ Keep a distance between interference-prone devices and magnets.

5 Components

5.1 Scanner

Front

Key

- 1 3D sensor
(LED projector, camera)
- 2 Rotary swivel unit
(removable)
- 3 LED pressure switch
(ON/OFF)

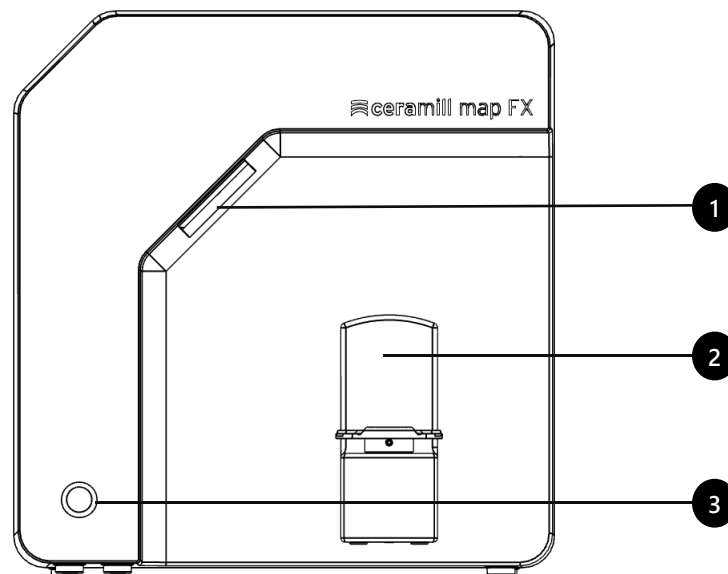
Fig. 1: Scanner - front

Right Side

Key

- 1 3D sensor
(LED projector, camera)
- 2 Rotary swivel unit
- 2a Retaining pins (2 at the top, 2 at the bottom)
- 2b Attachment plate for the rotary swivel unit
- 2c Base plate suitable for Splitex

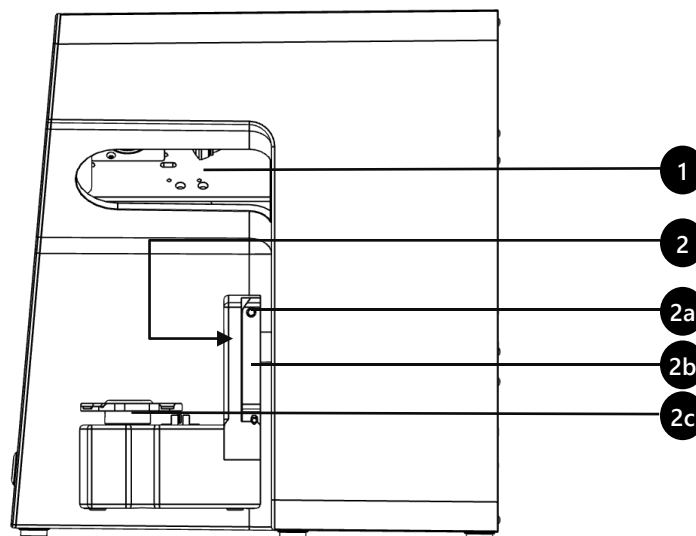
Fig. 2: Scanner – right side

Rear

Key

- 1 Rear panel screw connection
- 2 Ports (power, USB)
- 3 Type plate
- 4 Sensor number

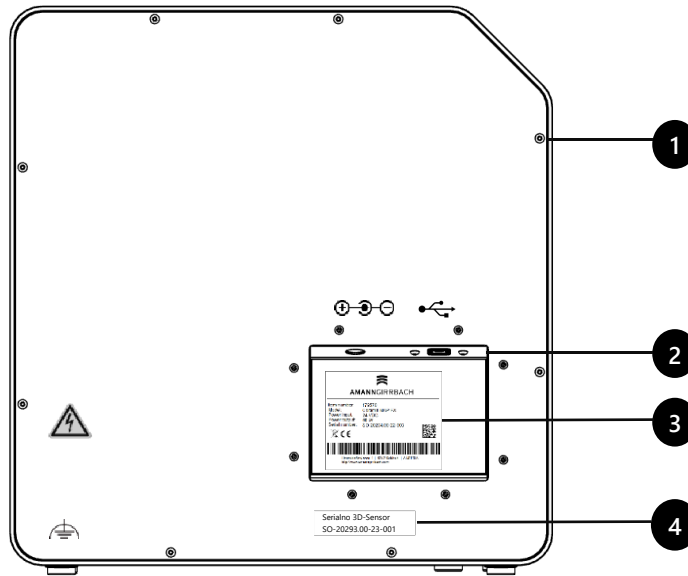
Fig. 3: Scanner – rear

5.2 Accessories

5.2.1 Universal Plate

Key

- 1 Place for jaw models
- 2 Recesses for rubber bands (simple occlusion)
- 3 Plate of type Splitex
- 4 Insertion hole for M-Die (part of the Pro Bundle)

View from Above

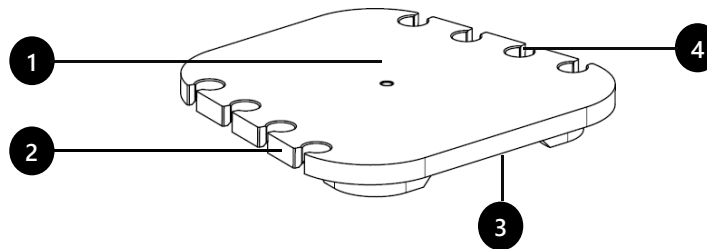
Fig. 4: Universal plate

View from Rear

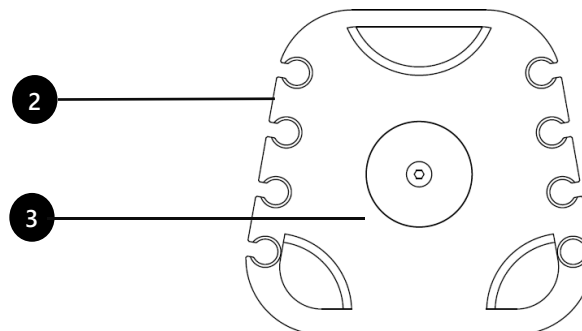
Fig. 5: Universal plate

5.2.2 3D Calibration Model

Key

- 1 Hemispheres (for distance measurement)
- 2 Cylinders (for height measurement)
- 1 Label with and reference values
- 4 Calibration model holder

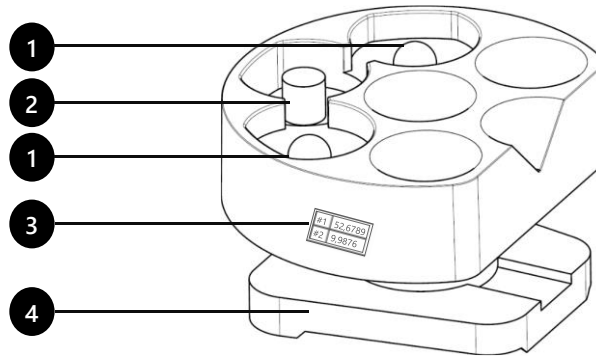
Fig. 6: 3D Calibration model

6 Commissioning

6.1 Workplace

The scanner must be next to the PC, preferably on a laboratory table or professional work table.

Check List

Laboratory, factory, manufacturing	✓
Table	opaque, stable, low vibration
Place	337 × 347 × 322 mm (W × H × D)
Stability	2 × 9.5 kg
Indirect light	✓
Temperature	18°C – 30°C
Wet room areas	✗

NOTICE

Inaccurate measurements or device defect due to non-compliance with workplace requirements

- ▷ Avoid reflections on the measurement object. Protect the scan area in intense ambient light (artificial light, sunlight), e.g. with opaque foil.
- ▷ Keep the scanner clean. Avoid, e.g., milling, grinding and sawing work nearby.

6.2 Unpacking

The original packaging is the best transport protection for scanners and accessories. Keep them for later transporting.

- ▷ Check the delivery for visible damage and discoloration on the shock indicators (indicator label). Log and complain damages in accordance to the shipping terms.
- ▷ Transport the box to the workplace.
- ▷ Open the box from above and remove the padding material.
- ▷ Remove the accessory boxes (stored on the side).
- ▷ Grab under the scanner and lift it out of the box.
- ▷ Place the scanner on the work table next to the PC (distance max. 2 meters).

Carrying points
 ✓ = allowed
 ✗ = prohibited

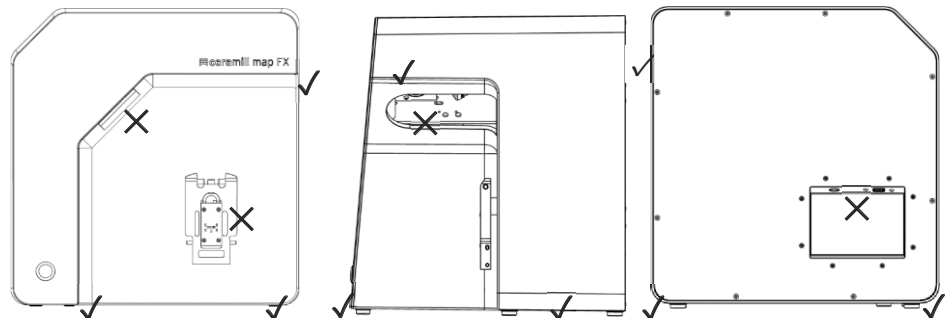
Fig. 7: Carrying points

6.3 Connecting

6.3.1 Checklist Connections

Distance scanner – PC	max. 2 m
Sockets	min. 3
Power cable without power supply	✗
Extension cable	✓
Multi-plug	✓
Socket adapters	✓
USB Hub	✗
USB extension	✗

NOTICE

Incorrect data transmission or device damage due to inappropriate cabling

- ▷ USB Hubs, USB extensions and USB cables longer than 2 meters deteriorate the data transfer.
- ▷ The scanner requires an external power supply for the power connection.
- ▷ USB cables or power supplies with the same specification as the included ones are suitable as a replacement.

6.3.2 Procedure

- ▷ Plug the USB and power connectors into the ports on the rear of the scanner.

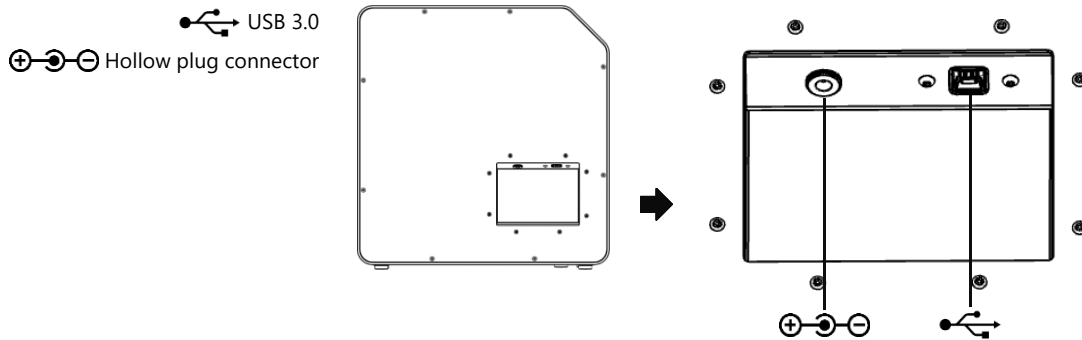
Fig. 8: Connections


- ▷ Connect the USB cable to a USB 3.0 port (blue) on the back of the PC.
- ▷ Connect the plug-in power supply to the socket. If necessary, insert a socket adapter in between (not included).

Once connected, the scanner can stand close to the wall.

6.3.3 Attaching the Rotary Swivel Unit

The rotary swivel unit is delivered separately in a box.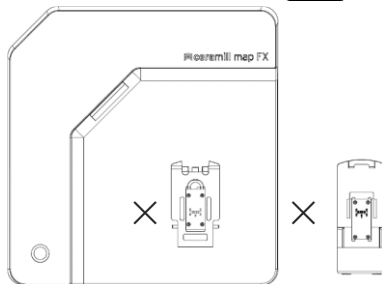

NOTICE

Malfunction due to damage to contact surfaces

The contact surfaces on the rotary swivel unit and on the attachment plate in the scanner are sensitive electronic components for the transmission of data and electricity.

- ▷ Do not touch the contact surfaces.
- ▷ Avoid soiling or mechanical damage.

- ▷ For fixing in the scanner, hang the rotary swivel unit from above on the holding pins of the attachment plate.
- ▷ Press against the rotary swivel unit to snap it into place on the lower retaining pins.

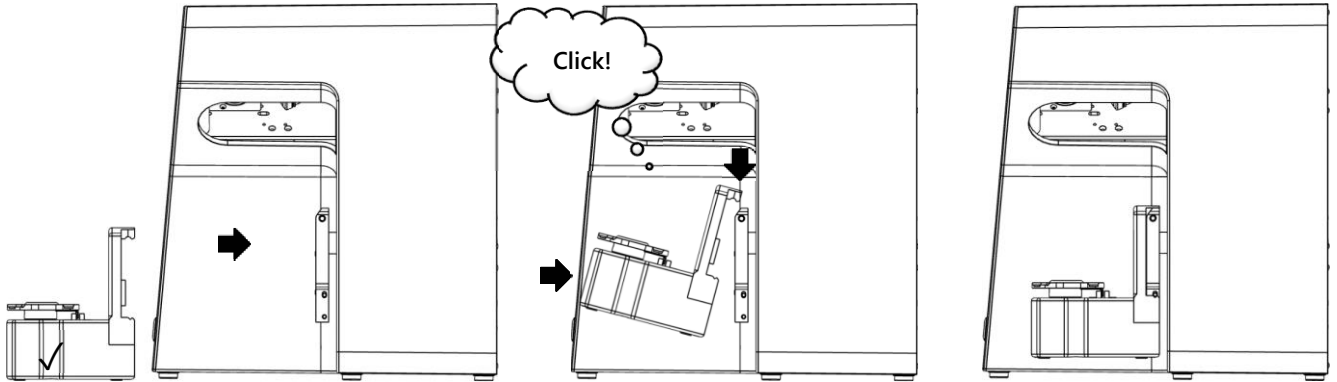
Fig. 9: Fixing the rotary swivel unit

6.4 Installing

6.4.1 Requirements

- 31 Does the PC meet the system requirements?
- If you are using a notebook: connection to the electricity supply system is required.
- Are all updates for Windows and drivers installed?
- Does the virus protection allow the scan software to run?
- Does the Windows user have the right to run setups and apps?

6.4.2 Installation

- ▷ Insert the USB stick of the scanner into a suitable PC port and open the drive in Windows Explorer.
- ▷ Execute the setup file `Ceramill Software Manager`.
- ▷ After installing the Ceramill Software Manager, log in to it with your AG.LIVE account.
- ▷ In the **Available Software** list, go to the **Ceramill Map FX** software and click on **Show Details**.
- ▷ Start the software download.
- ▷ Then run the installation by clicking **Install**.
- ✓ The program files are installed.

6.4.3 Updates

New versions of the software are listed in the **Installed** section of the Software Manager and can be installed in the same way as a new installation.

6.5 Calibration

Calibration is a prerequisite for correct measurements and the last step of commissioning. The 3D calibration model is required for calibration.

6.5.1 Placing the 3D Calibration Model

- ▷ Place the calibration model with its base plate on the Splitex plate in the scanner.

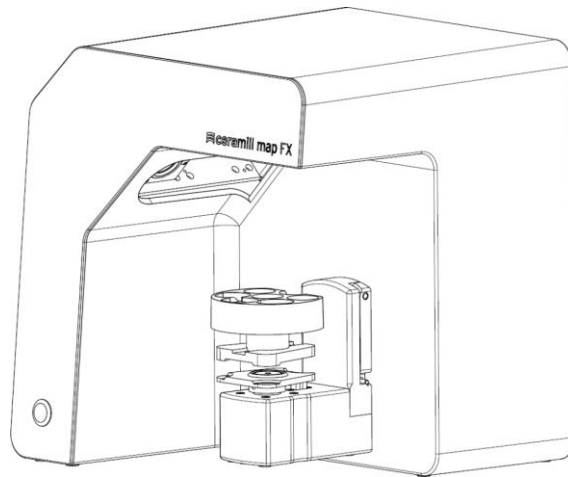
Fig. 10: Placing the 3D calibration model

6.5.2 Calibration Procedure

Calibration is mandatory for commissioning.

- ▷ Turn on the scanner (front LED pressure switch).
- ▷ Start the software. To do this, create a fictitious case in the Ceramill Cockpit, for example. Then start the Ceramill Map FX scanner software from the right-hand area of the Ceramill Cockpit.
- ▷ The software requires a calibration..
- ▷ Transfer the values from the label of the 3D calibration model to field #1 and #2 in the software (exact values with commas and decimal places).

Label with sample values

#1	52,6789
#2	9,9876

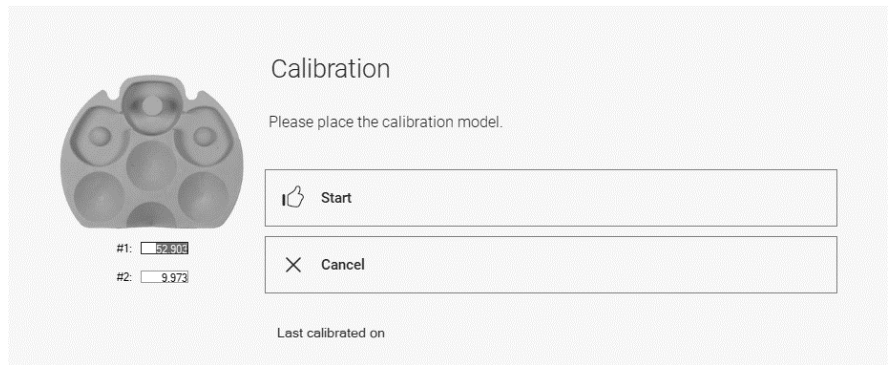
Fig. 11: "Calibration" dialog

- 41 ▷ Now the 3D place calibration model.
- ▷ Start the calibration. Wait until the final message.
- ✓ After successful calibration, scanning can be performed.

Data backup

Calibration data is not transferable to other scanners.

- ▷ Keep the USB stick for possible reinstallation of the scanner.

- 48 ▷ Re-calibration is part of regular maintenance.

7 Scanning

7.1 General Information

The information in this manual is focused on handling. Detailed instructions for scanning can be found integrated in the software.

Tips on specific projects can be found in an instruction or training.

Scanning with Attached Small Parts

Prosthetic small parts or modeling can be scanned on the jaw model:

- Mush bite as a representative of the antagonist jaw,
- Gingiva and wax-up,
- Scan bodies.
- ▷ Attach the small parts to the jaw model (with removable glue on a few points).

Mattifying the Scan Object with 3D Scan Spray

Reflective or dark scan objects must be matted with 3D scan spray (specified for dental technology, very finely grained).

Without matting, the scan result becomes worse. Use 3D scan spray outside the scanner.

- ▷ Spray the scan object thinly and evenly, do not touch the spray layer.

Service Position of the Rotary Swivel Unit

When placing scan objects, the rotary swivel unit can be moved unintentionally. As a result, the position of the rotary swivel unit stored in the scan software no longer corresponds to the actual position..

- ▷ Correct the position of the rotary swivel unit with the Service Position function (after positioning, before scanning).

7.2 Scanning a Jaw Model

7.2.1 Jaw Model on Splitex plate

Jaw models on a Splitex type mounting plate with magnetic adhesive disk can be scanned without any other holder.

- ▷ Place the Splitex plate directly on the base plate of the scanner.

7.2.2 Jaw Model on Universal Plate

Jaw models without a Splitex plate and articulated jaw models on non-Splitex plates are scanned on the universal plate.

The universal plate must be covered with fixing material.

- ▷ Press the jaw model or mounting plate firmly onto the adhesive pads (bottom side centered). Test the fixture by gently turning and tilting (bottom side centered).
- ▷ Place the universal plate on the base plate of the scanner.

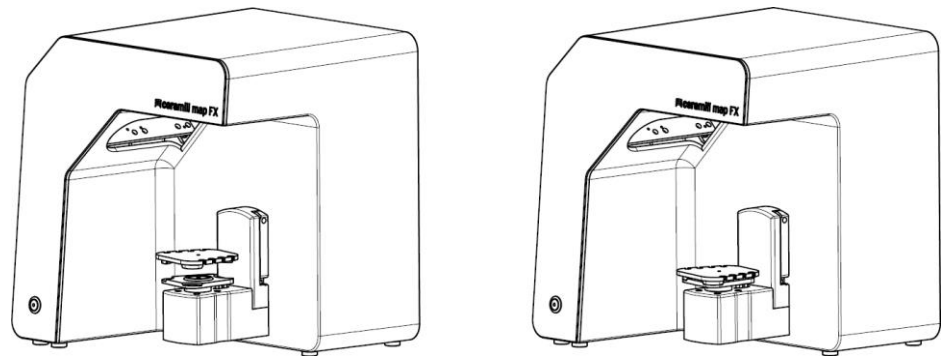
Fig. 12: Placing the universal plate

7.3 Jaw Models in Simple Occlusion

Jaw models in simple occlusion (without use of an articulator) are scanned on the universal plate. The jaw models must be fixed with rubber bands for this purpose (condition: new, stable, approx. 0.4 cm thick). Ring and cruciate bands are possible. The material does not interfere with the vestibular scan.

Rubber Band Attachment

- ▷ Loosely place the OK model in occlusion on the UK model.
- ▷ Select one of the following methods:
 - Form a long rubber ring (approx. 8.5 cm) into an eight and place it buccally over the upper jaw model.
 - Place two short rubber rings (shorter than approx. 8 cm) x-shaped over the upper jaw model.
 - Place a cross rubber band around both jaw models so that the crosses are on top and underneath and the bands are tensioned in an x-shape.
- ▷ Firmly press the connected jaw models onto the universal plate (with fixing material). Test the fixture by gently turning and tilting.
- ▷ Lay the rubbers around the recesses of the universal plate, several times if necessary, until the rubber is taut.

Fig. 13: Universal plate with recesses

7.4 Scanning Articulator

Mean-value vestibular scans can be created with all common articulators. With AMANN GIRRBACH Artex® condyle-related vestibular scans are possible.

7.4.1 Articulator Workflow

The software guides you through the following steps:

- Removing the rotary swivel unit.
- Putting the articulator in front of the camera.
- Starting the scan.
- Removing the articulator.
- Reattaching the rotary swivel unit..

After starting, the software waits for five seconds. Therefore, the steps of placing the articulator and starting the scan can be changed in order to save time.

7.4.2 Rotary Swivel Unit

- ▷ Wait until the scan software requires the rotary swivel unit to be removed.
- ▷ Pull the rotary swivel unit off the attachment plate in the scanner (slightly forward, then lift off the holding pins).
- ▷ Place the rotary swivel unit in a protected place.

The contact surfaces must not be soiled or touched.

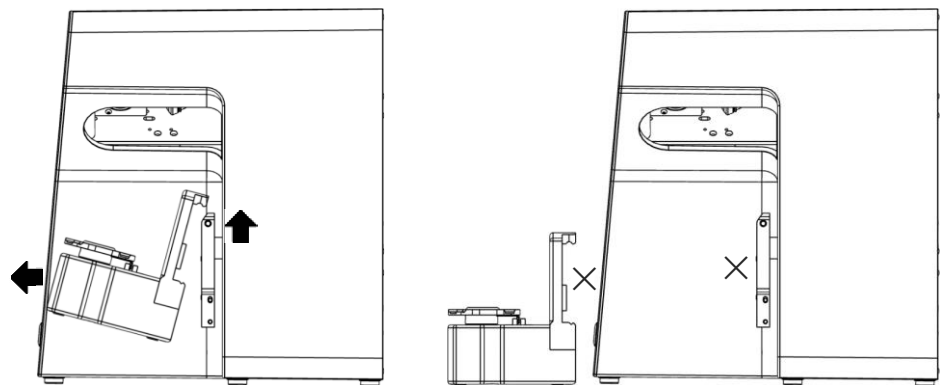
Fig. 14: Removing the rotary swivel unit

39

After the scan, wait until the scan software requires the rotary swivel unit to be attached.

7.4.3 Placing the Articulator

- ▷ If possible, remove the upper support pin of the articulator.
- ▷ Tilt the articulator so that the dental arch is facing the 3D sensor. Check the alignment in the software (live image).
- ▷ The articulator must remain motionless until measurement. The software reports when the articulator can be removed.

Image for illustrative purpose
Artex Articulator

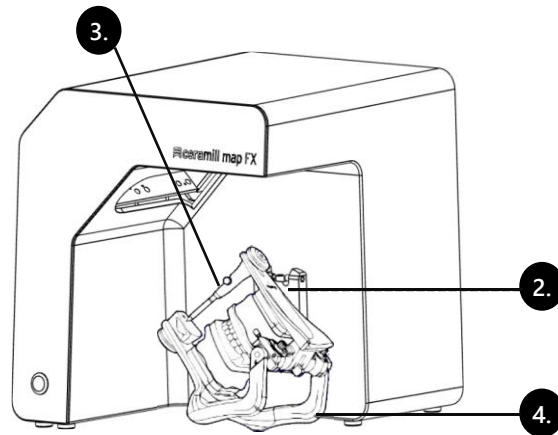
Fig. 15: Placing the articulator

8 Multi-Die (Pro Upgrade)

8.1 Product Description

Key

- 1 M-Die
- 2 Compartments for stumps
- 3 Retaining pin
- 4 Fixing material

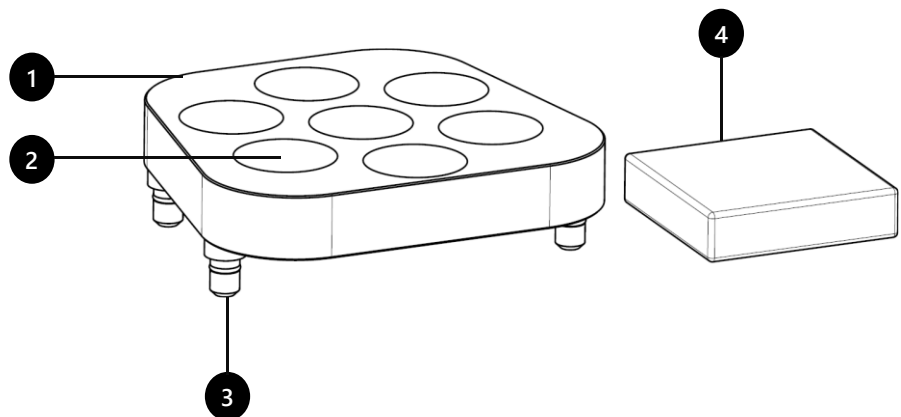
Fig. 16 Scope of delivery Multi-Die

53 Multi-Die is an additional component for the "Multi-Die" scanning process: With "Intelligent Multi-Die", up to seven dies can be captured from one patient within one scan.

M-Die can only be used in conjunction with the universal plate.

8.2 Usage

EN

The usage is explained in the "Multi-Die" mode.

Global Scan

- ▷ Attach the model to be scanned (upper or lower jaw) centrally on the universal plate.
- ▷ Place the universal plate in the scanner.
- ▷ Scan the models.

Placing Stumps

- ▷ Place M-Die on the universal plate (retaining pins in insertion holes).

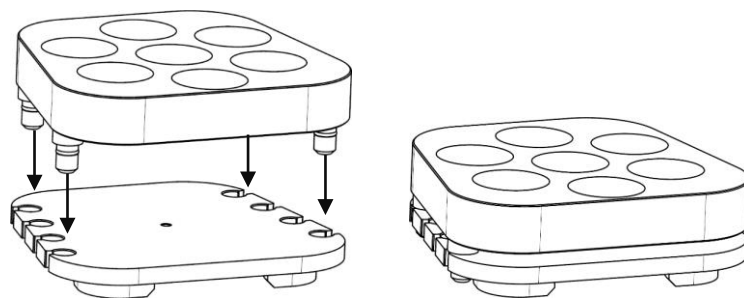
Fig. 17: M-Die on universal plate

- ▷ Fill the compartments in M-Die with fixing material.
- ▷ Place the stumps to be scanned in M-Die, centered in the position displayed in the software.
- ▷ Press the stumps into the fixing material so that they lie flat on the surface [A].
 - Do not fix with fixing material above the compartment [B].
 - Do not let it stick out too far [C].
 - Do not press in too deep [D].

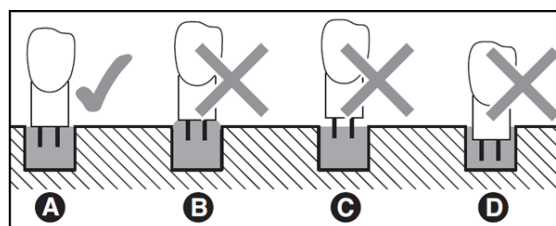
Fig. 18: Impression depth of the stumps

Scanning

- ▷ Place the universal plate with M-Die directly on the Splitex plate. The universal plate is sufficiently fixed by the integrated magnet.

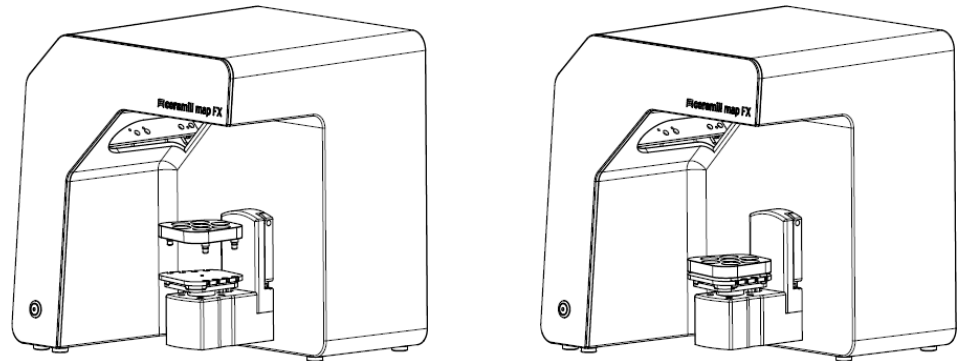
Fig. 19: Universal plate with M-Die in the scanner

- ▷ Scan the stumps in "Intelligent Multi Die" mode.
- ▷ To remove the universal plate from the scanner: Grasp the back or side of the plate and lift it off.

9 Maintenance and Cleaning

9.1 Maintenance

Precondition for accurate measurements is the calibration of the scanner.

Procedure and first help
 41, 51

- ▷ Calibrate the scanner at the following intervals:
 - 1 × monthly (with automatic reminder) at constant operating temperatures and normal production quantities,
 - 1 × daily at changing operating temperatures and high production quantities,
 - after an upgrade,
 - after a transportation.
- ▷ Start an optional calibration from the menu.
- ✓ The date of the last calibration is displayed and the last edited values are suggested.
- ▷ Calibrate even with diminishing scan results, e.g. with perforated data networks despite rescans.

9.2 Cleaning

The scanner is a high-precision measuring device and must be handled with care.

- ▷ Remove dust from surfaces, from the universal plate and the rotary swivel unit from the outside. For this purpose, the rotary swivel unit can remain in the scanner.

Contact the customer service if a non-cleanable component is dirty.

✓ Clean if Necessary	✗ Never clean
— Surfaces (outside + inside)	— 3D sensor
— Universal plate	— (LED projector, camera)
— Rotary swivel unit (external)	— Connections
	— Contact surfaces on rotary swivel unit and attachment plate
	— Inner components
✓ Suitable Agents	✗ Unsuitable Agents
— Cleaning cloth (smooth, lint-free, antistatic)	— Tools (knives, scrapers, brushes, brushes, etc.)
	— Chemical additives (e.g. cleaning agents, solvents and disinfectants)
	— Water

10 Fault and Repair

Do not to work with a defective scanner and defective accessories!

10.1 Customer Service

The customer service is responsible for complex fault analyses and repairs.

Due to improper repair, the scanner may no longer function as intended. Never repair it yourself.

Serial Numbers

The serial numbers can be found on the rear of the scanner. The first characters are fixed, the last ones individual.

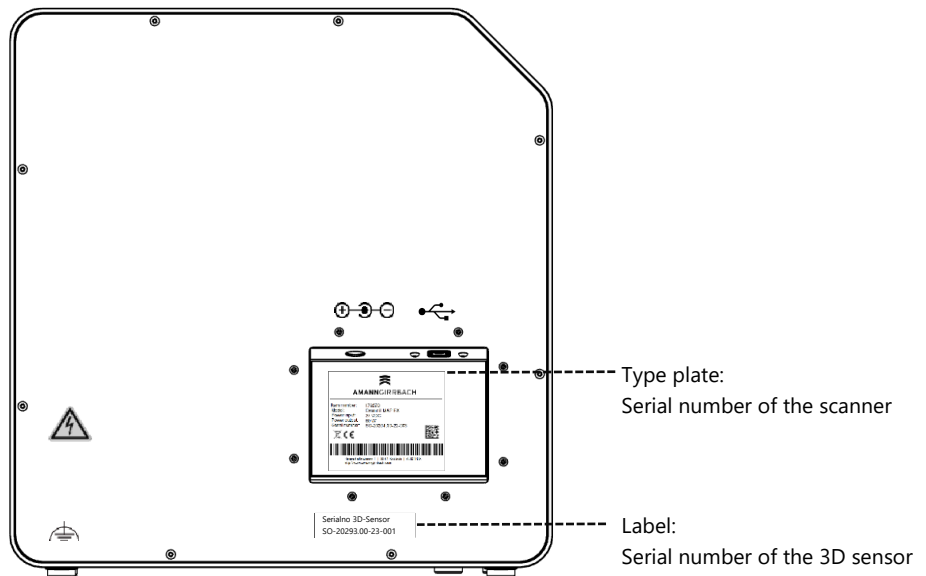
Fig. 20: Serial numbers

10.2 First Help

10.2.1 Base Check

Workplace

- 37 Is the indoor climate appropriate and constant?
- Is the work table suitable?
- Are scanners, accessories and work table clean?

Connections

- 38 Are all connections tight?
- Are the cables externally intact?
- Does the external power supply stay cool in the socket?
- Is the scanner connected directly to the PC?
Cable max. 2 meters, no USB hub, no USB extension
- Does the USB port of the PC have version 3.0?
Blue connection, version display in the device manager

Software Installation

- 40 Has the installation of the scan software been completed without error?
If in doubt, install again.
- Were the PC, scanner and software restarted after installation?

10.2.2 Failed Calibration

If calibration fails, exact measurements are not possible. After initial installation, you will not be able to scan.

Try the following measures:

Label with sample values

#1	52,6789
#2	9,9876

- ▷ Protect the scanner from direct light.
- ▷ Restart software and scanner.
- ▷ Correct the entered values on comma and decimal places. Compare with the label on the 3D calibration model.
- ▷ Check if the calibration data matches the scanner:
Serial number 3D sensor = name of the calibration data folder
File path:
C:\ProgramData\3D-Scanner\DeviceCharacteristics\Sensors
If in doubt, rename the existing folder (prefix a character) and copy the folder from the USB stick to this path again.
- ▷ Check the 3D calibration model for damages. Discolorations and damages to the edge are tolerable. In case of central damage, the 3D calibration model must be replaced.
- ▷ Check if there are any connectivity problems.

10.2.3 Hardware Problems

- ▷ In case of hardware problems, always contact the customer service. Do not attempt to repair any hardware.

Signs can occur on various components, such as:

Examples

- Rotary swivel unit
 - does not stop/does not move,
 - has dirty/damaged contact surfaces.
- 3D sensor (LED projector, camera)
 - Component is dirty/damaged,
 - No live image appears in the software.
- Motor
 - makes unusually loud noises,
 - Error message for engine control.

10.2.4 Connectivity Problems

Connectivity problems can result from a faulty USB connection, camera drivers, and unfavorable Windows settings.

Signs of connection problems may include:

- Error message concerning the camera connection,
 - Error message concerning the scanner connection,
 - Failure of calibration,
 - Measurement error despite correct procedure.
- ▷ If an error message appears with instructions, follow them.
 - ▷ Exit the scan software, turn off scanner and PC. Disconnect the scanner from electricity and wait briefly before restarting.
 - ▷ Remove unnecessary USB devices from the PC.
 - ▷ Replace the USB port on the PC (blue port), if necessary, also the USB cable (USB 3.0).

10.2.5 Scan Problems

Startup Problems

Clarify in advance:

-  50 Is the software installed correctly and completely?
- Has an upgrade of the software been installed?
- Has any other software been installed on the PC in the meantime?

Scan Results

Poor scan results or sudden loss of quality are some of the symptoms for software problems. Other causes can be::

- Is the scan area in sunlight/very bright artificial light?
- Does the **Plaster Color** setting match the scanned material?
- Is the **Quality** setting appropriately chosen for the application?
- Was dark or glossy material prepared with scan spray?
- Was the articulator motionless during the vestibular scan?

Scan Area

If the scan object does not appear in the 3D viewer of the software, the scan object is not placed in the scan area..

- Articulator: Does the dental arch point exactly to the 3D sensor?
Adjust the angle. Check the camera view in the live image.
- Jaw model: Articulated jaw models on mounting plates may be too high.
Disconnect the jaw model from the base.

11 Paid Bundles

Feature	Basic	Upgrade 1 Impression	Upgrade 2 Pro	Accessories
1 Plaster model squeeze bite	x	x	x	
2 Plaster models Artex CR	x	x	x	
2 Plaster models articulator	x	x	x	
2 Plaster models in occlusion	x	x	x	Universal plate + fixing material
Dark Mode	x	x	x	
Complete jaw scan	x	x	x	
Presentation method "Every single tooth"	x	x	x	
Standby mode	x	x	x	
Free mode	x	x	x	
Presentation method "Every second tooth"		x	x	
Auto-articulation		x	x	
Full jaw impression		x	x	Impression holder
HD scan		x	x	
Monochrome/color scan		x	x	
Triple Tray impression		x	x	Impression holder
Additional scans			x	
Background matching DNA			x	
Automatic cutting plane			x	
Advanced global scan strategy			x	
Combine meshes			x	
Multi-Die			x	M-Die + fixing material
Additional STL output			x	
Speed update			x	

11.1 Bundle Activation

With the purchase of a paid bundle you will receive an activation code (at least 13 digits) to unlock the corresponding software functions.

- ▷ Start the software. Switch to "Free mode".
- ▷ In the right menu, select **Add Activation Code** and enter your activation code.
- ▷ Repeat this step for further activation codes.
- ▷ Restart the software.

123

- ✓ The functions are now unlocked. If not, repeat the steps to exclude a capture error.

Data Backup

Activation codes are stored in the calibration data. They only apply to the sensor number of a single scanner.

- ▷ Keep the activation codes together with the data carrier from the scope of delivery.
- ▷ The activation must be repeated in case you:
 - reinstall the software with calibration data,
 - re-copy the calibration data,
 - install the software on another PC
(all from the USB stick).

12 Disposal

12.1 Disposing of Packaging

You can dispose the packaging after the warranty has expired. The manufacturer recommends storing the packaging for transportation of the scanner.

12.2 Disposing of the Device

This symbol is for devices according to the European Directive 2002/96/EC for WEEE (Waste Electrical and Electronic Equipment).

Electronic equipment is not for the household waste.

Disposal via a public disposal carrier is not possible, as the scanner is exclusively used commercially or industrially.

The scanner can be sent to the manufacturer/dealer for disposal (within the scope of the EU).

13 CE Conformity

This product was designed and manufactured based on careful selection of the harmonised standards to be observed, as well as additional technical specifications. It thus corresponds with the state-of-the-art and ensures maximum safety.

In terms of design and performance, this product complies with the European Directives and the supplementary national requirements. Conformity has been confirmed with the CE marking.

The product's Declaration of Conformity can be requested from the manufacturer.

EG Guidelines	Applied harmonised standards
Safety requirements for electrical equipment for measurement, control, and laboratory use	DIN EN 61010-1: 2020-03
Electrical equipment for measurement, control and laboratory use - EMC requirements	DIN EN IEC 61326-1:2022

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Amann Girrbach AG