

## NEXTDENT FOR CERAMILL MATERIALS – PROCESSING TIMES

| MATERIAL              | INDICATION  | BOTTLE MIXING TIME |            |                  | STIRRING TIME | CLEANING TIME     |              |               | CURING TIME     |
|-----------------------|---|--------------------|------------|------------------|---------------|-------------------|--------------|---------------|-----------------|
|                       |   | Bottle shaking     | New Bottle | After first use* | in Resin Tray | Cleaning solution | Pre-cleaning | Fine-cleaning | LC 3D Print Box |
| NextDent Cast         |    | 5 min.             | 2.5 h      | 1 h              | 60 sec.       | Ethanol           | 3 min.       | 2 min.        | 5 min.          |
| NextDent C&B MFH      |    | 5 min.             | 2.5 h      | 1 h              | 60 sec.       | Ethanol           | 3 min.       | 2 min.        | 30 min.         |
| NextDent Gingiva Mask |  | 5 min.             | 2.5 h      | 1 h              | 60 sec.       | Ethanol           | 3 min.       | 2 min.        | 10 min.         |
| NextDent Model 2.0    |  | 5 min.             | 2.5 h      | 1 h              | 60 sec.       | Ethanol           | 3 min.       | 2 min.        | 10 min.         |
| NextDent Ortho IBT    |  | -                  | 5 min.     | 5 min.           | 30 sec.       | Ethanol           | 3 min.       | 2 min.        | 10 min.         |
| NextDent Ortho Rigid  |  | -                  | 5 min.     | 5 min.           | 30 sec.       | Ethanol           | 3 min.       | 2 min.        | 10 min.         |
| NextDent SG / Orange  |  | -                  | 5 min.     | 5 min.           | 30 sec.       | Ethanol           | 3 min.       | 2 min.        | 10 min.         |
| NextDent Tray         |  | 5 min.             | 2.5 h      | 1 h              | 60 sec.       | Ethanol           | 3 min.       | 2 min.        | 10 min.         |
| NextDent Try-In       |  | 5 min.             | 2.5 h      | 1 h              | 60 sec.       | Ethanol           | 3 min.       | 2 min.        | 20 min.         |
| NextDent Denture 3D+  |  | 5 min.             | 2.5 h      | 1 h              | 60 sec.       | Ethanol           | 3 min.       | 2 min.        | 30 min.         |

\*Considerations for resin that has been sitting, unused, for over 1 month  
 For more information please see NextDent materials Instructions for Use enclosed in the bottle packaging