



Eko

Analysis Software

v1.0

User Manual

LBL046 Rev A

This manual is intended to provide information to guide trained operators in the safe and effective operation of the Eko Analysis Software. It is important that you read and understand all instructions in this manual before operating the device, and pay careful attention to the warnings and cautions throughout the manual.



CAUTION: Federal (USA) law restricts this software to sale to or on the order of a clinician.

Eko Devices, Inc. assumes no responsibility for any injury to anyone, or improper use of the product, that may result from failure to use this product in accordance with the instructions, cautions, warnings, or statement of intended use published in this manual.

Eko Devices, Inc. may have patents, patent applications, trademarks, copyrights, or other intellectual property (IP) rights covering subject matter in this document. The use of this document does not give anyone license to these patents, trademarks, copyrights, or other intellectual property.

For general and product related comments, questions, or concerns, please contact Eko directly

Please report any injury or adverse event to Eko using any of the contact methods below.

Manufacturer:

Eko
2600 10th St. Suite 260
Berkeley, CA 94710
USA

General Assistance and FAQs

ekohealth.com/getstarted

Direct Contact

support@ekohealth.com

Phone Support

+1.844.356.3384 (US)

Product Reference and Information

ekohealth.com

Patents

Listing of applicable U.S. Patents in compliance with 35 U.S.C. §287:
ekohealth.com/patents

Disclaimer

The information contained in this document is subject to change without notice.

Copyright

Copyright © 2019 Eko Devices, Inc.
All Rights Reserved.



Consult instructions for use.



Catalog Number



Lot Number



Serial Number

 CAUTION

- **This is NOT a diagnosis.** Findings from the Eko Analysis Software are only meaningful in conjunction with clinician over-read and interpretation. This software is not intended to be the sole means of diagnosis.
- The Eko Analysis Software can provide automatic computer interpretations. A computer generated interpretation cannot replace sound medical reasoning by a trained professional. Therefore, a physician should always review the interpretation and final analysis should always be obtained by a physician. Proper administration, diagnosis and implementation of the test is the physician's responsibility.
- The quality of the interpretations depends heavily upon the quality of the inputted data.
- Only analyze high quality recordings of ECG and heart sounds.
- Final judgment on the diagnosis still lies with the qualified medical personnel.
- The medical history and physical examination results must be taken into consideration during the decision-making process.
- The software is intended for use in a quiet environment on patients with an audible heart sound and a clear, noise free ECG tracing.
- The software should not be used on patients where the physician has difficulty to auscultate with an electronic stethoscope, obese patients and patients with pulmonary noise.

- The ECG classifier has only been evaluated for the detection of AFib or normal sinus rhythm and is not intended to detect any other type of arrhythmia. It cannot detect heart attacks.
- The murmur classifier only informs the clinician about the presence or absence of murmurs in the phonocardiogram. It does not attempt to classify murmurs as innocent or pathologic. The physician should conduct a more complete analysis of the detected murmur to determine whether it is innocent or pathologic.
- The Bradycardia detection indicates that atrial fibrillation is not detected and the heart rate is less than 50 beats per minute, which is slower than normal for most people. Please note, less than 50 beats per minute can be normal for health adults, athletes and during sleep.
- The Tachycardia detection indicates that atrial fibrillation is not detected and the heart rate is faster than 100 beats per minute. Please note, greater than 100 beats per minute can be normal with stress or physical activity.
- Notifications made by this feature are potential findings, not a complete diagnosis of cardiac conditions. All notifications should be reviewed by a medical professional for clinical decision-making.
- The Eko Analysis Software may be unable to return a result if the CORE / DUO device is unable to connect to the mobile device over Bluetooth or if the mobile device is unable to connect to the internet.

The Eko Analysis Software is intended to provide support to the physician in the evaluation of patients' heart sounds and ECG's. The software analyzes simultaneous ECG and heart sounds. The software will detect the presence of suspected murmurs in the heart sounds. The software also detects the presence of atrial fibrillation and normal sinus rhythm from the ECG signal. In addition, it calculates certain cardiac time intervals such as heart rate, QRS duration and EMAT.

It is not intended as a sole means of diagnosis. The interpretations of heart sounds and ECG offered by the software are only significant when used in conjunction with physician over-read and is for use on adults (> 18 years).

1.6 Notices

This manual provides instructions for the use of the Eko Analysis Software. It is assumed that the user is familiar with basic mobile application use on mobile and desktop devices.

Standard procedures for auscultation should be followed including background noise reduction and optimal patient positioning when capturing data for the Eko Analysis Software. The quality of the ECG is dependent on proper preparation practices including body hair, skin dryness, and clean contact area.

It is highly recommended that users of the mobile app and web dashboard use device and networking security features to protect patient data created and stored using this software, in addition to security features embedded in the system.

The Eko Analysis Software requires a minimum internet connection speed. The recommended upload speed for the mobile app is 4000 Kbps. 4G cellular data service or similar is recommended.

MURMUR DETECTION

The murmur detection analysis identifies possible murmurs in an heart sound recording. It is not a diagnosis, it is only a potential finding. You should conduct further evaluation if a murmur is detected.

It will not detect cardiac conditions that do not cause heart murmurs. A murmur finding is often a recommendation for echocardiography or specialist referral.



CAUTION

The software does not distinguish between different kinds of murmurs.

Proper handling of the stethoscope to minimize external background noises and hand rub/clicks should be practiced. **All heart sound positions should be analyzed before making a decision.** This software cannot analyze lung sounds. **This algorithm does not distinguish between innocent and pathologic heart murmurs.**

The possible results are:

- No Murmur Detected
- Murmur Detected in Heart Sounds
- Poor quality heart sounds.

ATRIAL FIBRILLATION AND SINUS RHYTHM

The Atrial Fibrillation (AF) analysis detects atrial fibrillation in an EKG tracing. After you have recorded an EKG, if atrial fibrillation is detected you will be notified within the app. This finding is not a diagnosis, it is only a potential finding. You should conduct further diagnosis if AF is found. **The AF analysis detects atrial fibrillation (AF) only.** It will not detect other potentially life threatening arrhythmias, and it is possible that other cardiac arrhythmias may be present. The AF detector only detects AF during a recording. If an ECG is neither AF nor sinus rhythm, the analysis will return "unclassified."



CAUTION

The software does not identify other arrhythmias.

The possible results are:

- Normal Sinus ECG Rhythm
- Atrial Fibrillation Detected
- Unclassified ECG
- Poor quality ECG

ELECTRO-MECHANICAL ACTIVATION TIME (EMAT)

The EMAT analysis calculates the time interval between the Q wave of the ECG and the first heart sound (S1). The analysis will be inaccurate if a clear Q wave on the ECG is not present or the S1 is not clearly present in the heart sounds. The EMAT is calculated as a percent of the R-R interval. This percent indicates the portion of the cardiac cycle spent in between the Q wave initialization and the closing of the mitral and tricuspid valves (S1).

BRADYCARDIA AND TACHYCARDIA

The Bradycardia and Tachycardia analysis provides alerts for low and high heart rates in the ECG or heart sounds.

Bradycardia is defined as a heart rate **below 50 BPM.**

Tachycardia is defined as a heart rate **above 100 BPM.**

ANALYSIS USAGE

Note that the analysis software has been trained and tested on ECGs captured from Lead II. Not every ECG vector will produce high accuracy. A high quality ECG and heart sound signal must be present.

SIGNAL QUALITY ANALYSIS

If a recording has poor ECG or heart sound signal, the interpretation will return with a result of poor quality ECG or heart sounds and will not analyze the recording.



Download the Eko app, available on the App Store® or Google Play Store® and follow the on-screen setup instructions to access the Eko Analysis Software.

Eko

ekohealth.com

© 2019 Eko Devices, Inc. All Rights Reserved

Eko, the Eko logo, CORE, and DUO are trademarks of Eko. Other company and product names may be trademarks of their respective owners.

LBL 046 Rev A