

Step 1

Assemble your CORE

1. Choose the standard or pediatric tubing connector and screw into the top of the CORE, the end closest to the power toggle

Attach the chest piece to the

2. tubing connector

Choose either the standard or

3. pediatric barb to screw into the base of the CORE

Attach your stethoscope tubing to the barb

4. Slide the On / Off toggle to power the device. CORE will only deliver amplified sound in the ON position
5. the device. CORE will only deliver amplified sound in the ON position

Your CORE is now ready to be used in Analog or Digital mode.



CORE is ON & seeking device



CORE is ON & connected



CORE is OFF & charging




CORE is fully charged



CORE is LOW on battery



CORE is recording

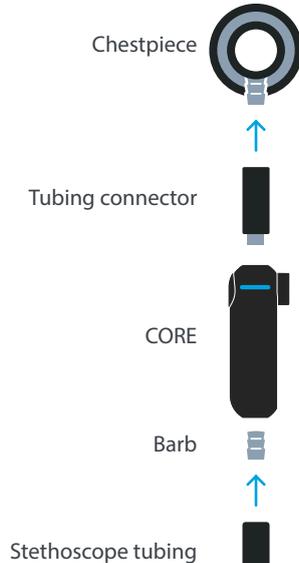

Step 2 (optional)

Connect CORE to the Eko App

1. Download and install the App
2. Create an account, and log in
3. Turn on the CORE
4. Follow the prompts to connect your device and select the CORE

With the app installed, you can now Visualize Auscultation Sounds, Create Recordings, Save, and Share Patient Data



Google Play and the Google Play logo are trademarks of Google LLC.

TM and © Eko Devices Inc.
LBL 055 Rev B

In the Box

CORE, pediatric and standard tubing connectors, and USB charging cable.

Compatibility

CORE is designed and tested to work with most acoustic stethoscopes.



User Manual and Support

Review the user manual before using your Eko CORE.

To view the manual visit:
ekohealth.com/getstarted

For support, please email:
contact@ekohealth.com

Charging

Connect the micro USB cable to the CORE and use a UL-certified or CE-marked USB wall charger. The CORE will take about 2 hours to fully charge. The light will turn green when it is fully charged.

Amplification

CORE provides up to 40x amplification for hearing support. CORE does not connect wirelessly to hearing aids.

Active Noise Cancellation

The CORE includes active noise cancellation to remove ambient noise from the environment. Active noise cancellation can be turned on or off through the Eko app.

CORE

Digital Attachment

