Eko

LBL055 Rev C.0

#\$TAB

D #\$TAB

2020-11-02

Eko CORE

© 2020 Eko Devices, Inc.

Eko Devices, Inc.

Confidential & Proprietary #\$TAB

L055 Rev C.0

Labeling Page 1 of

Eko

LBL055 Rev C.0

#\$TAB

D # \$TAB

2020-11-02

English	4 – 5
Español	6 – 7
Français	8 – 9
Português	10 – 11
Deutsch	12 – 13
Türkçe	14 – 15
Italiano	16 – 17
Polski	18 – 19
Română	20 – 21
Nederlands	22 – 23
Ελληνικά	24 – 25
Čeština	26 – 27
Svenska	28 – 29
Magyar	30 – 31
Български език	32 – 33

Eko Devices, Inc.

Confidential & Proprietary # \$TAB

L055 Rev C.0

Labeling Page 2 of

Eko

LBL055 Rev C.0

#\$TAB

D # \$TAB

2020-11-02

Dansk	34 – 35
Suomi	36 – 37
Slovensky	38 – 39
Norsk	40 – 41
Gaeilge	42 – 43
Hrvatski	44 – 45
Shqip	46 – 47
Lietuvių	48 – 49
Македонски	50 – 51
Slovenščina	52 – 53
Latviešu	54 – 55
Eesti	56 – 57
Crnogorski	58 – 59
Malti	60 – 61

Eko


LBL055 Rev C.0

#\$TAB

D # \$TAB

2020-11-02

Quick Start Guide
English

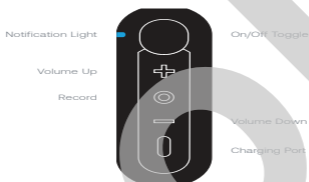Step 1

Assemble your CORE

1. Choose the standard or pediatric tubing connector and screw into the top of the CORE, the end closest to the power toggle
2. Attach the chest piece to the tubing connector
3. Choose either the standard or pediatric barb to screw into the base of the CORE
4. Attach your stethoscope tubing to the barb
5. Slide the On / Off toggle to power the device. CORE will only deliver amplified sound in the ON position

Your CORE is now ready to be used in Analog or Digital mode.

Step 2 (optional)

Connect CORE to the Eko App

1. Download and install the App
2. Create an account, and log in
3. Turn on the CORE
4. Follow the prompts to connect your device and select the CORE

With the app installed, you can now visualize auscultation sounds, create recordings, save, and share patient data.

App Store is a trademark of Apple Inc.
Google Play and the Google Play logo are trademarks of Google LLC.

Eko

LBL055 Rev C.0

#\$TAB

D # \$TAB

2020-11-02

In the Box

CORE, pediatric and standard tubing connectors, and USB charging cable.

Compatibility

CORE is designed and tested to work with most acoustic stethoscopes.

User Manual and Support

Review the user manual before using your Eko CORE.

To view the manual visit: ekohealth.com/getstarted

For support, please email: contact@ekohealth.com

Charging

Connect the micro USB cable to the CORE and use a UL-certified or CE-marked USB wall charger.

Amplification

CORE provides up to 40x amplification for hearing support. CORE does not connect wirelessly to hearing aids.

Active Noise Cancellation

The CORE includes active noise cancellation to remove ambient noise from the environment. Active noise cancellation can be turned on or off through the Eko app.

(iii) CORE is ON & seeking device
○ CORE is ON & connected

● CORE is OFF & charging
(iv) CORE is LOW on battery

(v) CORE is recording
● CORE is fully charged

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