


Eko CORE

English	4 – 5
Spanish	6 – 7
French	8 – 9
Portuguese	10 – 11
German	12 – 13
Turkish	14 – 15
Italian	16 – 17
Polish	18 – 19
Romanian	20 – 21
Dutch	22 – 23
Greek	24 – 25
Czech	26 – 27
Swedish	28 – 29
Hungarian	30 – 31
Bulgarian	32 – 33

Danish	34 – 35
Finnish	36 – 37
Slovak	38 – 39
Norwegian	40 – 41
Irish	42 – 43
Croatian	44 – 45
Albanian	46 – 47
Lithuanian	48 – 49
Macedonian	50 – 51
Slovenian	52 – 53
Latvian	54 – 55
Estonian	56 – 57
Montenegrin	58 – 59
Maltese	60 – 61

Quick Start Guide

English

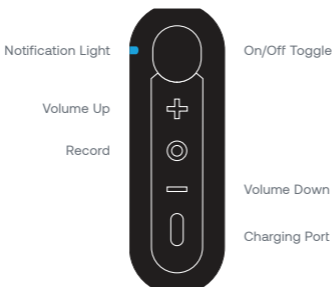

Step 1

Assemble your CORE

1. Choose the standard or pediatric tubing connector and screw into the top of the CORE, the end closest to the power toggle
2. Attach the chest piece to the tubing connector
3. Choose either the standard or pediatric barb to screw into the base of the CORE
4. Attach your stethoscope tubing to the barb
5. Slide the On / Off toggle to power the device. CORE will only deliver amplified sound in the ON position

Your CORE is now ready to be used in Analog or Digital mode.

Step 2 (optional)

Connect CORE to the Eko App

1. Download and install the App
2. Create an account, and log in
3. Turn on the CORE
4. Follow the prompts to connect your device and select the CORE

With the app installed, you can now visualize auscultation sounds, create recordings, save, and share patient data.



App Store is a trademark of Apple Inc.
Google Play and the Google Play logo are trademarks of Google LLC.

Chestpiece

Tubing connector

CORE

Barb

Stethoscope Tubing



 CORE is ON & seeking device

 CORE is ON & connected

 CORE is OFF & charging

 CORE is LOW on battery

 CORE is recording

 CORE is fully charged

In the Box

CORE, pediatric and standard tubing connectors, and USB charging cable.

Compatibility

CORE is designed and tested to work with most acoustic stethoscopes.

User Manual and Support

Review the user manual before using your Eko CORE.

To view the manual visit:
ekohealth.com/getstarted

For support, please email:
contact@ekohealth.com

Charging

Connect the micro USB cable to the CORE and use a UL-certified or CE-marked USB wall charger.

Amplification

CORE provides up to 40x amplification for hearing support. CORE does not connect wirelessly to hearing aids.

Active Noise Cancellation


The CORE includes active noise cancellation to remove ambient noise from the environment. Active noise cancellation can be turned on or off through the Eko app.



Eko Devices, Inc.
1212 Broadway, Suite 100
Oakland, CA 94612 USA
www.ekohealth.com



0537

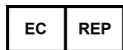

Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands