

CORE

Quick Start Guide

Eko


Table of Contents

Languages

English	p. 4	Dansk	p. 34
Español	p. 6	Suomi	p. 36
Français	p. 8	Slovensky	p. 38
Português	p. 10	Norsk	p. 40
Deutsch	p. 12	Gaeilge	p. 42
Türkçe	p. 14	Hrvatski	p. 44
Italiano	p. 16	Shqip	p. 46
Polski	p. 18	Lietuvių	p. 48
Română	p. 20	Македонски	p. 50
Nederlands	p. 22	Slovenščina	p. 52
Ελληνικά	p. 24	Latviešu	p. 54
Čeština	p. 26	Eesti	p. 56
Svenska	p. 28	Crnogorski	p. 58
Magyar	p. 30	Malti	p. 60
Български език	p. 32		

Quick Start Guide

English

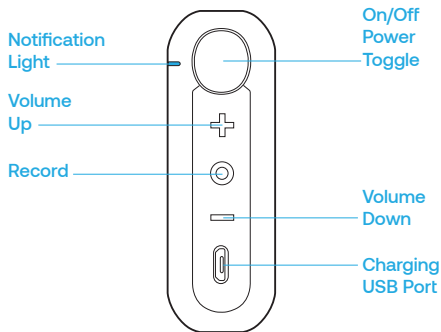

Eko App

Scan the QR code below to download the Eko app or search "Eko App" in the Apple App Store® or Google Play™ store.



Scan with your phone's camera to download the Eko App


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




It is important to have the Eko App ready to complete the set up of your CORE.

The Eko App is required for: wireless listening, advanced sound control and filtering, automated murmur detection*, recording, and telehealth live streaming.

Inside the Box

CORE digital attachment, pediatric and standard tubing and barb connectors, Micro-USB charging cable and Quick Start Guide.

CORE Modes and Corresponding LED States

	CORE is on and seeking device
	CORE is on & connected
	CORE is recording
	CORE is low on battery
	CORE is off & charging
	CORE is fully charged

Assemble your CORE


Step 1: Remove the chest piece from your stethoscope by pulling the chest piece and tubing apart.

Step 2: Screw the tubing connector into the CORE and then insert the chest piece (for pediatric stethoscopes, use the smaller connector).

Step 3: Screw the barb connector into the bottom of the CORE. Insert that barb into the stethoscope/earpiece tubing.

To Power ON and OFF

Press the power toggle to turn the CORE ON and OFF. OFF is when the toggle is protruding from the surface with the volume buttons. ON is when the toggle is flush with the surface with the volume buttons.

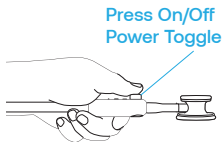


User manual and support

Review the user manual before using your Eko CORE.

To view the manual visit: support.ekohealth.com

For support, please visit: support.ekohealth.com

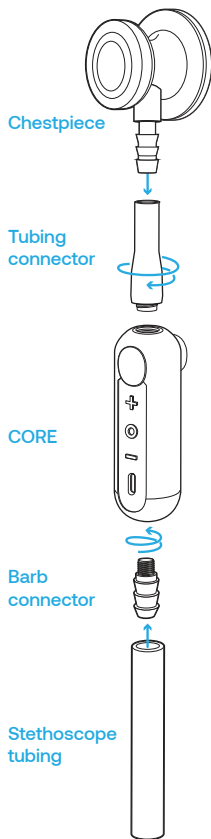

*May require a subscription to the Eko Analysis Software

Please see the full Instructions for Use at
support.ekohealth.com



Eko Devices, Inc.
1212 Broadway, Suite 100
Oakland, CA 94612 USA
www.ekohealth.com



Emergo Europe
Prinsessegracht 20
2514 AP The Hague
The Netherlands

Eko

TM and ©2021 Eko Devices Inc.
LBL 055 Rev E - September, 2021