# Littmann® CORE Stethoscope

Connects to Eko Software

Eko

3M

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# 3M™ Littmann® CORE Digital Stethoscope

## Quick Start Guide

Charge the Littmann CORE stethoscope prior to first use. After it is charged, your stethoscope is ready to use.

Eko

Download the Eko App (where available) from the Apple App Store® or the Google Play Store® to access additional features of the Littmann CORE stethoscope including wireless listening, advanced sound and filtering control, automated cardiac disease detection (where available, may require additional subscription), as well as saving and sharing recordings.

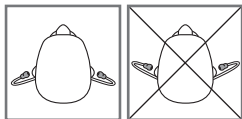
Scan with your phone's camera to download the Eko App.

## In the Box

- Eko CORE Digital Attachment and 3M™ Littmann® Cardiology IV™ Stethoscope
- Additional set of 3M Littmann Snap Tight Soft-Sealing Eartips (small)
- Nonchill Bell Sleeve
- USB Charging Cable
- Quick Start Guide

## Eartips

Insert eartips in a forward direction.

## Volume

Use the (+) and (-) buttons to adjust volume.  
There are 7 volume steps.

## **Turn On and Off**

When the CORE stethoscope is off, it functions as an unamplified stethoscope.

To turn the CORE stethoscope on, press the power toggle in. The light will begin flashing. You will now hear amplified and filtered sound in the eartips.

## **Telemedicine and AI Features**

Eko offers telemedicine and AI features that require a software subscription (where available, sold separately).

You can setup a subscription online at [ekohealth.com](https://ekohealth.com)

## **Active Noise Cancelling**

The CORE stethoscope includes active noise cancelling to remove ambient noise from the surrounding environment. Active noise cancelling can be turned off or on through the Eko App.

## Notifications

(((o))) (blinking)

CORE  
stethoscope is  
on & seeking  
device

o

CORE  
stethoscope is  
on & connected

● (yellow)

CORE  
stethoscope is  
off & charging

● (green)

CORE  
stethoscope is  
fully charged

(●●) (yellow)

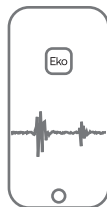
CORE  
stethoscope is  
low on battery

(●●) (blue)

CORE  
stethoscope is  
recording

## Recording

When connected to the Eko App, press  
the (o) button to begin recording.

## Charging

Connect the micro USB cable to the CORE Stethoscope and use a UL-certified or CE-marked 5V USB wall charger. The CORE stethoscope will take about 2 hours to fully charge. The light will turn green when it is fully charged.

## User Manual

Review the user manual before using your Littmann CORE Stethoscope. To view the manual visit:  
[ekohealth.com/getstarted](http://ekohealth.com/getstarted)

For more information on 3M devices,  
see [HCBGregulatory.3M.com](http://HCBGregulatory.3M.com)

## Service and Support

In the USA, please visit [littmann.com/service](http://littmann.com/service) or call 1-800-228-3957. If you are outside of the USA, please visit [littmann.com](http://littmann.com) for the contact information of your local 3M office.

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Stethoscope Made in the USA  
CORE Digital Attachment Made in Malaysia  
Assembled and distributed by 3M Company for

▮ Eko Devices, Inc.

1212 Broadway, Suite 100 Oakland, CA 94612 USA  
1-800-228-3957 (USA only) 1-844-356-3384 [www.littmann.com](http://www.littmann.com)

©2022 3M; Eko Devices Inc. All rights reserved.

3M, Littmann, the L Littmann logo, the L, and Cardiology IV are marks and/or registered marks of 3M. Unauthorized use prohibited. Used under license in Canada.

Eko trademarks, including CORE, are marks and/or registered marks of Eko Devices Inc.

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© 3M, 2022; Eko Devices Inc. Tous droits réservés.
Les marques de commerce Eko, y compris CORE, sont des marques et/ou des marques déposées d'Eko Devices Inc.