WARNING

- Breathe™ is designed to clean stuffy noses; do not use it in baby's mouth or ears.
- Do not insert the silicone tip into the nostril too deeply. Do not use it on the baby for a long time. A few seconds will be enough.
- Do not use breathe™ if there are preexisting wounds, inflammation, or bleeding from the nose.
- Do not allow children to play with this device. It is not a toy. Keep the device out of children's reach to prevent small parts and accessories from being swallowed accidentally.
- Breathe™ is intended for personal use only. This device is not meant for commercial use.
- Breathe™ should always be operated by an adult or used under adult supervision.
- Before use, make sure the silicone tip is firmly in place on the collection reservoir.
- Clean breathe™ before and after use per the instructions provided in the Wash & Care section.
- Breathe™ is not a waterproof device, please follow cleaning instructions provided in the Wash & Care section.

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Limited Product Warranty

ZoLi Inc., the distributor of the breathe™ nasal aspirator, warrants to the original owner of this product that this product is free from defects in materials and workmanship for a period of **6 (6) months** from the date of the original purchase. If this aspirator fails to function properly when used as directed under normal conditions within a one-year period, ZoLi Inc. will either repair or replace the product, at its option free of charge, when it is returned in its original packaging, postage prepaid, with proof of purchase and a Return Merchandise Authorization (RMA) number. ZoLi Inc. will bear the cost of repairing or replacing the product and shipping it back to you. Repair or replacement of the defective item is subject to verification of the malfunction or defect due to normal usage when delivered to:

ZoLi Inc. Service Center
1801 Murchinson Drive #204
Burlingame, CA 94010

Note: The warrant expressly disclaims liability for incidental, consequential, or special damages of any nature. This warranty does not cover 1) a defect that has been caused by repair or modification by any person other than a ZoLi technician; 2) a defect caused by negligence, accident, or misuse of the product; 3) a defect caused by incorrect installation of and/or defective batteries; 4) battery life; or 5) damage resulting from improper operation.

LEGAL RIGHTS VARY FROM STATE TO STATE

If you have any questions, please contact ZoLi Customer Care at 1.888.321.ZOLI (9654) Mon-Fri from 9:00AM-5:00PM PST.

The breathe™ nasal aspirator is a personal care product and cannot be returned to the retailer. All exchanges and repair requests should be directed to the ZoLi Customer Care at 1.888.321.ZOLI (9654) Mon-Fri from 9:00AM-5:00PM PST. No product returns will be accepted without first obtaining a Return Merchandise Authorization (RMA) number from ZoLi.

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3

Contents

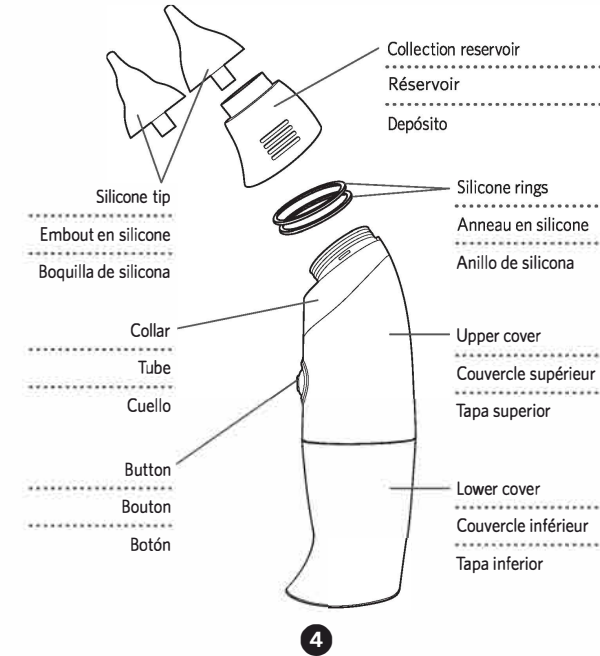
1 main unit / 2 sizes of silicone tips / 1 instruction manual / 2 spare silicone ring

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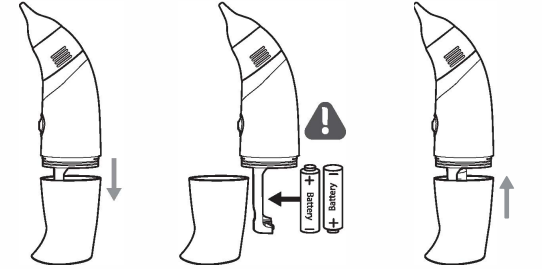
Instructions for use / Mode d'emploi / Instrucciones de uso

- Separate the upper cover from the lower cover in a straight direction. Do not bend the cover by using force, as it can be slid open easily.
- Insert two AA alkaline batteries with the correct polarity as shown.
- Check to make sure the two silicone rings are properly seated on the grooves. Close the covers until they click back into place.

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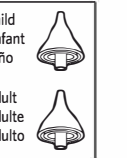
Choose the appropriate silicone tip to use based on the size and shape of your baby's nostrils, as well as the level of congestion.

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5

- Place the silicone tip securely on the collection reservoir.

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- Hold baby upright and insert the tip into baby's nose. Do not place tip too deep into baby's nostril or press against the wall of nostril. Place the tip at the outer side of baby's nostril at a 90-degree angle.

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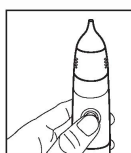
- Press the power button to start the gentle, steady suction to clear congestion.

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- Release the power button and remove from nostril when you are done.

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- Clean thoroughly as indicated in the Wash & Care section below.

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6

Note: The mucus may not be immediately visible in the collection reservoir while you are suctioning. However, you will see it once you remove the reservoir to clean. If the accumulated volume of mucus is nearly over the safe capacity of the collection reservoir (about 4 ml), follow the instructions in the Wash & Care section. Do not use this device if the mucus level is nearly over the opening channel.

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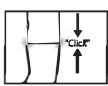
- Make sure that the upper and lower covers are closed firmly before cleaning the device.

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Retirez l'embout en silicone.

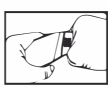
Retire la boquilla de silicona.

- Hold collection reservoir between your thumb and index finger.

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- Bend the collection reservoir downwards gently to remove it from the collar.

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- Remove the silicone rings from the collar for cleaning. Note: the silicone rings do not need to be cleaned every time, clean after several uses.

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- Silicone ring, silicone tips and collection cup may be sterilized by placing in boiling water or sterilizer. We recommend sterilizing for no longer than 2 minutes.

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- Dry all parts thoroughly before assembling them. Reassemble all parts as shown.

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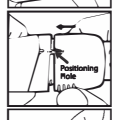
- Make sure all parts fit tightly together on the device.

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
When the accumulated volume of mucus is over the safe capacity of the collection reservoir during use of the device or if you see mucus dripping out of the air vent on the back side of the collar, please follow instruction below.

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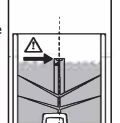
- Be sure the upper and lower covers are closed firmly.

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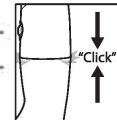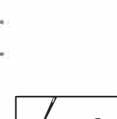
- Have the collar immersed in clean water as shown. Press the button for 15 seconds and the inner pump will be cleaned.

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- The water flows in and out of the inner pump in the direction as shown.

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- Change the water and repeat step 3 several times if necessary.

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- Remove the device from the water once clean. Keep pressing the button for approximately 30 seconds until the inner pump is dried.

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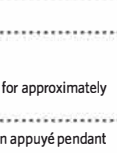
- Dry the collar and reassemble all the parts.

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operational manual / mode d'emploi / manual de uso

INTENDED USE: quickly and gently remove excess mucus. Please read the following instructions carefully to ensure proper use. Retain these instructions in a safe place for future reference.

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