


Elite Ear®

Elite Ear® is a sound amplifier that helps to amplify sound up to 30 times, with no worries of having to replace batteries! It is equipped with a charger and a rechargeable battery that can last up to 40 hours of continuous use. It includes 6 interchangeable ear tips to SEAL OUT the noise!

Set Includes

- ELITE EAR® Earpiece
- Adapter
- Charging Dock
- Cleaning Brush
- Six (6) Silicone Adjustable Tips

Diagram 1

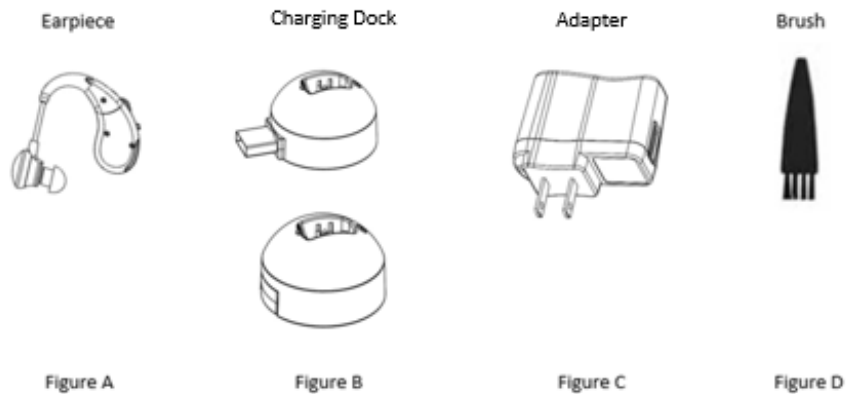

CAUTION:


- Elite Ear® is NOT for use by children under 15.
- Keep out of Reach of children and pets
- The supply terminals are not to be short-circuited
- DO NOT wet
- **Elite Ear® should be charged ONLY with the adapter and charging dock station included in the set. Using another adapter, charger or charging cord may result in reduced life, overheating, or damage of the charging dock station, adapter, charging cord, rechargeable battery, and/or damage to the Elite Ear® product.**

Note: Discontinue use of the Elite Ear® in the event of irritation. It is recommended to consult a doctor before continued use of this product.

Getting Started

It is important to follow the instructions (steps 1-4) below for the first two charging times of the Elite Ear® Sound Amplifier:

1. Must slide the switch down located on the back of the earpiece to turn the Elite Ear® OFF.
2. Connect the adapter (Diagram 1: Figure C) plug into the charging dock (Diagram 1: Figure B).

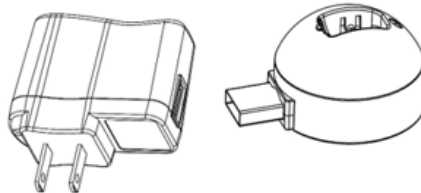

3. Plug adapter into an outlet, the red LED light on the charging dock (Diagram 1: Figure B) will illuminate and the red light on the adapter (Diagram 1: Figure C) will light and quickly flash simultaneously.
4. Place the Elite Ear® (Diagram 1: Figure A) into the charging dock.



5. The red LED light on the adapter (Diagram 1: Figure C) will change to slow flash immediately



6. The red LED light on the charging dock (Diagram 1: Figure B) will auto cutoff if fully charged

NOTE: DO NOT REMOVE THE ELITE EAR® (Diagram 1: Figure A) FROM THE CHARGING DOCK UNTIL THE RED LED LIGHT AUTO CUTOFF (AGAIN), SIGNALING FULLY CHARGED.

Important: Follow these steps to ensure a longer life of the Elite Ear®.

Directions:

How to wear the Elite Ear®

Note: One (1) silicone tip is already on the Elite Ear®

- The Elite Ear® includes 6 different silicone ear tips. Select the silicone tip that best fits the ear and place onto the Elite Ear®
- Elite Ear® can be worn on either the left or right ear
- On the ear piece, make sure the volume tab is in position “1” before use
- Below the volume wheel is the ON/OFF switch. This must be in the ON position for use. To turn the device OFF slide switch down.
- Place the device over the top of the desired ear, the device will curve around and fit behind the ear
- Carefully insert the silicone tip of the Elite Ear®, which is at the end of the small wire into the ear canal
- Adjust the volume to the desired level

NOTE: THE DEVICE WHISTLES IF THE VOLUME LEVEL IS TOO HIGH, ADJUST VOLUME TO COMFORT LEVEL

NOTE: CHARGING TIME WILL VARY DEPENDING UPON THE LEVEL OF BATTERY POWER REMAINING PRIOR TO RE-CHARGING(WILL BE APPROXIMATELY 8+ HOURS)

General Care:

- Clean the Elite Ear® daily with the provided brush to avoid unwanted earwax build up
- Brush the device back and forth across the ear bud using the supplied brush to remove wax and dirt deposited within the ear bud
- Make sure the ear bud is clear of all debris and dirt
- Wipe the Elite Ear® with a clean, dry cloth only, DO NOT WET
- There are no serviceable parts to this item

Storage:

When Elite Ear® is not in use, clean it and store in a dry cool place

Troubleshooting:

IMPORTANT INFORMATION:

It is recommended consulting a certified physician before deciding to wear Elite Ear® and to follow up with constant checkups to ensure none of the listed condition below apply

DO NOT wear Elite Ear® if the following condition persist:

- Have pain and discomfort in the ear
- Have traumatic or visible congenial deformities on your ear
- Have acute or chronic dizziness
- Have history of sudden or rapidly progressive hearing loss that has occurred within the last 90 days
- Have significant build up of wax or other material in the ear canal
- Bone conduction thresholds are better than the air conduction thresholds by 15dB or greater

Specification

- Battery: 1.2V
- Frequency: 100 ~ 4000Hz
- Working Current : $\leq 1\text{mA}$
- Signal to Noise Ratio: $\leq 28\text{dB}$
- Distortion: $\leq 6\text{dB}$

PLEASE KEEP THESE INSTRUCTIONS IN A SAFE PLACE FOR FUTURE REFERENCE

DUE TO CONTINUOUS PRODUCT IMPROVEMENT THE PICTURE ON THE PACKAGE MAY DIFFER SLIGHTLY FROM THE ACTUAL PRODUCT

The Elite Ear® is not a substitute for a hearing aid or intended for use by hearing impaired people, but rather is intended for non-hearing-impaired consumers to amplify sounds in the environment for a number of reasons, such as for the recreational activities. The FDA advises anyone who suspects hearing loss to seek evaluation by health care professionals.

Designed in USA

Assembled in China

Distributed by: Global TV Concepts, LTD.

Deerfield Beach, FL 33442

©2018 Global Concepts LTD. All Rights Reserved

All photographs & graphics are the property of their respective owner. Infringers will be prosecuted.