


## Hear-acle™ Sound Amplifier

Hear-acle™ is a sound amplifier that helps amplify sound up to 45 decibels. The Hear-acle™ includes 3 interchangeable silicone cushioned tips, volume control bar, carrying case and 1 battery for up to 75 hours of continuous use.

### Set Includes

- Hear-acle™ ear piece unit
- 3 Interchangeable silicone cushioned tips
- Volume Control Bar
- Travel Case

**Diagram 1**

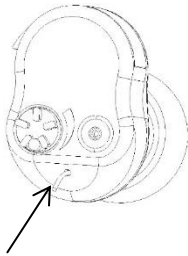


(Use this piece to easily remove Hear-acle™ from the ear)

### Directions


*Note: One (1) silicone-cushioned tip is already attached to the Hear-acle™ earpiece.*

- Hear-acle™ can be worn in either the left or the right ear.
- Silicone cushioned tips can be changed by removing one silicone cushioned tip with a simple pull and pressing on a new tip choosing one with the most comfortable and desired fit.
- Open the cartridge cover and place battery inside in the correct direction. (diagram 2 below)
- Turn the Hear-acle™ earpiece unit ON by turning the volume wheel using the volume control bar (diagram 3 below) slowly and starting at the lowest volume position. Adjust the volume by continuing to turn the volume wheel with the volume control bar to preferred volume. (diagram 3 below)

**Diagram 2**

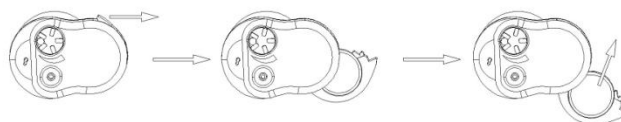


**Diagram 3**

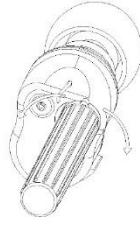

**IMPORTANT NOTES:**

- If whistling sound can be heard, please check to ensure Hear-acle™ is snug but comfortable inside the ear. If not, change to another silicone cushioned tip.
- NEVER push Hear-acle™ into the ear.
- When Hear-acle™ is not in use, please open the battery door or take out the battery to prolong the use of the battery. (Diagram 4 below) when this is not done, the unit will remain on and the battery will drain.

**Diagram 4**

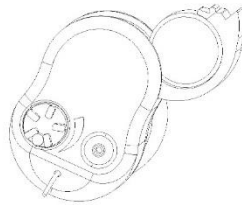

General Care

- Clean Hear-acle™ daily to avoid unwanted earwax build-up.
- Make sure the ear bud is clear of all debris and dirt.
- Wipe the Hear-acle™ with a clean, dry cloth only. NEVER wet. Note: There are no serviceable parts to this item.

Storage

- When Hear-acle™ is not in use, clean and store in a cool dry place.

Troubleshooting

**IMPORTANT INFORMATION**

It is recommended to consult a certified physician before deciding to wear Hear-acle™ and to follow up with constant checkups to ensure none of the listed conditions below apply.

Do not use/wear if any of the below conditions apply:

- Have pain and discomfort in the ear.
- Have traumatic or visible congenital deformities on the ear.
- Have acute or chronic dizziness.
- Have history of sudden or rapidly progressive hearing loss that has occurred within the last 90 days.
- Have unilateral hearing loss or sudden/recent onset within the last 90 days.
- Have significant build-up of wax or other materials in your ear canal.
- Bone conduction thresholds are better than the air conduction thresholds by 15dB or greater.

Specifications:

Max. Sound Output: 120+\_5dB

Sound Gain up to 45 dB

Total harmonic Wave Distortion equal to or less than 5%

Frequency range: 200 to 4000 Hz

Input Noise: equal to or less than 30%

Voltage: D.C. 1.5V

Current equal to or less than 4.5mA

Frequency Range : 150—4600Hz

Distortion : 3.8%

PLEASE KEEP THESE INSTRUCTIONS IN A SAFE PLACE FOR FUTURE REFERENCE.

DUE TO CONTINUOUS PRODUCT IMPROVEMENT, THE PICTURE ON THE PACKAGING MAY DIFFER SLIGHTLY FROM THE ACTUAL PRODUCT.

The Hear-acle™ is not a substitute for a hearing aid or intended for use by the hearing impaired. The FDA advises anyone who suspects hearing loss to seek evaluation by a health care professional.

Made Responsibly in China

Distributed by:

Global TV Concepts, LTD. Deerfield Beach, FL  
33442

©2016 Global Concepts LTD. All Rights Reserved