




MSA Lil' Listeners™

MSA Lil' Listeners™ is a sound amplifier that helps to amplify sound up to 30 times, with no need to replace batteries! It is equipped with a charger and a rechargeable battery that can last up to 40 hours of continuous use. It includes 6 interchangeable ear tips to SEAL OUT the noise!

Set Includes

- MSA LIL' LISTENERS™ Earpiece
- Adapter
- Charging Dock Station
- Cleaning Brush
- Six (6) Silicone Adjustable Tips

Diagram 1

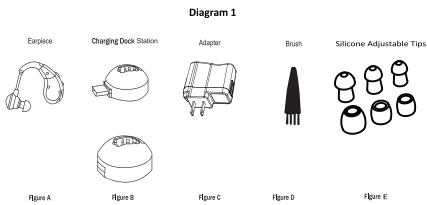

CAUTION:

- **WARNING – Possible Choking Hazard – Small parts may cause choking risk for children under 3 years**
- **Adult Supervision required**
- **The supply terminals are not to be short-circuited**
- **DO NOT expose to water and KEEP AWAY from any liquid**
- **MSA Lil' Listeners™ should be charged ONLY with the adapter and charging dock station included in the set. Using another adapter, charger or charging cord may result in reduced life, overheating, or damage of the charging dock station, adapter, charging cord, rechargeable battery, and/or damage to the MSA Lil' Listeners™ product.**



WARNING:

CHOKING HAZARD — Small parts not for children under 3 years or any individuals who have a tendency to place inedible objects in their mouths




Note: Discontinue use of the MSA Lil' Listeners™ in the event of irritation. It is recommended to consult a doctor before continued use of this product.

Getting Started

It is important to follow the instructions (steps 1-4) below for the first two charging times of the MSA Lil' Listeners™ Sound Amplifier:

1. Slide ON/OFF switch down located on the back of the earpiece to turn the MSA Lil' Listeners™ OFF.
2. Connect the adapter (Diagram 1: Figure C) and the charging dock station (Diagram 1: Figure B).

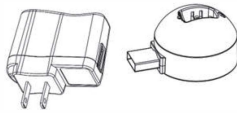

3. Plug the adapter into an outlet, immediately the red light on the adapter (Diagram 1: Figure C) will illuminate. Simultaneously the red light on the charging dock station will also illuminate, with an initial quick flash (Diagram 1: Figure B).
4. Place the MSA Lil' Listeners™ (Diagram 1: Figure A) into the charging dock station.



5. Once the MSA Lil' Listeners™ earpiece is inserted into the charging dock station, the red light will start to slowly flash.



6. When the MSA Lil' Listeners™ earpiece is fully charged the red light will automatically turn off.

NOTE: DO NOT REMOVE THE MSA LIL' LISTENERS™ (Diagram 1: Figure A) FROM THE CHARGING DOCK STATION UNTIL THE RED LIGHT TURNS OFF INDICATING A FULL CHARGE.

Important: Follow these steps to ensure a longer life of the MSA Lil' Listeners™.



Directions:

How to wear the MSA Lil' Listener™ earpiece:

Note: One (1) silicone ear tip is already placed on the MSA Lil' Listeners™ inside the packaging

- The MSA Lil' Listeners™ includes 6 different silicone ear tips. Select the silicone tip that best fits the ear and place onto the MSA Lil' Listeners™
- MSA Lil' Listeners™ can be worn on either the left or right ear
- On the earpiece, make sure the volume tab is in position "1" before use
- Below the volume wheel is the ON/OFF switch. This must be in the ON position for use. To turn the device OFF slide switch down
- Place the earpiece over the top of the desired ear, the earpiece will curve around and fit behind the ear
- Carefully insert the silicone tip of the MSA Lil' Listeners™, which is at the end of the thin tube into the ear canal
- Adjust the volume to the desired level

NOTE: THE MSA Lil' Listener™ WHISTLES IF THE VOLUME LEVEL IS TOO HIGH, ADJUST VOLUME TO COMFORT LEVEL

NOTE: CHARGING TIME WILL VARY DEPENDING UPON THE LEVEL OF BATTERY POWER REMAINING PRIOR TO RE-CHARGING (WILL BE APPROXIMATELY 4+ HOURS FOR A FULL CHARGE)

General Care:

- Clean the MSA Lil' Listeners™ daily with the provided brush to avoid unwanted earwax build up
- Brush the device back and forth across the ear bud using the supplied brush to remove wax and dirt deposited within the ear bud
- Make sure the ear bud is clear of all debris and dirt
- Wipe the MSA Lil' Listeners™ with a clean, dry cloth only, DO NOT WET
- There are no serviceable parts to this item

Storage:

When MSA Lil' Listeners™ is not in use, clean it and store in a dry cool place



Troubleshooting:

IMPORTANT INFORMATION:

It is recommended consulting a certified physician before deciding to wear MSA Lil' Listeners™ and to follow up with constant checkups to ensure none of the listed condition below apply:

DO NOT wear MSA Lil' Listeners™ if the following conditions are present or occur:

- Have pain and discomfort in the ear
- Have traumatic or visible congenial deformities on your ear
- Have acute or chronic dizziness
- Have history of sudden or rapidly progressive hearing loss that has occurred within the last 90 days
- Have significant buildup of wax or other material in the ear canal
- Bone conduction thresholds are better than the air conduction thresholds by 15dB or greater

Specification

- Battery: 1.2V
- Frequency: 100 ~ 4000Hz
- Working Current: $\leq 1\text{mA}$
- Signal to Noise Ration: $\leq 28\text{dB}$
- Distortion: $\leq 6\text{dB}$

PLEASE KEEP THESE INSTRUCTIONS IN A SAFE PLACE FOR FUTURE REFERENCE

DUE TO CONTINUIOUS PRODUCT IMPROVENET THE PICTURE ON THE PACKAGE MAY DIFFER SLIGHTLY FROM THE ACTUAL PRODUCT

The MSA Lil' Listeners™ is not a substitute for a hearing aid or intended for use by hearing impaired people, but rather is intended for non-hearing-impaired consumers to amplify sounds in the environment for several reasons, such as for the recreational activities. The FDA advises anyone who suspects hearing loss to seek evaluation by health care professionals.

Designed in USA

Assembled in China

Distributed by: Global Concepts Limited, Inc.

Coconut Creek, FL 33073

©2018 Global Concepts LTD. All Rights Reserved

All photographs & graphics are the property of their respective owner. Infringers will be prosecuted.