## MSA SoundOut™ Earplugs

#### What is included:

- (2) MSA SoundOut™ Earplugs
- (3) Sets of MSA Accoustic Filters

Hexagon Shaped Filter: Mild Sound Protection
 Triangle Shaped Filter: Medium Sound Protection
 Circle Shaped Filter: Maximum Sound Protection




### Warning:

- Carefully read and follow these instructions and always insert MSA SoundOut™ earplugs until snug and comfortable
- DO NOT OVER INSERT INTO THE EAR
- MSA SoundOut<sup>™</sup> earplugs are designed to provide discrete, comfortable hearing protection against snoring
- NEVER use MSA SoundOut<sup>™</sup> if you feel that your ears simply will not accommodate a good comfortable fit
- DO NOT use MSA SoundOut<sup>™</sup> when intense impulse noise situations i.e. heavy machinery such as jack hammers or firearms
- If you cannot remove the MSA SoundOut<sup>™</sup> earplug or it has become damaged in the ear canal, IMMEDIATELY seek doctors assistance
- Effectiveness of this product relies on the individual following these instructions and varying factors, such as size of the user's ear canal
- Keep earplugs away from infants and small children
- ALWAYS wear the MSA SoundOut<sup>™</sup> as intended ensuring that they are fitted, adjusted and maintained as intended and as directed in these instructions
- ONLY use as directed

- NEVER place the MSA SoundOut<sup>™</sup> earplug too deep in the ear canal. Improper fit, use or failure to follow these instructions will decrease the effectiveness of this product, may increase the risk of hearing damage and affect the lifespan of this product
- User of these earplugs assumes all risk and full responsibility for any and all results or situations that may arise with the use of these earplugs

#### How to Use:


To insert and remove the MSA SoundOut<sup>™</sup> earplugs properly and safely, please follow the instructions and images below:

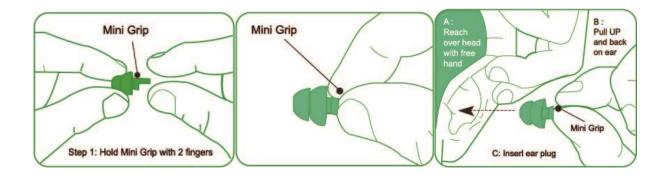

Securely hold the Mini Grip between index finger and thumb as illustrated above, making sure the Mimi Grip is on top.

- It is critical to position the Mini Grip on top as you insert the MSA SoundOut™ earplug. This will allow for easy removal of the earplug from the ear.
- Reach over head with opposite hand and gently pull up and back on the top outer ear. This allows the ear canal to straighten. Insert the MSA SoundOut™ until comfortably snug, using the finger to gently apply pressure against the plastic filter to guide the earplug into the ear. Wiggling the earplug may help properly position.
- Pulling up on the ear straightens the ear canal and is necessary for a comfortable and proper fit. Without this step the MSA SoundOut<sup>™</sup> earplugs will not provide maximum protection.
  - NEVER release the Mini Grip until the earplug has successfully been positioned into the ear
  - NEVER push the earplug further into the ear after releasing the Mini Grip
  - Be Gentle and NEVER over-insert
- Clean after each use with a damp cloth

# HOW TO CHANGE MSA ACCOUSTIC FILTERS IN YOUR MSA SOUNDOUT™ EARPLUGS

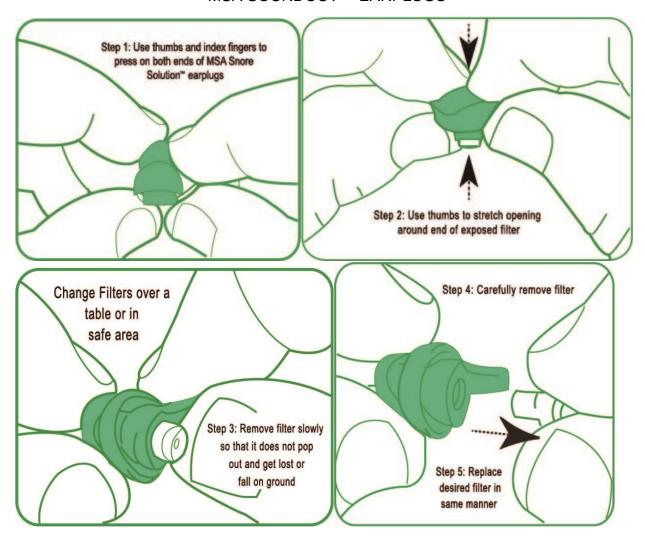

# HOW DO YOU KNOW THE MSA SOUNDOUT™ EARPLUGS are inserted CORRECTLY?

All ears are different and it could take some time and practice inserting the MSA SoundOut™ earplugs to get it just right.

• MSA SoundOut™ earplugs are designed to fit comfortably and snugly in your ear while being almost invisible.

• **NEVER** over insert MSA SoundOut<sup>™</sup> earplugs and NEVER push the earplugs further into the ear canal after releasing the Mini Grip.

#### **CLEANING:**

MSA SoundOut<sup>™</sup> earplugs are easy to clean and maintain. Simply wipe the earplugs off with a damp cloth after each use. Keep them stored in the included travel case. If they become particularly dirty, they can be washed in warm soapy water. However, be sure to remove the filter before submerging the earplugs into water. Reinsert the filter only after the earplugs have dried thoroughly. The earplugs should be replaced at any sign of wear and tear.



Global Concepts Limited, Inc. is not liable for any damage arising from misuse.

Made in China
Designed in USA
Distributed by: Global TV Concepts, LTD. Deerfield
Beach, FL 33442
©2021 Global Concepts Limited, Inc. All Rights Reserved
All photographs & graphics are the property of their respective owner Infringers will be prosecuted