

It is very IMPORTANT for you to read this booklet before using **Turbo Ear™** sound amplifier. Step-by-step, you will learn how to use and care for this sound amplifier.

Technical Information

Battery: 1.2V
Working current: \cong 3mA
Signal to Noise Ration: \cong 28dB
Gain: 47dB (800Hz; \pm 4dB)
Distortion (800Hz) : \cong 8%
FDA Registration # 1063974

Product appearance and color may vary.

Made in China

Designed in USA

Distributed by: Global TV Concepts, LTD. Deerfield Beach,
FL 33442

©2008 Global Concepts Limited, Inc. All Rights Reserved All
photographs & graphics are the property of their respective
owner Infringers will be prosecuted

Do not wear Turbo Ear™ if:

- You have pain or discomfort in your ear
- You have traumatic or visible congenial deformities of the ear.
- You have acute or chronic dizziness.
- You have a history of sudden or rapidly progressing hearing loss that has occurred within the last 90 days.
- You have/had unilateral hearing loss or sudden/recent onset within the last 90 days.
- Your bone conduction thresholds are better than your air conduction thresholds by 15dB or greater.
- You have Significant build up of wax or other material in your ear canal .

Charging Instructions:

Turbo Ear™ is an air-induction device that fits inside the ear. This device can last up to seven continuous hours of use. Replacement batteries are not needed; the earpiece can be recharged up to 500 times in the included charging case.

Important: Follow steps 1-5 for the first three times you charge the Turbo Ear™:

- 1. Make sure the earpiece battery is completely drained.**
- 2. Move the switch to the off position on the back of the earpiece.**
- 3. Place earpiece in charging base.**
- 4. Make sure the cover is securely closed.**
- 5. Charge the earpiece overnight or for a minimum of 12 hours.**

Note: Even though the charging light might be off indicating the charging is complete, the earpiece must Remain in the charging case for a minimum of 12 hours.

Important: You must follow these steps to ensure the battery will have optimal memory for longer use.

Make sure the earpiece has been turned off before placing in the charging case and make sure the cover is securely closed.

Care and Maintenance


Cleaning your **Turbo Ear™** Daily:

1. With a soft bristled brush, brush the device gently back and forth across the receiver end to remove all the wax. This prevents wax buildup.
2. Make sure that the receiver end is cleared of all debris and dirt.
3. Wipe the case with a clean, dry cloth only.
DO NOT WET.



ON/OFF & Volume Control

Use your index finger to control the volume tab.

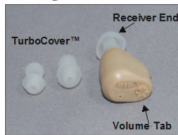

Slightly rotate the tab clockwise to increase the volume. To reduce volume, rotate the tab counter clockwise.

If you hear high pitched whistling sounds or feedback, this means that the volume is too high and you should reduce it. It could also mean that you device is loose. You can remedy this by adjusting the device or putting on the Turbo Cover, which is a “soft cover” that gives more comfort and a better fit. Insert **Turbo Ear™** into the cover by inserting the receiver until it flushes with the small opening. Make sure that the cover does not block the end of your **Turbo Ear™**.

Using Adaptor:

Remove the adaptor from the box and plug it into the back of the charging base. Next plug the adaptor into the outlet to start charging. When the charging red light turns to green the **Turbo Ear™** is charged and ready to use (except as noted for the first three times you charge your **Turbo Ear™**).

How to Wear **Turbo Ear™**

Hold the device between your thumb and forefinger in your right hand if you are placing the device in your right ear.

Make sure the volume tab is at position 1.

Position the **Turbo Ear™** with the volume tab at the lower side and the receiver end facing upward towards your ear. Be careful not to twist your hand when inserting **Turbo Ear™** and risk putting the device in at a wrong angle. The receiver should be gently placed into the ear and then further adjusted by lightly pushing the device.



Remember to turn off the device each time before you remove **Turbo Ear™**. As this prolongs the life of the battery. To remove **Turbo Ear™**, push upward under your ear with your forefinger. This action should release the device so that it can be easily seized by your fingers and removed. **NEVER** remove the device by the battery door or the volume tab and **NEVER** force insertion or removal.