

Helpful Hints:

1. **Soften Earwax:** Apply a few drops of saline solution, mineral oil or diluted hydrogen peroxide to the ear canal to help loosen and soften earwax.
2. **Soften Earwax:** Soak a cotton ball with a few drops of saline solution, mineral oil or diluted hydrogen peroxide and place in ear with head tilted and the ear pointing up. Keep in this position for a few minutes to allow gravity to pull the liquid down and through the ear wax.

NEVER:

1. Dig out Earwax: Never Attempt to dig excessive or hardened earwax out of the ear using paper clip, cotton swab or hair pin (for example). The wax could be pushed into the ear canal farther and cause serious damage.
2. Push the tip of the product too hard or too deep into the ear.


CAUTION:

Ask a doctor before use if any of the following conditions are present:


- ear pain
- irritation or rash in and/or around the ear
- dizziness
- an injury, wound or perforation of the ear drum
- recently had ear surgery
- When in use, please sit in a safe place
- Do not let children use WITHOUT adult supervision
- When removing wet earwax, please ensure the tip and filter are not blocked
- If there are abnormalities such as pain in the ear, please consult a professional doctor.

ONLY USE AS DIRECTED

DO NOT USE EAR DROPS if an ear infection is suspected, unless recommended by a doctor.

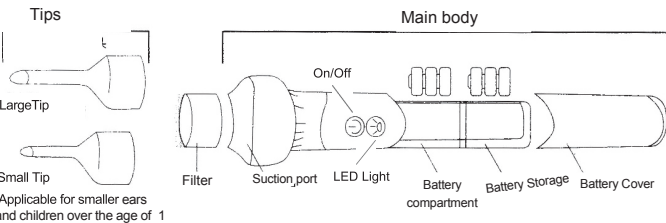

Large Soft Tip:

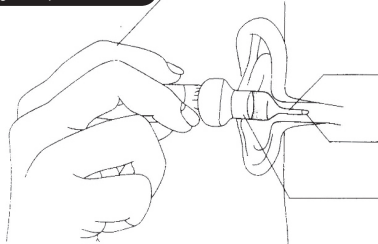


Always keep the product in a horizontal position (as shown) with the opening faced down

Gently insert the tip into the ear. Repeat when necessary

To ensure the ear is not harmed, NEVER press hard or push too deep into the ear.

Small Soft Tip:

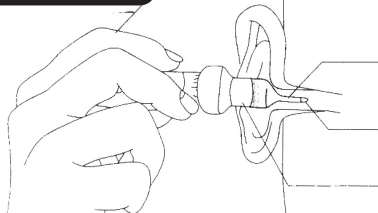

Always keep the product in a horizontal position (as shown) with the opening faced down

Gently insert the tip into the ear. Repeat when necessary

To ensure the ear is not harmed, NEVER press hard or push too deep into the ear.

To ensure the ear is not harmed, NEVER press hard or push too deep into the ear

Wax Extract™

Thank you for purchasing the Wax Extract™. Please read this manual and use the Wax Extract™ as directed. Please keep this manual for future reference.

The Wax Extract™ provides gentle suction and ear cleaning. The Wax Extract™ has LED light, powerful & gentle suction and includes 2 soft flexible tips, large and small.

Due to our policy of continuous product improvement, the appearance of the actual product may slightly differ than pictured.


Made in China
Distributed by: Global TV Concepts LTD, Deerfield Beach, FL 33442
©2017 Global TV Concepts LTD. All Right Reserved
All photography & graphic are the property of their respective owner, Infringer's will be prosecuted

0621-2931

Battery Installation:


1. Slide the battery cover downward (Figure 1)

Figure 1

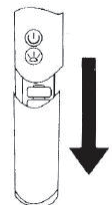

2. Place the button battery (LR44) into the battery compartment inside the main unit. (Figure 2) (Batteries will last up to 1 hour of continuous use)
3. Slide the battery cover back into place


Figure 2

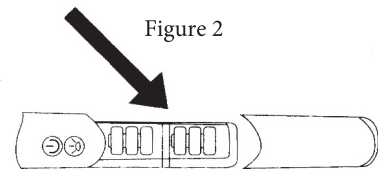

To Operate:

1. Place desired tip onto the main body (Figure 3)
2. To operate, press down and hold the ON/OFF button. If the LED light is needed, press down and hold the ON/OFF button for the LED light.
3. To turn off, press the ON/OFF button again.

Figure 3

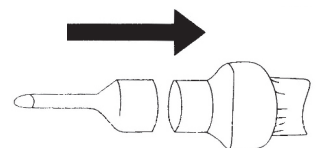

Cleaning & Maintenance:

1. Turn Product Off prior to cleaning
2. Clean tips with warm water and air dry after each use
3. The filter can be removed and cleaned with water and air dry
4. NEVER submerge the unit into water