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Duschhocker SC10+

MOBILITY **Plus+**

# MOBILITY Plus+

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# MOBILITY **Plus+**

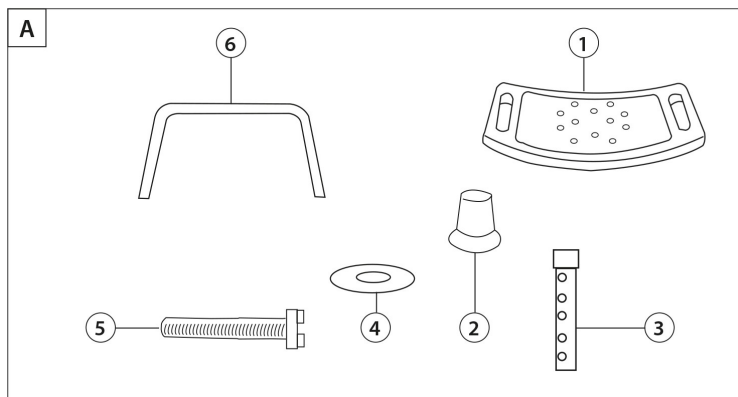
## **Contents**

<b>Items included in the delivery</b> .....	<b>3</b>
<b>General information.</b> .....	<b>4</b>
Reading and keeping the instruction manual.....	4
Signs and symbols.....	4
<b>Safety</b> .....	<b>5</b>
Intended Use.....	5
Safety information.....	5
<b>Check included items.</b> .....	<b>7</b>
<b>Installation of the shower stool.</b> .....	<b>8</b>
<b>Cleaning and maintenance</b> .....	<b>9</b>
<b>Technical data.</b> .....	<b>10</b>
<b>Disposal</b> .....	<b>10</b>
Disposal of packaging.....	10
Disposal of shower stool.....	10

# MOBILITY Plus+

## Items included in the delivery

**(1)** seat

**(2)** rubber feet, 4pcs

**(3)** foot tube, 4pcs

**(4)** washer, 5pcs

**(5)** cross-head screw (M6 x 35mm), 5pcs

**(6)** seating tube, 2pcs

# MOBILITY Plus+

## General information

Reading and keeping the instruction manual

This instruction manual concerns the delivered shower stool and embodies all important information for the installation and usage of the stool. Please read the instruction manual carefully, especially the safety information, before using the shower stool.

The non-observance of this manual may lead to serious injuries or damage of the stool. The instruction manual is based on the legal norms and requirements in the European Union. When using the stool overseas, please pay attention to the country-specific standards and laws. Keep the instruction manual for further use. When passing along the stool to third parties, absolutely include the instruction manual.

## Signs and symbols

The following symbols and signal words are used in this manual, on the stool or on its packaging.

This symbol/signal word indicates a danger of moderate risk level which may result in death or serious injuries when ignored.

This symbol/signal word indicates a danger of low risk level which may result in a minor or moderate injury when ignored.

**NOTICE!**

This signal word warns of possible material damage.

# MOBILITY Plus+

## Safety

### Intended Use

The shower stool is exclusively designed as seating option for showering. It should only be used in standard shower trays that ensure a secure stand.

The shower stool is exclusively designed for a maximum load of 150 kg.

It should only be utilized for private use and not in a commercial setting. The shower stool is not a children's toy or a medical device and is not suited for therapeutic purposes. The shower stool should only be used as described in this instruction manual. Every other usage is not intentional and may lead to material damage or even personal injuries. The manufacturer or the retailer assume no liability for damage during improper use.

## Safety information

### Risk of suffocation!

Children may entangle themselves, impose the film over their heads or swallow small parts when playing with the packaging film or small parts.

- Keep packaging and small parts away from children.

## Safety information

### **Risk of injury!**

Improper handling of the shower stool may result in injuries.

- Never stand on the shower stool.
- Only use the shower stool in the shower tray. Never use the shower stool in a bathtub or at a sink.
- Always make sure that the shower stool stands its four legs.
- Note, that there is an increased danger of slipping when sitting down or getting up in a wet shower tray, especially when bath additives or showering gel/shampoos/etc where used.

### **NOTICE!**

### **Risk of damage!**

Improper handling of the shower stool may damage the stool.

- Do not use the shower stool if there are loosened screws or if there are deformations, cracks or other damages on metal or plastic parts.
- Do not change the construction of the shower stool.
- Do not attach other elements on the shower stool.

## **NOTICE!**

### **Risk of damage!**

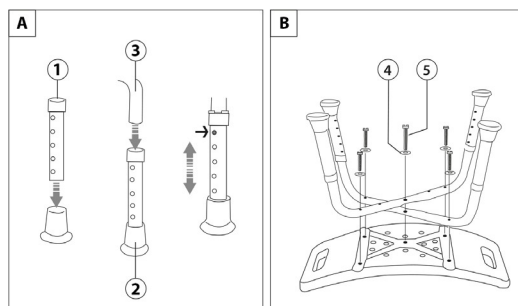
If the packaging is opened incautiously with a knife or other sharp devices the shower stool may be damaged.

– Caution when opening the package!

1. Take the shower stool out of the package and remove the packaging material.
2. Check if the delivery is complete (see **Items included in the delivery**).
3. Check if the shower stool is damaged. If this is the case do not use the shower stool. Please contact the manufacturer.

# MOBILITY Plus+

## Assembling the shower stool

1. Push the rubber feet (2) on the foot tubes (1).
2. Connect the upright tubes with the seating tubes (3)(see fig. A).
3. Press the button and change the upright tubes to the desired height.
4. Make sure that the button on the seating tube snaps into a hole of the upright tubes and check that every upright tube has the same height.
5. Screw the seating tubes and the seat with the cross-head screws (5) and the washers (4) together (see fig. B).
6. Align the rubber feet so that the shower chair with back rest has a solid, secure position.  
You can now use the shower chair.

# MOBILITY Plus+

## Cleaning and maintenance

### NOTICE!

#### Risk of damage!

Improper cleaning of the shower stool may lead to damages on the surface.

– Do not use aggressive detergents, brushes with metal or nylon bristles as well as sharp or metal cleaning items such as knives or scrapers.

1. Dry the shower stool after utilization and after cleaning with a soft and dry cloth.
2. Check all screws and connections regularly. If necessary retighten.

# MOBILITY Plus+

## Technical data

- **Model:**  
SC10+
- **Weight:**  
ca. 1,9 kg
- **Measurements H x W x L:**  
max. 56 x 50 x 47 cm
- **Item number:**  
10-02-111-1211
- **Maximum load:**  
150 kg
- **Height adjustment:**  
38-56 cm
- **Size of seat:**  
50 x 30 cm
- **Material:**  
Plastic (HDPE), Aluminium

## Disposal

### Disposal of packaging

Dispose the packaging correctly sorted. Add cardboard and carton to waste-paper and films to the collection of recyclables.

### Disposal of shower stool

Dispose the shower stool according to the countries applicable regulations and laws.

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Rehab Direct GmbH & Co. KG