**ZAMST ELBOW BAND (ELBOW SUPPORT) INSTRUCTIONS FOR USE (For left or right side)****EN**

Thank you for purchasing this product. For safety reasons, be sure to read this user manual before use. Also, store it in a location where you can refer to it at any time.

Read this first:

- This product can be used for the left or right side.
- Support may not be provided by this product under certain usage conditions.
- This product is not designed to treat elbow disorders.
- Check that the product size is correct before use.

When using the product:

- Due to the characteristics of the materials used in this product, the color may change depending on storage conditions.
- When storing the product, avoid applying excessive weight to the hook-and-loop fasteners or raised fabric. Otherwise, the hook-and-loop fasteners may not work effectively.

Precautions for use - Be sure to read this:

Safety precautions are divided into the following sections, according to the level of danger involved.

⚠ WARNING: Indicates a potentially hazardous situation, which may result in serious personal injuries if not avoided.

⚠ CAUTION: Indicates a potentially hazardous situation, which may result in minor personal injuries and/or property damage if not avoided.

⚠ WARNING

- To prevent abnormalities or worsening of symptoms, people who suffer from the following conditions should refrain from using the product:
 - Those predisposed to allergies or with sensitive skin.
 - Those who have a fracture, cut, numbness, swelling, eczema, rash or other abnormalities in the area where the product will be worn.
- Be sure to follow the warnings below and use the product correctly. If the product is not used correctly, it may cause circulatory problems or injury.
 - The product should only be used on elbows.
 - Be sure to use the correct size product.
 - Do not use the product while sleeping.
 - If wearing the product for a long period of time, readjust it as necessary. The application of constant pressure for a long period of time may cause skin and/or circulatory problems.
 - When using the product, be sure to put it on in the correct orientation (inside/outside).
 - Follow the attachment steps in order to use this product correctly.
 - Do not fasten the strap excessively tight.
 - If the product loosens or comes off during use, be sure to refasten it correctly by following the noted steps from the beginning.
 - If the product shows any signs of wear or damage, or a hook-and-loop fastener no longer works well, do not use the product.
 - Do not attempt to modify or repair the product.
- The loop of the product is made of plastic. To prevent personal injury, do not use the product in the following situation:
 - Where contact with the product may injure others.
- If any of the following abnormalities occur when using this product, immediately stop using it, and consult a physician. Continued use may worsen symptoms:
 - If this product causes discomfort.
 - If an injury, fracture, cut, pain, numbness, swelling, eczema, rashes, etc. occurs in the area where the product is worn.

⚠ CAUTION

- Hard materials are used for the hook-and-loop fasteners. In certain rare instances, fingers or skin may be damaged, so use the product carefully.
- Sweat and friction may cause the color of this product to fade or be transferred to other materials.
- Make sure the hook-and-loop fasteners not attach to clothing. If they do, clothing may become frayed or be damaged.



Care and cleaning**⚠ CAUTION**

- Do not wash this product with light colored items. By doing so, color may fade or be transferred to other materials.
- If washing the product together with clothing, etc., be sure to use a laundry net. If a hook-and-loop fastener becomes attached to clothing, it may cause the clothing to become frayed or be damaged.

- Remove the elbow pad before washing.
- This product can be washed in a washing machine. After washing, hang it to dry in the shade.
- When washing this product, attach the hook-and-loop fasteners to the main body. Attempting to wash this product without closing the hook-and-loop fasteners may weaken its fastening capability.
- Do not use an iron or chlorine bleach, however, oxygen bleach can be used.
- Do not tumble dry. Doing so may damage the product.
- This product cannot be dry cleaned. Wet cleaning is allowed, however, avoid tumble drying.

Where to apply


Place the elbow pad 4 to 5 cm (1.5 to 2 inches) (toward wrist) below the area where you feel pain in your elbow.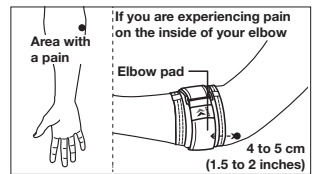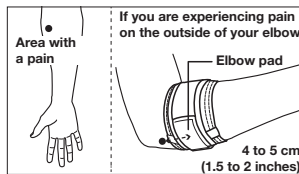

How to apply


- Before attaching this product, carefully read the "Precautions for use - Be sure to read this" section. Then follow the steps provided to ensure the product is correctly attached.
- The steps below use the right arm for reference. Follow the same steps for the left arm.

1. Unfasten all the hook-and-loop fasteners of the product. Confirm the position of the pad.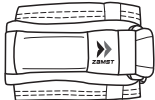


<Side with a pad>

Make sure the pad is on the ZAMST logo side.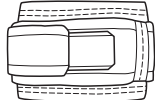


<Side without a pad>

2. Pass your arm through the product.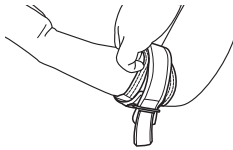

Pass your arm through the product and adjust the position of the product so that the product is located just below your elbow. Be careful not to cover your elbow joint with the product.


3. Fasten the hook-and-loop fastener.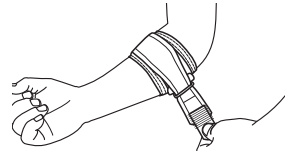

Adjust the position of the product so that the elbow pad is positioned where you feel pain. Wrap the product around your forearm while adjusting the compression to the required level, and then fasten the hook-and-loop fastener.

4. Make sure that the product is not loose or out of place. If you feel any discomfort, repeat the steps from the beginning to reattach the product.


The product correctly fastened
(if you are experiencing pain on the outside of your right elbow)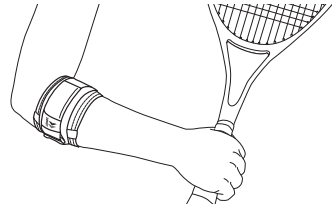


Removing the elbow pad

The elbow pad is stored in the pad pocket on the back of the strap. Remove the pad before washing.

1. Remove the strap end from the loop and fasten the hook-and-loop fastener on the strap end temporarily. Because the elbow band has an anti-slip design, hold the tag when you remove the pad.

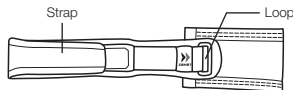
<Outside of the product>
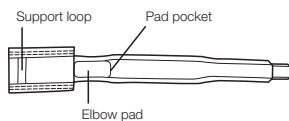
<Inside of the product>

2. Turn the strap over and remove the pad from the pad pocket. (Be careful not to lose the removed pad.)

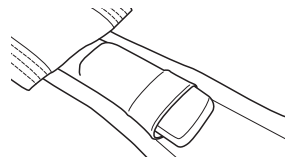3. After washing, put the pad into the pad pocket as far as possible before using the product.
4. Pass the strap through the support loop, followed by the loop before applying the product by following the attachment steps.

Reporting Obligation

If you have any complaints about the product, please contact your medical retailer directly. It is only necessary to notify the manufacturer and the competent authority of the member state in the event of any serious incidents that could lead to a significant deterioration of health or to death. Serious incidents are defined in Article 2 No. 65 of Regulation (EU) 2017/745 (MDR).

Disclaimer:

- * The product specifications and appearance are subject to change without notice for the purpose of improvement.
- * When disposing of this product, be sure to follow all disposal ordinances and regulations as specified by the local government.
- * We take all possible measures to ensure product quality, however, if you find any defects, etc. in our products, please contact Nippon Sigmaz Co., Ltd.

"ZAMST" and "ZAMST logo" are trademarks or registered trademarks of Nippon Sigmaz Co., Ltd in Japan, the United States and other countries.

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