

Body-REACH+ BENDABLE BODY WAND



TOTAL BODY CARE

Manufacturer of the #1 dermatologist-recommended applicator

Tips for Using Your Body-Reach+ Bendable Body Wand +Attachments



Bend it, it's Unbreakable!


The **Body-Reach+ Bendable Body Wand** is made out of a fairly stiff plastic that will take "some strength" to bend. Once bent, your wand will hold its curved position for a period of time to allow you to use pressure to scrub or apply lotions or medications even with limited range of motion. The **Body-Reach+ Bendable Body Wand** can withstand repeated bending in a variety of positions and remain unbreakable!

To bend: Grip each end of the lotion applicator in both hands and bend the ends towards the center. **Not enough strength?** You may also bend the lotion applicator by placing the head of the wand against a slip-resistant hard surface and using both hands to grip the handle and bend to curve the wand.

Wash it, it's Reusable!

Both sides of the unique **Sponge-Able +Attachment** applicator sponges are made from a soft, hypo-allergenic material. The removable, washable, reversible and nonabsorbent **Sponge-Able** is designed to allow you to apply any topical treatment without it soaking into the sponge, thereby eliminating waste. Properly cared for, the **Sponge-Ables** have lasted for more than 100 uses.

To use: Dampen sponge surface with water. Attach the sponge to the VELCRO® brand fastener disc and place your lotion or medication onto the sponge. **After use:** Immediately remove the **Sponge-Able** by grasping it firmly and pulling it away from your body. The VELCRO® disk will remain in place on the wand. Wash the **Sponge-Able** with a mild soap, rinse with water and allow it to air dry.

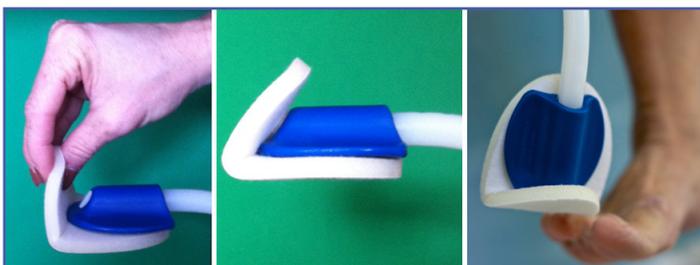

Wrap it, it's Adaptable!

Utilize your **Body-Reach+ Bendable Body Wand** to easily reach and clean between your toes. **To use:** Attach the **Sponge-Able** to the bottom of the wand's head first (completely covering the large VELCRO® fastener disc) and then wrap the end of the **Sponge-Able** over the front of the wand's head and attach it to the 1/2" VELCRO® disc. Now the contoured nose of the wand's head and **Sponge-Able** are ready to fit between your toes.

Questions or comments:

ReachMate+

P. O. Box 2688
Laurel, Maryland
20709-2688

Phone: 866-834-3535
Fax: 301-490-3262
www.ReachMatePlus.com