INSTALLATION
BEDIENUNG
WARTUNG

ARTIKEL 2801/2802

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INSTALLATION
OPERATION
MAINTENANCE

ARTICLE 2801/2802

SEKI
SENSOR SEAT CUSHION
FOR MOBILE AND
STATIONARY USE

Dear users,

You have chosen the IQcare SEKI Sensor Seat Cushion so that you are informed via the nurse call system or a mobile receiver if your residents should not get up without your help.

If handled properly, you can rely on this quality product at all times. Therefore, please observe the information in these installation and operating instructions.

Your IQcare team

FOR QUESTIONS:
JUST CALL OR EMAIL!

FON 0 23 54 / 944 99 69

E-MAIL INFO@IQFY.DE

NO ELEKTROSMOG

All IQcare sensors and switches transmit only very briefly when they are activated. This means that there is no electrosmog!

AREAS OF APPLICATION

Mobile: Everywhere in the house with a range of approx. 40 m
Stationary: In the room and on the associated balcony.

SCOPE OF DELIVERY

SEKI

Sensor-
Seat cushion
SEKI

with mobile receiver
NAVI GO

Sensor-Seat cushion SEKI
with mobile receiver NAVI

(for connection to the call system)

PREPARATION

Please decide now which situation you have and what you want to install in addition:

1 SEKI Mobile

Continue on page 21

2 SEKI on a call system - preparations

Continue on page 23

3 SEKI on a call system (working current)

Continue on page 24

4 SEKI on a call system (quiescent current)

Continue on page 26

5 Teach-in with existing NAVI

Continue on page 29

6 Error handling

30

7 Maintenance

30

8 Desinfection

31

9 Technical Data

31

1 SEKI MOBILE

- Plug the radio receiver NAVI GO into a socket. Caution: Do not go further than approx. 40 m from the seat cushion.

- The CLR-button now flashes green.

- Press 3 x briefly in succession on the sensor in the seat cushion.

- The CLR-button goes off and the NAVI GO lights up completely red.

- The seat cushion is now ready for use.

- You can now install the NAVI GO in any into any socket in the house.

It lights up green, when the seat cushion is occupied, and red when it is not occupied.

In addition, an acoustic signal sounds an acoustic signal sounds if the seat cushion is not occupied.

Deleting the settings - e.g. if there has been an error in the teach-in process.

Press the LRN-button for approx.

Press the LRN-button for approx.

10 seconds until the CLR-button flashes; then press the CLR-button for 3 seconds to delete the settings.

- **Attention: The cushion for the mobile receiver can not be used for a stationary connection to a call system be used.**

- Cushion and receiver can be identified by an identification sticker.

2 SEAT CUSHION SEKI ON A CALL SYSTEM

PREPARATION

If the radio receiver and the adapter cable are not yet connected, please do so.

Important notice: This is a **connector**, please do not twist the plug onto the connector. You will find a white line on both the radio receiver and the adapter cable you will find a white line as a mark - these must fit over each other.

3 ANNOUNCING IN WORKING CURRENT MODE

- Find a socket that is close to the connection of your detection system so that you can plug in the free end of the adapter cable **after the end of the teach-in process.**
- Plug the radio receiver into this socket.
- The CLR-button now flashes green, the radio receiver is in in teach-in mode.
- Press the sensor in the the sensor in the middle of the centre of the seat cushion.

- The LRN-button is now is now permanently red.
- The seat cushion and the radio receiver are connected to each other. This is additionally confirmed by an acoustic signal.
- Now connect the radio receiver NAVI with the connection of your call system.

- The red LRN-button signals that the system is ready for use.
- If the seat cushion is occupied, the CLR-button also lights up green.

- The seat cushion, the radio receiver and any other accessories available, such as the bed escape system, always form a fixed unit. It is best to number each unit - corresponding stickers are enclosed or can be requested at info@iqfy.de.

4 TEACH-IN IN QUIESCENT CURRENT MODE

- Find a socket that is close to the connection of your detection system so that you can plug in the free end of the adapter cable **after the end of the teach-in process.**
- Plug the radio receiver plug the radio receiver into this socket.
- The CLR-button now flashes green, the radio receiver is in in teach-in mode.

- Briefly press the LRN-button. The radio receiver is thereby switched to the The radio receiver is then switched to standby mode.

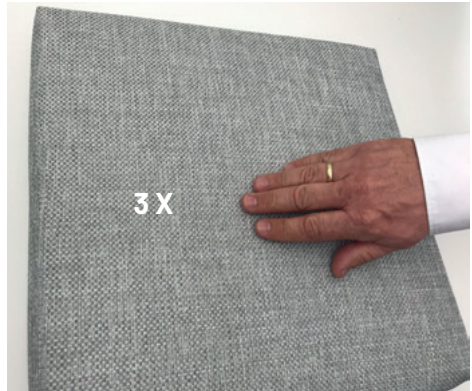
- The CLR-button flashes red.

- Press 3 x briefly and quickly in succession on the sensor in the centre of the seat cushion.

- The LRN-button is now permanently red.
- The seat cushion and the radio receiver are connected. This is additionally confirmed by an acoustic signal.
- Now connect the radio receiver NAVI to the connection of your call system.

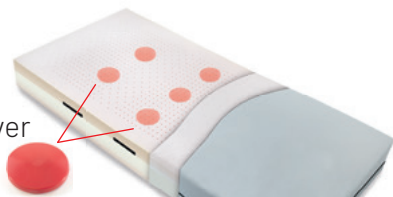
- The red LRN-button signals that the system is ready for use.
- If the seat cushion is occupied, the CLR-button also lights up green.
- The seat cushion, the radio receiver and any other accessories available, such as the bed escape system, always form a fixed unit. It is best to number each unit - corresponding stickers are enclosed or can be requested at info@iqfy.de.

5 ADAPTING TO A EXISTING SYSTEM

- You already use an IQcare system such as the bed escape system SESY bed escape system - accordingly the radio receiver NAVI is already available.

Proceed as follows:

- Unplug the Navi from the socket, wait 3 sec. (until the light goes out) and plug it back in.
- Press the LRN-button for approx. 10 sec. button until the CLR-button flashes:
 - green for operating current
 - red at quiescent current.
- Press 3 x the sensor in the centre of the seat cushion.
- The receiver signals the connection with an acoustic signal.
- Finally, press the CLR-button once.

ERROR HANDLING

- In rare cases, the installation may not work as described.
- **Then it is recommended to completely reset the radio receiver to factory settings. To do this, please proceed as follows:**
 1. The setting can be deleted within 30 seconds by unplugging the radio receiver and plugging it back into the socket.
 2. Press the LRN-button for approx. 10 seconds, until the CLR-button flashes.
 3. Press the CLR-button for approx. 3 sec. until the light goes out briefly before it flashes green again. The radio receiver is now back in the delivery state and can be reset according to the operating instructions.

MAINTENANCE AND CARE

- IQcare products can be cleaned carefully with a soft cloth and normal household cleaning agents.
- The cover is washable at 60°.
- Exchange possible!

DISINFECTION

- Use commercially available disinfectants. A list of recommended disinfectants can be requested from: info@iqcare.info
- ... or directly via the QR code on the right. can be loaded.

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IQfy GmbH