INSTALLATION
BEDIENUNG
WARTUNG

ARTIKEL 8002

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INSTALLATION
OPERATION
MAINTENANCE

ARTICLE 8002

ALARM SYSTEM
„OUTBOUND-AWAY
RUN-TENDENCY“
WORKING CURRENT
STILL CURRENT

Dear users,

you have chosen the IQcare alarm system with door transmitter so that you can be informed via the call system when your residents leave an agreed area.

When handled properly, you can rely on this quality product at all times. Therefore, please observe the information in these installation and operating instructions.

Your IQcare team

FOR QUESTIONS:
JUST CALL OR EMAIL!

FON 0 23 54 / 944 99 69

E-MAIL INFO@IQFY.DE

NO ELEKTROSMOG

All IQcare sensors and switches transmit only very briefly when they are activated. This means that there is no electrosmog!


AREAS OF APPLICATION

Room doors, bathroom doors, balcony doors, windows ...

SCOPE OF DELIVERY

Radio receiver incl. adapter cable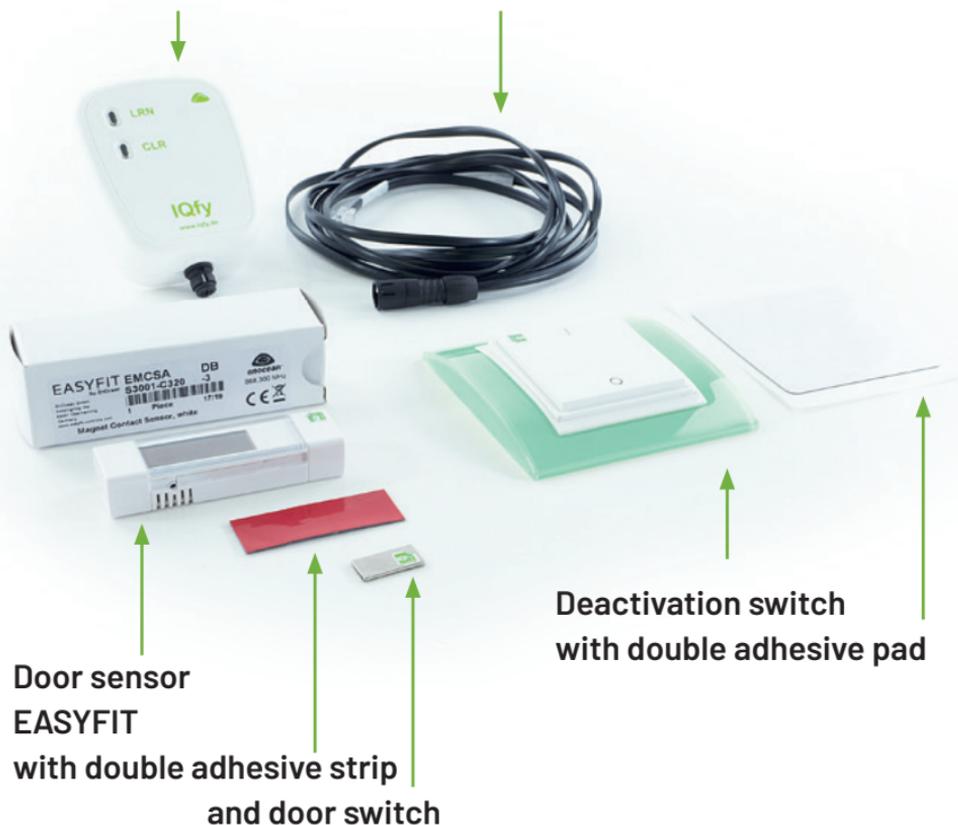


PREPARATION 1

The Easyfit door sensor draws its power supply in an environmentally friendly way from the built-in solar cells. Before using the door sensor for the first time, charge the built-in battery by placing the Easyfit door sensor on the windowsill or under a lamp for 15 minutes.

Important notice: If you use the Easyfit door sensor in dark rooms such as bathrooms or basements, the built-in solar cells will not get enough light and the built-in battery will not be charged. In such cases, you can insert a lithium button cell into the Easyfit door sensor. You can find these on the Internet under the designation „CR1225“.

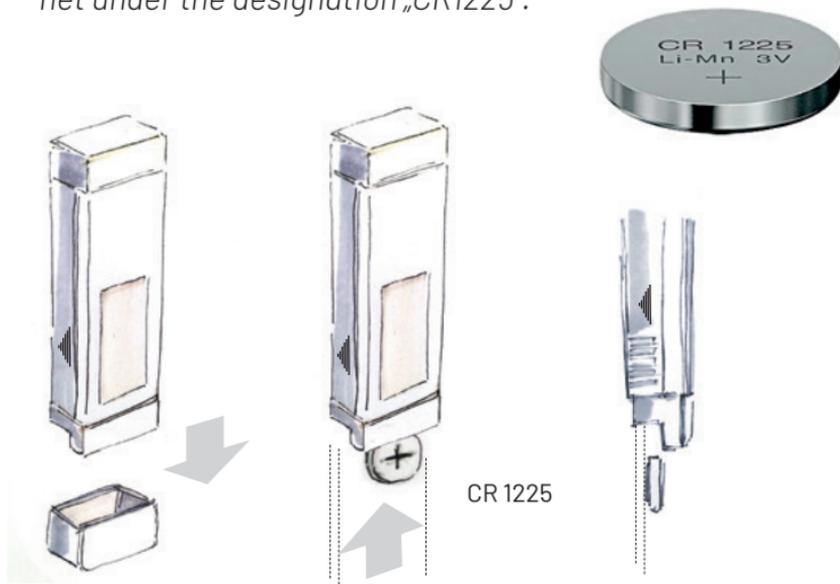PREPARATION 2

If the radio receiver and the adapter cable are not yet connected, please do so.

Important notice: This is a **plug connection connector**, so please do not twist the plug onto the connector. You will find a white line on both the radio receiver and the adapter cable. They must fit over each other.

TEACH-IN IN WORKING CURRENT MODE

- Find a socket that is close to the connection of your detection system so that you can plug in the free end of the adapter cable **after the end of the teach-in process.**
- Plug the radio receiver into this socket.
- The CLR-indicator now flashes green, the radio receiver is in teach-in mode.

- Now press the programming button of the Easyfit door sensor once with a pointed object.
- If the radio receiver has detected the Easyfit door sensor, the radio receiver beeps.

- The LRN indicator of the radio receiver now lights up permanently red.
- The radio receiver and the Easyfit door sensor are now permanently connected to each other.

TURNING ON THE DEACTIVATION SWITCH

- Unplug the radio receiver from the socket.
- Plug the radio receiver into the socket again.
- The LRN- button now lights up permanently red, as the radio receiver is already connected to a sensor.
- Press the LRN-button for 10 seconds until the CLR-button flashes green.

- Press the „0“ of the deactivation switch 3 times in quick succession.

- The CLR-button of the radio receiver lights up green continuously for 2 seconds.

- The radio receiver beeps, the deactivation switch has been detected.

- The deactivation switch is now permanently connected to the radio receiver.

- To save the setting, briefly press the CLR-key. The LRN-key now lights up permanently red again.

**3 X QUICKLY****SHORT**

STARTING IN THE STANDBY CURRENT MODE

- Find a socket that is close to the connection of your detection system so that you can plug in the free end of the adapter cable **after the end of the teach-in process.**
- Plug the radio receiver into this socket.
- The CLR-indicator now flashes green, the radio receiver is in teach-in mode.
- Press the LRN-button once to switch the radio receiver to standby mode.
- The CLR-button now flashes red.

- Now press the programming button of the Easyfit door sensor once with a pointed object.

- If the radio receiver has detected the Easyfit door sensor, the radio receiver beeps.
- The LRN-indicator of the radio receiver now lights up permanently red.
- The radio receiver and the Easyfit door sensor are now permanently connected to each other.

TURNING ON THE DEACTIVATION SWITCH

- Unplug the radio receiver from the socket.
- Plug the radio receiver into the socket again.
- The LRN-button now lights up permanently red, as the radio receiver is already connected to a sensor.
- Press the LRN-button for 10 seconds until the CLR-button flashes red.

10 SEC.


- Press the „0“ of the deactivation switch 3 times in quick succession.
- The CLR-button of the radio receiver lights up red continuously for 2 seconds.
- The radio receiver beeps, the deactivation switch has been detected.
- The deactivation switch is now permanently connected to the radio receiver.
- To save the setting, briefly press the CLR-key. The LRN-key now lights up permanently red again.

**3 X QUICKLY****SHORT**

FIXED DEACTIVATION SWITCH

- Place the deactivation switch outside the room so that you can operate it well before opening the door. Either stick the double adhesive pad directly to the back of the switch or use the DEA switch-support-plate (not included in the scope of delivery).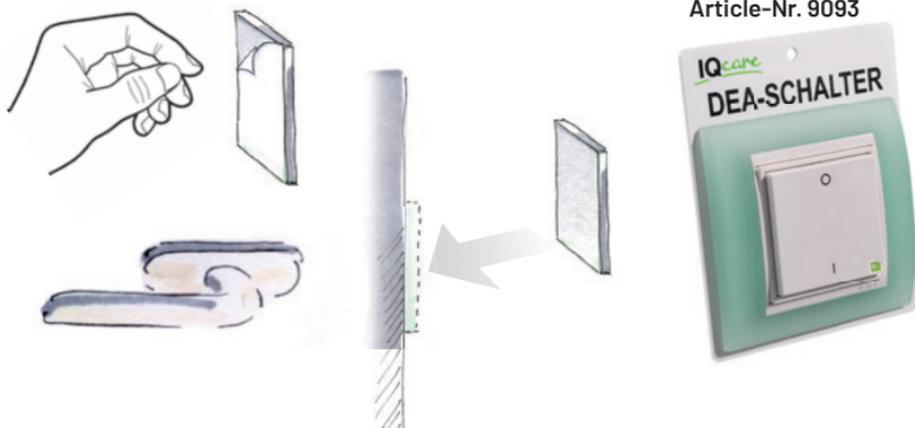

OPERATION DEACTIVATION SWITCH

- When you enter the room, press the „0“ to deactivate the message for 15 minutes.
- Leave the room, press the „1“: the message is activated again.
- If you forget to activate the system, it will arm itself again after 15 minutes.
- **If you are in the room for a longer period of time and the door was closed during this time, a signal is sent to the nurse call system. If you leave the door open, the switch is only activated again when the door is closed.**

FIXING DOOR-SENSOR EASYFIT


- Stick the Easyfit door-sensor to the frame of the door with the double-sided adhesive tape and the door switch to the door. Preferably on the upper edge so that it cannot be removed accidentally or deliberately.

Mounting with normal door

- Mounting the Easyfit door-sensor on the top of the frame.


Important notice: A triangle can be seen on the door-sensor. This side must have contact with the door-switch.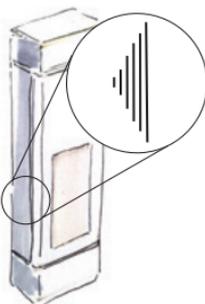

Mounting with door flush with wall

- Remove the Easyfit door-sensor from the outside.

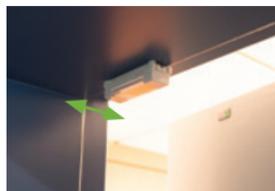CONNECTING TO THE NURSE CALL SYSTEM

- Now insert the adapter cable of the radio receiver into the connection of your call system. You will now automatically receive a signal as soon as the door or window is opened.

ERROR HANDLING

- In rare cases, the installation may not work as described.
- **Then it is recommended to completely reset the radio receiver to factory settings. To do this, please proceed as follows:**
 1. The setting can be deleted within 30 seconds by unplugging the radio receiver and plugging it back into the socket.
 2. Press the LRN button for approx. 10 seconds, until the CLR button flashes.
 3. Press the CLR button for approx. 3 seconds until the light goes out briefly before it flashes green again. The radio receiver is now back in the delivery state and can be reset according to the operating instructions.

TIP

If you have several IQCare wireless solutions in your home, number these units using our helpful identification labels.

MAINTENANCE AND CARE

- IQCare products can be cleaned carefully with a soft cloth and normal household cleaning agents.

DISINFECTION

- Use commercially available disinfectants. A list of recommended disinfectants can be requested from be requested from: info@iqcare.info

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IQfy GmbH