

HOW TO ASSEMBLE YOUR



OVERBED TABLE NON-TILT

Item # 6070

Capacity not to exceed 22 lbs




PARTS INCLUDED

- | | | |
|---------------------|---------------|-----------------|
| a. (1) Table top | d. (1) Knob | g. (1) Hardware |
| b. (1) Column | e. (1) Lever | |
| c. H Base w/ Wheels | f. (2) Spacer | |
-

Follow these
SIMPLE STEPS


- 1** Stand the column assembly right side up. Be sure the rod is not protruding out of the top. Insert the lever with the rounded side facing up into the column assembly. Ensure that the flat side of the lever rest on the rod (*see figure 1*).

Figure 1

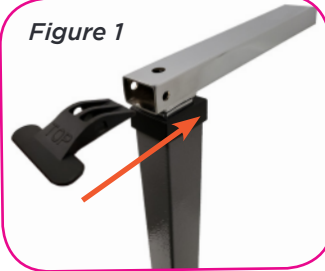

- 2** Lay table top upside down on a flat surface with the mounting bracket facing up. While holding the lever securely in place on top of the rod, flip the column assembly upside down and insert the column assembly into the table top mounting bracket (*see figure 2*).


Figure 2

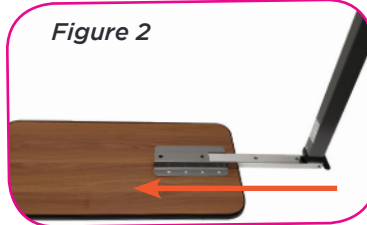

Note: Do not tilt the column assembly until the lever is holding the rod to prevent it from falling off.


- 3** Align the horizontal hole of the column to the table top mounting bracket, and align the holes on the column assembly to the hole through the lever. Insert the lever screw and fully tighten the nut (*see figure 3*).

Figure 3

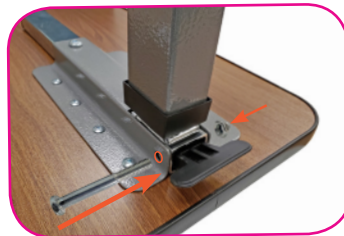

- 4** Secure the column assembly to the mounting bracket with 2 mounting screws through the vertical holes in the mounting bracket and into the threaded holes in the column assembly (*see figure 4*).


Figure 4

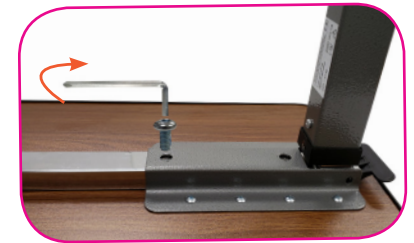

- 5** Locate the 2 holes on the H base. Insert both bolt sleeves into the bottom of the H base. **Do not tilt the H base** (*see figure 5*). Align the H base to the column with wheels facing up while table top is upside down. Place the metal plate to the H base holes and tighten both bolts (*see figure 6*).

Figure 5

Figure 6

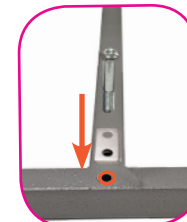

- 6** Turn the table over. Replace the top black screw that's located on the column (*see figure 7 & 8*) with the chrome screw that came with the hardware.

Note: Do not take off the black screw until step 1 - 6 are completed.

Figure 7

Figure 8



- 7** Insert the knob onto the hole that is located on the side of the column (*see figure 9*).

Figure 9



WARNING

Read all instructions carefully and consult with a qualified professional before assembling to avoid possible injury. NOVA Medical Products shall have no liability for any injury resulting from improper assembly or misuse of this product.

- DO NOT over tighten the bolts, screws or use any **powertolls** when **assembling this product**.
- DO NOT use the product if it appears to be **damaged or unstable**.
- DO NOT modify the product in any way.
- DO NOT Overload the product as that can cause it to **brake and/or serious injury**.
- Check frequently for cracks, damage, or looseness.
- Maximum weight capacity is 22lbs.

HOW TO USE

Adjusting the height of the table by removing or loosening the knob.

1. To raise - Pull up from the bottom of the table top until you reach your desired height
2. To lower- Pull up on the lever and push down on the table top. Release the lever at your desired height.

CARE AND MAINTENANCE

Check to make sure all hardware is securely tightened before using the table. Use a mild detergent and warm water to clean. Dry with a soft cloth to avoid moisture build-up.

WARRANTY INFORMATION

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under NORMAL use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	Limited Lifetime Warranty
Non-Metal/Metal parts	One Year Limited Warranty (rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.

CONTACT US!
We're here to help



info@novajoy.com



1 800 557 6682

novajoy.com

Connect with us!

novajoy.com

novamedicalprod

novamedicalprod

novawalkertalk

NOVA Medical Products



CARE • QUALITY • INNOVATION

THE **NOVA** PROMISE

IMPORTANT WARNING

FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.