

Guide to fitting

The “Earkart Hearing” App makes it easy to control & adjust your hearing aid with your Smart phone.



BY earKART

Step 1:

Download the “Earkart Hearing” App from Google play store for Android device & App store for Apple devices.


earKART Hearing



Step 2:

(Remove the hearing aids from the charger & enable Bluetooth on the smart phone.)


Search the name “EQFY 16” on the smart phone. Tap to connect your hearing aids with your phone.

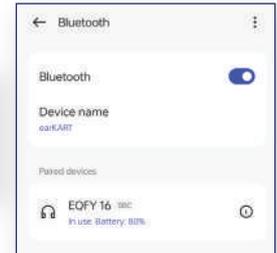

Step 3:



Open the “Earkart Hearing” App. Tap “Connect Aids”. Scan & select the device name of “EQFY 16-R” & “EQFY 16-L”.

(Now the hearing aids are connected and redirect to Home screen)

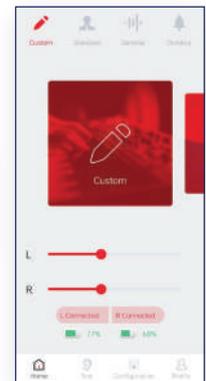

Step 4:

Tap on “Test” for hearing screening / Audiogram input & click on “Auto-Setting” for first fit.

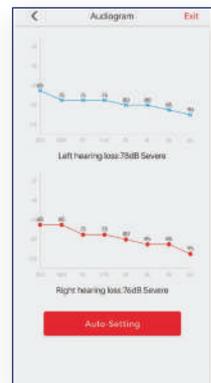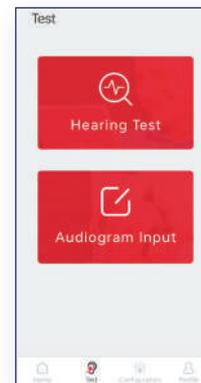

Step 5:

For fine tuning, Tap “Configuration” & do the necessary changes in the “Equalizer” section & Save.

