

Guide to fitting

The "Audientes" App makes it easy to control & adjust your hearing aid with your Smart phone.

Step 1:


Download the "AUDIENTES" App from Google play store for Android device & App store for Apple devices.

Audientes



Step 2:



Switch on the hearing aid by pressing **power button**. Search the app name "Audientes" on the smart phone. Tap to connect to Audientes device on your phone.

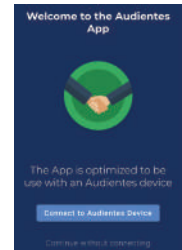

Step 3:




On the "Audientes" app. Tap on "Audientes Ven".

If you can not see Audientes Ven on the screen then press & hold P Button until Blue light flashes & rescan & select the device name of "Audientes Ven"

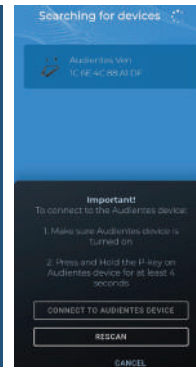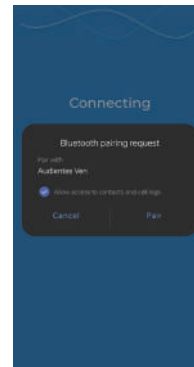




Step 4:

Tap on "Make New Hearing Test" for hearing screening / Enter Audiogram to input the audiogram by clicking on "settings written on top of the screen" 6-7 times to enable the option.

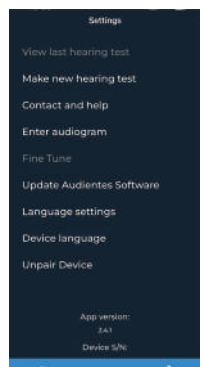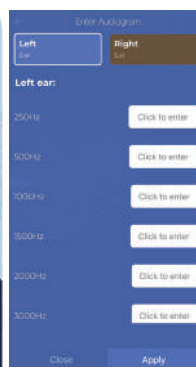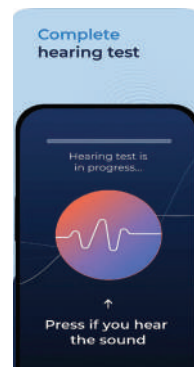


Step 5:

For fine tuning, repeat Step-4 & Tap "Fine tune" & do the necessary changes in the "Fine tuning" section & do it.

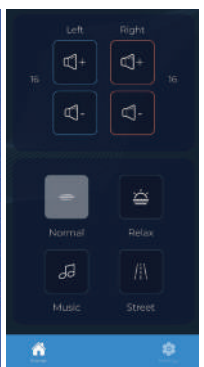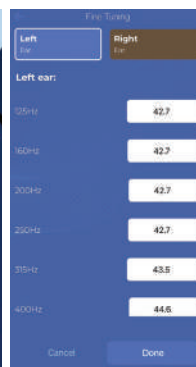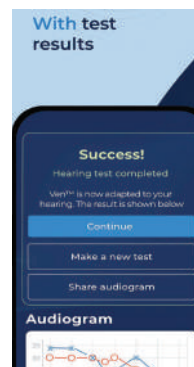

 Charging & Switch ON

 Switching Off & Battery Low Warning

 Streaming, Talking, Hearing Test, During a phone call