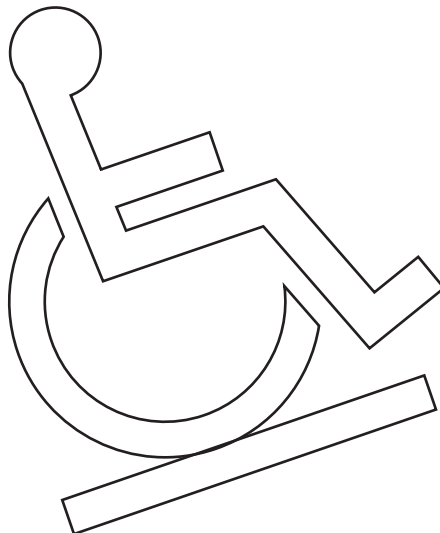
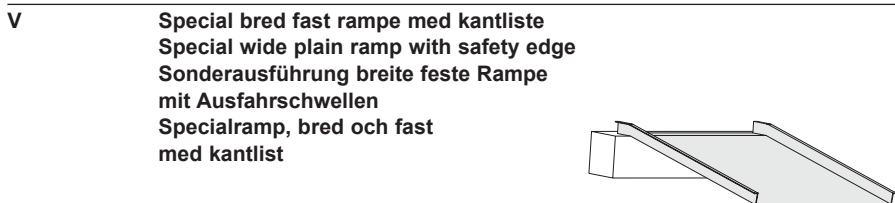
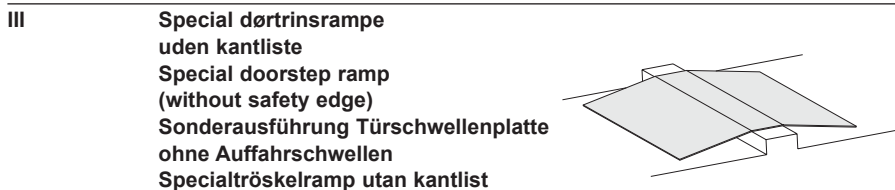
DK . . . . .	Dækplader, dørtrinsramper & special dørtrinsramper . . . . .	3
GB/US . . . . .	Cover plates, doorstep ramps & special doorstep ramps . . . . .	8
DE . . . . .	Abdeckbleche, Türschwellenrampen und spezielle Türschwellenrampen . . . . .	14
SE . . . . .	Täckplattor, tröskelramper och specialtröskelramper . . . . .	19

Vers. 2.00

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# GB/US . . . Cover plates, doorstep ramps and special ramps

## Product numbers

780xxx	Cover plate
39167/77	Doorstep ramp
39199	Special doorstep ramp without safety edge
39299	Special doorstep ramp with safety edge
39499	Special wide plain ramp with safety edge

<b>1.00</b>	<b>Purpose and use</b>	<b>9</b>
1.01	Manufacturer	9
1.02	Purpose and use	9
1.03	Use	9
1.04	Important/warnings	10
1.05	Unpacking and preparation of the ramp/cover plate	10
<b>2.00</b>	<b>Description of features</b>	<b>10</b>
2.01	Pictograms/labels	10
<b>3.00</b>	<b>Transport</b>	<b>10</b>
<b>4.00</b>	<b>Cleaning and maintenance</b>	<b>10</b>
4.01	Cleaning the ramp and cover plate	10
4.02	Corrosion	11
4.03	Maintenance	11
<b>5.00</b>	<b>Storage and stacking</b>	<b>11</b>
<b>6.00</b>	<b>Service and useful life</b>	<b>11</b>
6.01	Useful life	11
6.02	Service	11
<b>7.00</b>	<b>Technical specifications</b>	<b>12</b>
<b>8.00</b>	<b>EC-Declaration of conformity</b>	<b>12</b>
	<b>USA and countries outside the EU</b>	<b>13</b>
A.	Users guide	13
B.	WARRANTY	13

---

**1.00 Purpose and use**

---

**1.01 Manufacturer**

V. Guldmann A/S  
Graham Bells Vej 21-23A  
DK-8200 Århus N  
Tel. +45 8741 3100  
Fax. +45 8741 3131

---

**1.02 Purpose and use****Purpose**

Stepless cover plates, doorstep ramps and special ramps are used where wheelchair users need to overcome height differences.

**Application**

*The cover plate* replaces the doorstep. By using the cover plate, the wheelchair user can access one room from another regardless of the height difference.

*The doorstep ramp and special doorstep ramp with/without safety edge* have ramps on both sides of the doorstep. The wheelchair user can therefore use the ramp to get from room to room.

*The special wide plain ramp with safety edge* has a ramp on only one side of the doorstep and is used by placing the folded contact surface on the doorstep, for example. The wheelchair user can then use the ramp to access a higher level from a lower level.

---

**1.03 Use****Cover plate**

A doorstep creates an obstacle for wheelchairs, rollators and other mobility aids. This obstacle can be permanently removed by taking out the existing doorstep and installing a Stepless cover plate over the opening that is created as a result. The cover plate is tailored to the actual length of the installation site so that it fits exactly to the width of the door.

**Doorstep ramp, special doorstep ramp with/without safety edge**

The doorstep ramp is placed over the existing doorstep and offsets the height difference between two rooms. The doorstep ramp is not fixed and can therefore be easily removed after use.

**Special wide plain ramp with safety edge**

The ramp must be placed on a firm underlay. Position the ramp so that its folded contact surface is supported. The ramp can be installed at a maximum gradient of 1:5 when being used by electric wheelchairs and wheelchairs operated by caregivers. The ramp can be installed at a maximum gradient of 1:7 when being used by self-operated manual wheelchair users.

---

**1.04****Important/warnings**

- Read the entire user manual prior to putting the ramp/cover plate into operation.
- The maximum load must not be exceeded.
- Movement up and down the ramp must take place slowly – without run-ups.
- The ramp must be positioned so that the entire contact surface is supported.
- The ramp must be used on a firm underlay.
- The cover plate must be completely supported and must not, for example, be used as a bridge between two rooms.
- The ramp should not be used for gradients greater than 1:5 by electric wheelchairs or wheelchairs operated by caregivers.
- The ramp should not be used for gradients greater than 1:7 for manual wheelchairs.

---

**1.05****Unpacking and preparation of the ramp/cover plate****Visual check of the ramp/cover plate**

If the packaging is damaged on receipt, the ramp/cover plate should be carefully checked for noticeable imperfections, defects or flaws.

If the ramp/cover plate is thought to be damaged, it must not be used until quality service technicians or the Guldmann Service Team have approved it.

---

**2.00****Description of features**

---

**2.01****Pictograms/labels**

- Stepless label with maximum load indication.

---

**3.00****Transport**

The special wide plain ramp can be supplied with a carrying handle on the right/left side, depending on your needs and requirements. The handle is practical for transportation.

---

**4.00****Cleaning and maintenance**

---

**4.01****Cleaning the ramp and cover plate**

Clean the ramp and cover plate with a normal cleaning agent applied with a damp cloth, sponge or soft brush. The ramp and cover plate may be rinsed under running water. No chemicals or autoclave must be used to clean the ramp and cover plate.

---

**4.02 Corrosion**  
The ramp and cover plate are made of corrosion-resistant materials and do not require further corrosion prevention.

---

**4.03 Maintenance**  
No particular requirements other than cleaning.

If the ramp or cover plate surface is covered with ice and/or snow, remove the ice/snow before use.

**Note:** Salt (NaCl) is corrosive to the ramps and cover plates and will damage them over time. Instead, use a salt substitute, such as magnesium chloride (MgCl<sub>2</sub>). When using magnesium chloride to de-ice the ramps and cover plates instead of salt, you avoid oxidation and corrosion.

---

**5.00 Storage and stacking**  
No particular requirements.

---

**6.00 Service and useful life**

---

**6.01 Useful life**  
Given the maintenance/cleaning prescribed, the ramp and cover plate will have a useful life of 15 years.

---

**6.02 Service**  
Service inspections must be carried out in accordance with national legislation.  
Parts that are worn or defective must be replaced with new parts from V. Guldmann A/S.

**1. Visual check of the product**

- Check the product for wear and tear.
- Check the product for any faults.
- Check that there is no other form of damage to the product

**2. Have new problems arisen/been found?**

- If the product is damaged or parts need to be replaced, please contact the Guldmann Service Team or your distributor.

---

**7.00****Technical specifications****Doorstep ramp (391xxx)**

Width x length x height ..... 670x600x25 / 770x600x25 mm  
Max. load ..... 350 kg  
Materials ..... Bent 3 mm aluminium plate with black non-slip covering.

**Cover plates (780xxx)**

Lengths ..... 1000, 1800 mm  
Widths ..... 80, 105, 130, 150 and 180 mm  
Max. load ..... 500 kg  
Materials ..... Extruded aluminium or bronze/champagne-coloured aluminium with built-in non-slip protection (grooves)

**Special doorstep ramp without safety edge (39199)**

This ramp has been designed in specific dimensions so that the width/height varies from ramp to ramp.

Max. load ..... 500 kg  
Materials ..... Bent 3 mm aluminium plate with black non-slip covering

**Special doorstep ramp with safety edge (39299)**

This ramp has been designed in specific dimensions so that the width/height varies from ramp to ramp.

Height of edge ..... 36 mm  
Max. load ..... 500 kg  
Materials ..... Bent 3 mm aluminium plate with black non-slip covering

**Special wide plain ramp with safety edge (39499)**

This ramp has been designed in specific dimensions so that the width/height varies from ramp to ramp.

Height of edge ..... 54 mm  
Max. load ..... 325 kg  
Materials ..... Bent 3 mm aluminium plate with black non-slip covering

---

**8.00****EC-Declaration of conformity**

The product is manufactured in compliance with the Council Directive 93/42/EEC of June 14th 1993, including amendments, as medical device class 1.

## USA and countries outside the EU

---

### A. **Users guide**

Before using the product, read the entire operation manual including warranty.

---

### B. **WARRANTY**

Guldmann warrants its equipment is free from material defects under normal use, and will perform substantially in accordance with the specifications set forth in documentation provided with the equipment.

This express warranty shall be in effect for one year from the date of original purchase and installation (the "Warranty Period"). If a valid claim is made during the Warranty Period for malfunction or equipment defect, Guldmann will repair or replace the equipment at no additional cost to you. Guldmann retains sole discretion as to whether the equipment will be repaired or replaced.

This warranty shall be null and void if the equipment is operated and maintained in any manner inconsistent with its intended use or the instructions provided with the product. Further, in order for the warranty to remain in effect for the full Warranty Period, all service to the equipment must be provided by a Guldmann designated technician. Any parts or components repaired or replaced by a Guldmann designated technician will be guaranteed for the remainder of the Warranty Period.

The warranty does not cover any part of the equipment which has been subject to damage or abuse by the user or others. The warranty does not cover any part of the equipment which has been altered or changed in any way by the user or others. Guldmann does not warrant that the lifting device functions will meet your requirements, be uninterrupted or error free.

The warranty set forth is in lieu of all other express and implied warranties, whether oral, written or implied, and the remedies set forth above are your sole and exclusive remedies. Only an authorized officer of Guldmann may make modifications to this warranty, or additional warranties binding on Guldmann. Accordingly, additional statements such as advertising or presentations, whether oral or written, do not constitute warranties by Guldmann.

#### **Service or Repair**

Contact Guldmann Repair for an authorization to return any defective item during the Warranty Period. You will be provided with a return authorization number and address for returning the item for warranty service or replacement. Do not return items to Guldmann under warranty without receiving a Return Authorization Number.

If mailing the item, pack it carefully in a sturdy carton to prevent damage. Include your Return Authorization Number, a brief description of the problem and your return address and phone number. Guldmann does not assume the risk of loss or damage while in transit, so it is recommended you insure the package.

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## **Guldmann Inc.**
## **Guldmann GmbH**
## **Guldmann Sverige AB**