**CRYSTAL 3D** Aesthetic Self Ligating Brackets



## **Invisible and Fast**



The Only One Aesthetic Self Ligating Bracket developed from Clinical and Tomographyc Evidence

Several thousands of cases treated successfully.







Aesthetic Comparison



Aesthetic Comparison Bracket



The same patient with CRYATAL 3D



Aesthetic Self Ligating Bracket



The same patient with CRYATAL 3D



Metallic Conventional Bracket



The same patient with CRYATAL 3D

**CRYSTAL 3D** Aesthetic Self Ligating Brackets




## **Easy Opening and Closing**

#### Opening:

TURN gently the instrument that comes with the kit **Closing:** 

Easy to close and with security lock

\*All the doors open onto incise/oclusal, avoiding any damage to the soft tissue

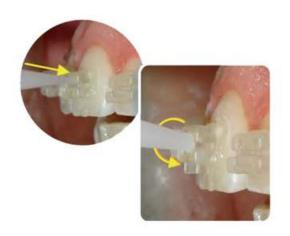

Removal of markings brackets with roller moistened cotton alcohol



Removal bracket with pliers cutting ligature, with simple shift at the junction bracket x resin.



### **CRYSTAL 3D** Aesthetic Self Ligating Brackets

# CRYSTAL 3D SMILES®

#### Initial



0.014"

#### 12 months treatment



0.014×0.025"

# The 8 Clinical Principles of Maxillary Transverse Bioadaptation Philosophy (MTB)

1st:Clinical analysis of the upper second molar
2nd:Initiate with a 0.014" thermoactivated arch wire
3rd:Understand the importance of the STOP positions
4th:Upper Second molar composite Build up's
5th:Bonding of the Lower archu after Transverse Bioadaptation
6th:Light elastic used at night only
7th:Final arch definition with 0.014×0.025" thermoactivated
8th:Maximum of 0.018"ss or 0.014×0.025"ss for sliding mechanics

#### **Aesthetic Self Ligating Brackets Kit**





- Kit 7×7 100% aesthetic,01 (ONE) instrumental-Opening and Closing
- 4 Aesthetic Thermoactivated Arch Wire (0.014" and 0.014×0.025")
- 8 Aesthetic Wins
- Double Sputinik

## **CRYSTAL 3D** Aesthetic Self Ligating Brackets



## Several thousands of cases treated successfully.











12 months