

journey

**UPWALKER
BREEZE**
ROLLATOR

PRODUCT MANUAL



 journey
HEALTH & LIFESTYLE®

www.shopjourney.com

1. Product Overview 3

2. Getting Started 4-6

3. Height Adjustment 7

4. Brake Adjustment 8

5. Storage & Transport 9

6. Technical Specifications 10

7. Maintenance & Warranty..... 11

Do not exceed the weight limit stated — doing so could put the user at risk.

*** For avoidance of doubt if user is 275 lbs then no more than 25 lbs can be carried.**

Maximum Load: 300 lbs

Maximum User Weight: 300 lbs

Getting Started

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product and immediately contact Journey Customer Service.

Intended Use

The Journey UPWalker Breeze rollator has been designed to assist you when walking and / or carrying small items. This rollator is accompanied with a personal accessory bag. It is suggested to only utilize this bag when using or carrying objects. Do not use or combine bags or baskets from other manufacturers. This product can be used indoors and outdoors.

Features

Ergonomic handgrips to reduce pressure

- User friendly loop brakes with lock
- Simple brake cable adjustment
- Soft solid wheels
- Swivel front wheels
- Height adjustable
- Folding locking mechanism
- Cane holder
- Accessory bag for personal items
- Carry bag for easy storage and transport

2. GETTING STARTED

Carefully remove all packaging, including the cardboard tubes covering the knurled tube part of the brake assembly. Check the product thoroughly for damage or obvious faults.



WARNING!

If you suspect the unit to be faulty, please do not use, but contact Journey Customer Service immediately.



2. GETTING STARTED

Our rollator is supplied partially assembled in its folded position. To unfold the rollator follow these instructions.

Step 1:

Remove the rollator out of the packaging and lay flat on the floor.



Step 1:

Unfold front wheels until you hear a click. Once secure, repeat on the other side.


DO NOT USE THE ROLLATOR UNLESS THE WHEEL HAS SECURELY LOCKED INTO PLACE.



Step 3:

The legs of the rollator have a sliding red lock to help secure and unsecure the rollator into position. Pull this sliding red lock up and unfold the leg of the rollator. Once you hear a click, the leg is secure. Repeat to the other side.

DO NOT USE THE ROLLATOR UNLESS THE WHEEL HAS SECURELY LOCKED INTO PLACE.

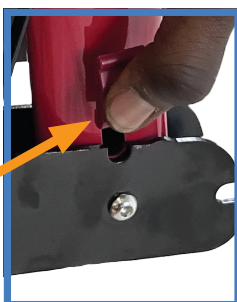

2. GETTING STARTED

Step 4:

When installing the arms of the rollator, note that the brake handles should be pointed towards the rear wheels.

To install the arms of the rollator, press down on the red button on the body of the rollator and insert rollator arm simultaneously. Once desired height is reached, release the red button. Repeat to the other side.



DO NOT USE THE ROLLATOR UNLESS THE WHEEL HAS SECURELY LOCKED INTO PLACE.

Step 5:

When installing the back rest, please note that the narrow end should be pointed up.

Each end of the back rest has a connecting groove that connects it to the rollator. Align each end of the back rest to the corresponding part of the rollator and slide down.



Step 6:


When installing the bag, please note that the front of the bag bears the Journey Logo.

Each end of the travel bag has a half circle clip meant to connect it to the rollator. Connect your travel bag to the front of the rollator, once complete repeat to the other side.




Step 7:

The cane holder has a connecting groove that connects it to the rollator. Connect your cane holder to the rollator.

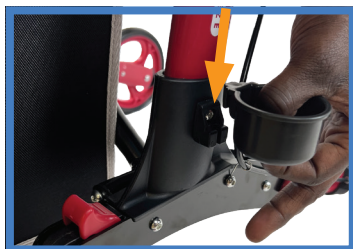

Height Adjustment

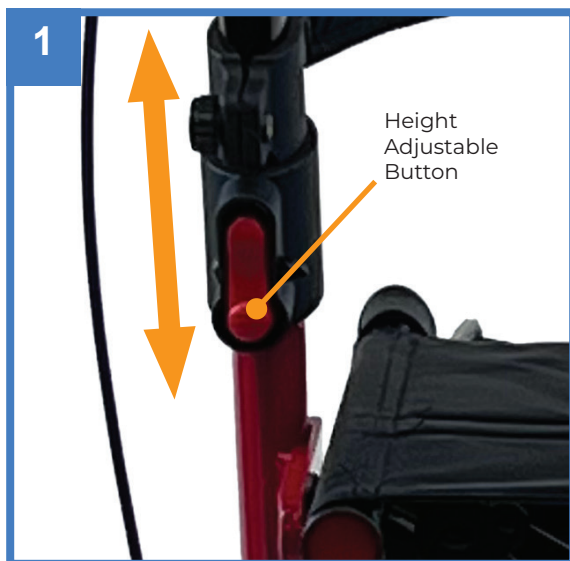

The height of the handgrips can be raised or lowered to suit the user.

To adjust the height of the handgrips, press the height adjustment button (see fig. 1). Raise or lower the handgrips to the desired user height. You will hear a click as the handgrip locks into one of the height adjustment positions (see fig. 1).

ENSURE THAT BOTH HANDGRIPS ARE AT THE SAME HEIGHT BEFORE USE.

DO NOT USE THE ROLLATOR UNLESS THE HANDGRIPS HAVE LOCKED INTO ONE OF THE HEIGHT ADJUSTMENT POSITIONS.

Operating The Brakes




1. To stop, gently apply a gripping pressure to both brakes simultaneously (see fig. 2), this will slow down and stop your rollator.



2. The brakes on your rollator have a park option, push downwards on both brakes simultaneously. You will feel the brake make a very slight click. Your brakes are now in the 'lock' position and 'park' option is engaged.
3. To unlock, simply pull up the brake again, you will hear a slight click as the 'park' option disengages.

Brake Adjustment

Your rollator brake system will require periodic adjustment. To adjust the brakes loosen Nut B and then screw/unscrew Nut A. When adjusted re-tighten Nut B. if further adjustment is required then loosen the Locking Screw and then, using a pair of pliers, pull the cable down by approximately a millimeter. Re-tighten Locking Screw. The brakes can then be adjusted further by loosening Nut B and screwing / unscrewing Nut A as before (see fig. 4).

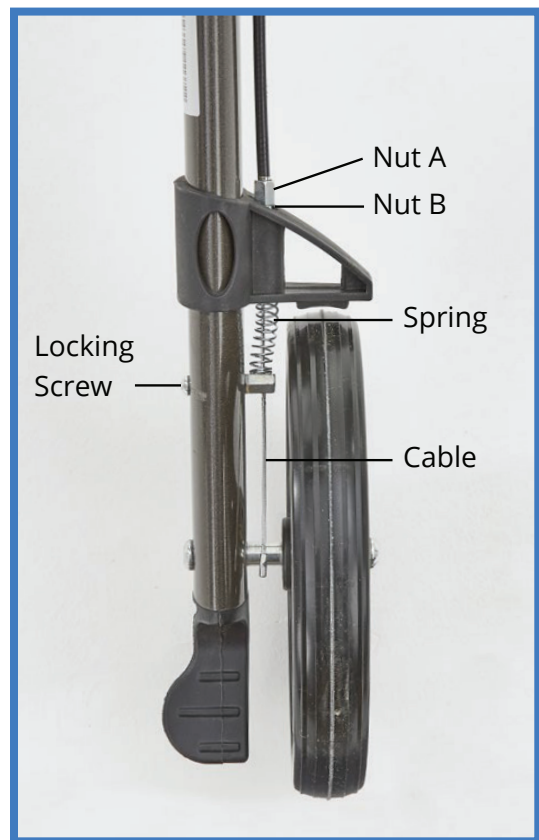

Step 1:

While in upright position, remove arms, storage bag, and backrest.



Step 2:

Fold Breeze legs and wheels.



Step 3:

Place Breeze into carry bag with seat reflectors and front wheels facing upward.



Step 4:

Zip up bag.



Step 5:

Breeze is ready to transport or store.

6. TECHNICAL SPECIFICATIONS

Min Height	31 inches
Max Height	38 inches
Length	26 inches
Width	25.4 inches
Folded Width	10.6 inches
Folded Height	13.4 inches
Folded Length	23.6 inches
Min / Max Seat Height	21 inches
Seat Width	18.1 inches
Seat Depth	9.8 inches
Products (lbs)	18
Rear Wheel Size	199x30mm
Front Wheel Size	199x30mm
Max User Weight (lbs)	300
Frame Material	Aluminum
Suspension	No
Bag Depth	13.4 inches
Bag Width	4 inches
Bag Length	15 inches

MD Mobility Aid

SN SEE PRODUCT LABEL



SEE PRODUCT LABEL



CARE AND MAINTENANCE & YOUR OBLIGATION

1. Only use your rollator for the purpose for which it is intended, which is as an aid to walking. Your rollator is only intended to be used on standard pavements or at home. It should not be used on rough terrain or pot-holed surfaces. The use of your rollator on these surfaces can seriously compromise your safety and may damage your rollator beyond repair.
2. Keep your rollator clean by wiping it down with a damp cloth with a mild detergent. Do not under any circumstances use an abrasive or solvent based cleaner, as this will seriously damage the rollator and void your warranty.
3. Wipe off all surfaces with a dry cloth and use a water dispersing agent such as WD40 on all metal joints.
4. Check that all wheels and the folding mechanism function correctly that all wheels move freely. Pay particular attention to the front wheels, ensuring that they rotate freely and without obstruction and that they are attached securely to the frame.



WARNING!

If you suspect the unit to be faulty, please do not use, but contact Journey Customer Service immediately.

5. Monthly Checks — Check that no components are loose. Check the adjustment to your brakes (fig. 9). Check castors.
6. Do not fit any parts or accessories to your rollator manufactured by another company. To do so will void your warranty and may seriously compromise your safety.
7. We suggest that you check your rollator visually every day.

Maintenance

Please conduct a safety check of the product at regular intervals or if you have any concerns.

Warranty

This product carries a 12-month warranty from the date of purchase against manufacturing defects. Misuse or Damage is not covered.



REGISTER
Your Product and
Get Helpful Info



journey **UPWALKER
BREEZE**
ROLLATOR

If you have any questions, call us at our toll free number:

1-800-958-8324

www.shopjourney.com