• MiniVision 2

Pikaohje

www.kapsys.com

Huomaa

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4. ABOUT SPECIFIC ENERGY ABSORPTION RATE

Your MiniVision2 is provided with a radio transmitter/receiver. It has been designed and manufactured to comply with the radiofrequency (RF) exposure thresholds recommended by the European Union.

This device meets the EU (1999/519/EC) requirements on the limitation of exposure of the General Public to electromagnetic fields by way of health protection. The unit of measurement for European council's recommended limit for mobile telephones is the "Specific Absorption Rate" (SAR). This SAR limit is 2.0 W/kg averaged over 10 g of body tissue. This device meets the requirements of the International Commission on Non-Ionizing Radiation Protection (ICNIRP) and EN 62209-1 / EN 62209-2 / NF EN 50566 European Standards for mobile radio devices.

For body-worm operation, this phone has been tested with the use of accessories that contained no metal and that positioned the handset at a minimum of 1.5 cm from the body. Use of other accessories may not ensure compliance with RF exposure. If you do not use a body-worm accessory and are not holding the phone at the ear, position the handset at a minimum of 1.5 cm from your body when phone is switched on.

During use, the actual SAR level is usually much lower than the maximum value. In general, the closer you are to the base station, the lower the transmission output of your mobile telephone. To reduce exposure to RF energy, use a hands-free accessory or other similar option to keep the device away from your head and your body.

The highest SAR value of this device is listed below:

Head: 1,77 W/kg (averaged over 10 g of body tissue)

Body: 1,85 W/kg (averaged over 10 g of body tissue)

Limbs: 3.02 W/kg (averaged over 10 g of body tissue)

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RoHS COMPLIANCE:

This product is compliant with the 2015/86/EC Directive of European Parliament amending ROHS directive 2011/65/EU, on the use restriction of certain hazardous substances (RoHS) in electrical and electronic equipment and its amendments.

WEEE DIRECTIVE

This symbol on the product or on its packaging indicates that this product must not be disposed of with your other household waste. In accordance with the 2012/19/EU European directive on waste electric and electronic equipment (WEEE), his electrical product must not, under any circumstances, be discarded as unsorted municipal waste. Please dispose of this product by returning it to its point of sale or to your council's local designated collection points for recycling.

CE MARKING

This product complies with the CE marking requirements for use in residential, commercial or light industry installations, and satisfies with the provisions of the R&TTE (1999/5/EC) Directive. For more information about CE marking, you can also visit our website at www.kapsys.com.

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